


Planmeca ProMax Spare Parts Manual

Publication part number 10005021

Released 23 June 2017

PLANMECA

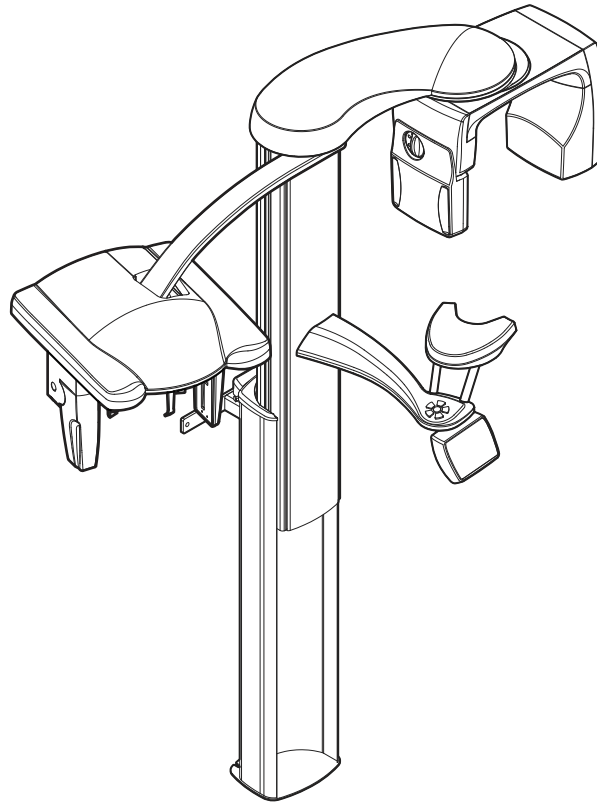

Column	4	SCARA3 Shoulder arm parts 3DPlus 3DMid	36	Head support	68
Exposure switch	5	Shoulder arm cover and elbow arm 3DPlus 3DMid	37	Housings and arm ProCeph Film	69
Column	6	SCARA3 Shoulder arm parts 3DMax	38	ProCeph body and sensor assembly	70
Telescopic column parts	7	Shoulder arm cover and elbow arm 3DMax	39	Primary collimator 3D ProCeph	71
3D EMC cover assembly	8	C-arm	40	ProCeph cables	72
Frankfort laser	9	C-arm 2D 3Ds 3DClassic	41	Movable sensor head connector	73
Column motor 3DPlus 3DMid 3DMax	10	C-arm 3DPlus 3DMid	42	Ceph second primary collimator	74
Input module	11	C-arm 120 kV 3DPlus 3DMid	43	PCBs	75
Fixed input module V2	12	C-arm 3DMax	44	PCBs 2D 3Ds 3DClassic 3DPlus 3DMid	76
Input module cables	13	C-arm 120 kV 3DMax	45	PCBs 3DMax	77
Fixed input module cables V2	14	Laser lights	46	Quality assurance and installation	78
Column upper front and rear panels	15	C-arm cables 2D 3Ds 3DClassic 3DPlus 3DMid	47	Calibration tools 2D	79
Column lower front panels	16	C-arm cables 3DMax	48	Calibration tools 3D and ProFace	80
Column side covers and stickers	17	Covers 2D 3Ds 3DClassic	49	Quality assurance tools	81
ProMax column and shoulder arm cables 3DPlus 3DMid 3DMax	18	Covers 3DPlus 3DMid	50	Mounting accessories	82
ProMax column and shoulder arm cables 3D	19	Covers 3DMax	51	Wall bracket and transportation support bar	83
Patient support arm	20	Sensor head connector	52	Additional options	84
Patient support arm 2D 3Ds 3DClassic	21	Sensor head connector covers	53	Wall exposure switch	85
Patient support arm 3DPlus 3DMid 3DMax	22	Tube head	54	Exposure lamp	86
Patient support table	23	Tube head fixed	55	External cables	87
Layer light	24	Tube head rotating	56	External cables V2	88
Midsagittal plane light	25	Tube head 120 kV rotating 3DPlus 3DMid	57	Imaging support	89
Patient supports and bite registration plate	26	Tube head 120 kV rotating locked 3DMax	58	Sensor head wall holder	90
3D patient support	27	Rotating tube head cables	59	4D Jaw motion	91
Patient support retrofit kits	28	Primary collimator 2D digital	60	Film	92
ProTouch 3 user interface and retrofit kit	29	Primary collimator Cu 3Ds 3DClassic 3DPlus 3DMid	61	Graphic colour LED user interface	93
Patient support arm and user interface cables	30	Primary collimator ProMax 3DMax	62	User interface support	94
Upper arm	31	Primary collimator 120 kV	63	Primary collimator film panoramic	95
SCARA2 Shoulder arm parts	32	Cephalostat	64	Cassette carriage	96
Shoulder arm cover and elbow arm	33	Scanning housings and arm	65	Cassette carriage rotation	97
SCARA3 Linear actuator	34	Scanning mechanism	66	Cassette carriage covers	98
SCARA3 Shoulder arm parts	35	Scanning cables	67	Cassette 15x30cm AEC compatible	99

Secondary collimator.....	100
Fixed secondary collimator.....	101
Fixed shield without AEC.....	102
Fixed shield with AEC.....	103
Moving shield without AEC.....	104
Moving shield with AEC.....	105
Primary collimator film ceph.....	106
Ceph CA body and cassette holder.....	107
Ceph CA laser light assembly.....	108
Ceph cassettes.....	109
PCBs Film.....	110
Column and shoulder arm cables Film.....	111
C-arm cables Film.....	112
Cephalostat cables Film.....	113
Not in production	114
Column 2D 3Ds 3DClassic.....	115
Column motor 2D 3Ds 3DClassic.....	116
3D patient support.....	117
Soft patient support.....	118
Patient support table.....	119
SCARA2 Shoulder arm parts 1.....	120
SCARA2 Shoulder arm parts 2.....	121
Primary collimator fixed.....	122
Fixed sensor head connector.....	123
Fixed cephalostat sensor head connector.....	124
ProTouch 2 user interface and retrofit kit.....	125


Planmeca pursues a policy of continual product development. Although every effort is made to produce up-to-date product documentation this publication should not be regarded as an infallible guide to current specifications. We reserve the right to make changes without prior notice.

COPYRIGHT PLANMECA HELSINKI FINLAND 2017




Planmeca ProMax Column

PLANMECA

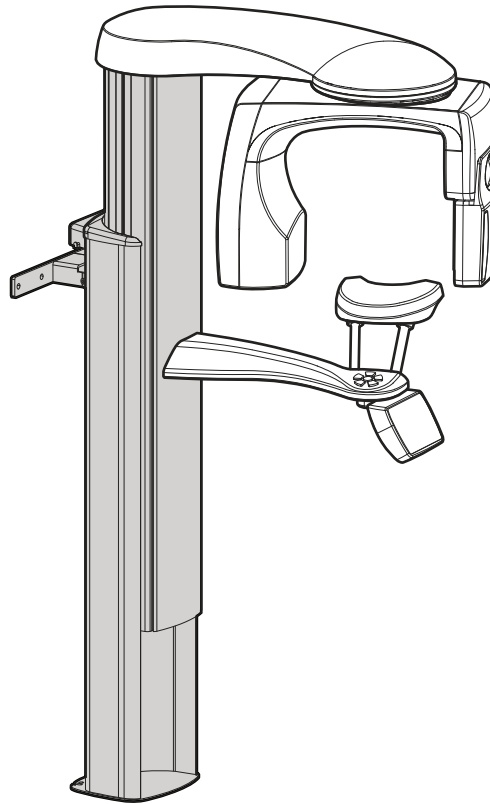


Exposure switch

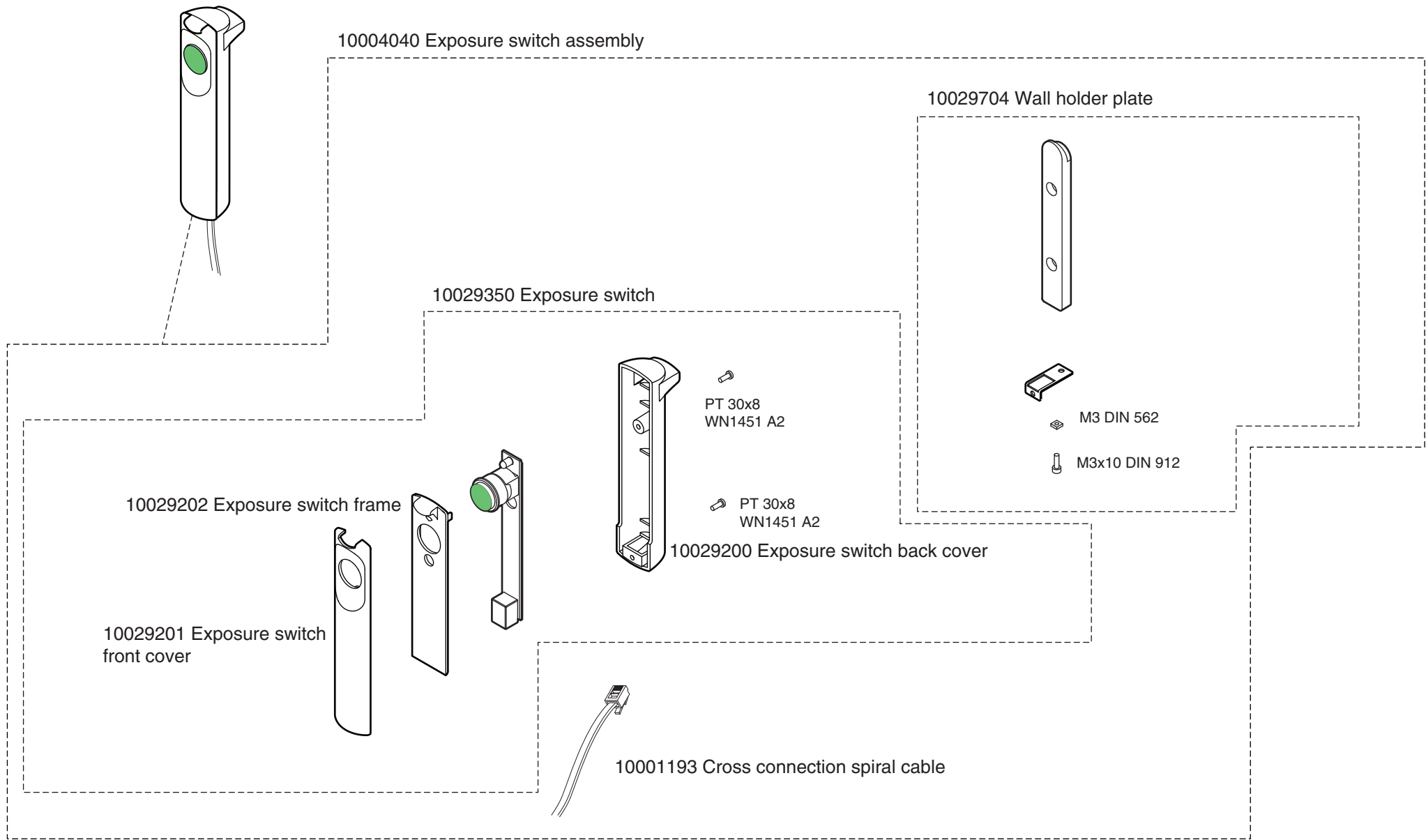


Column


NOTE! Used also in 2D and 3D models with serial number larger than TPX357567

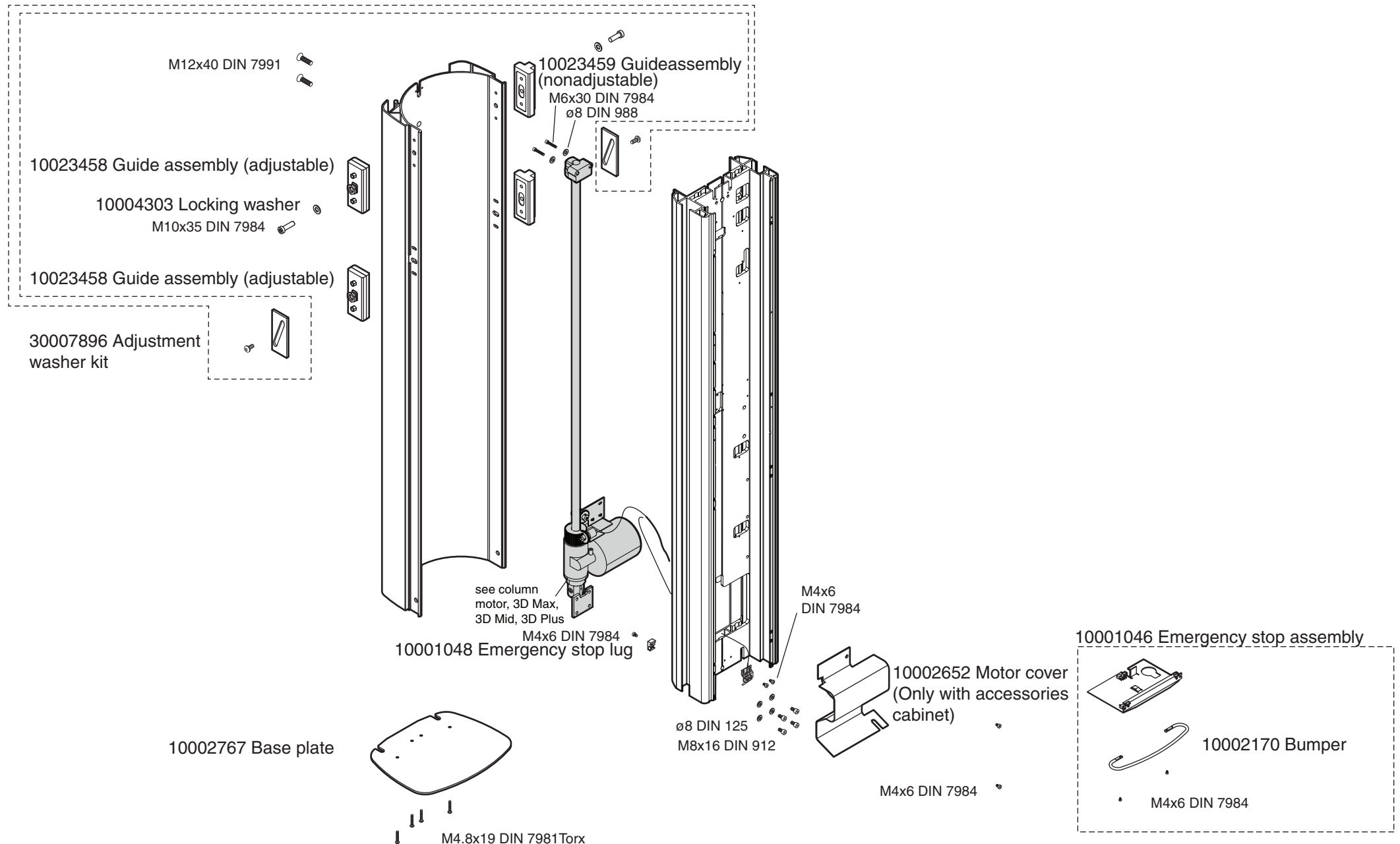


Telescopic column parts

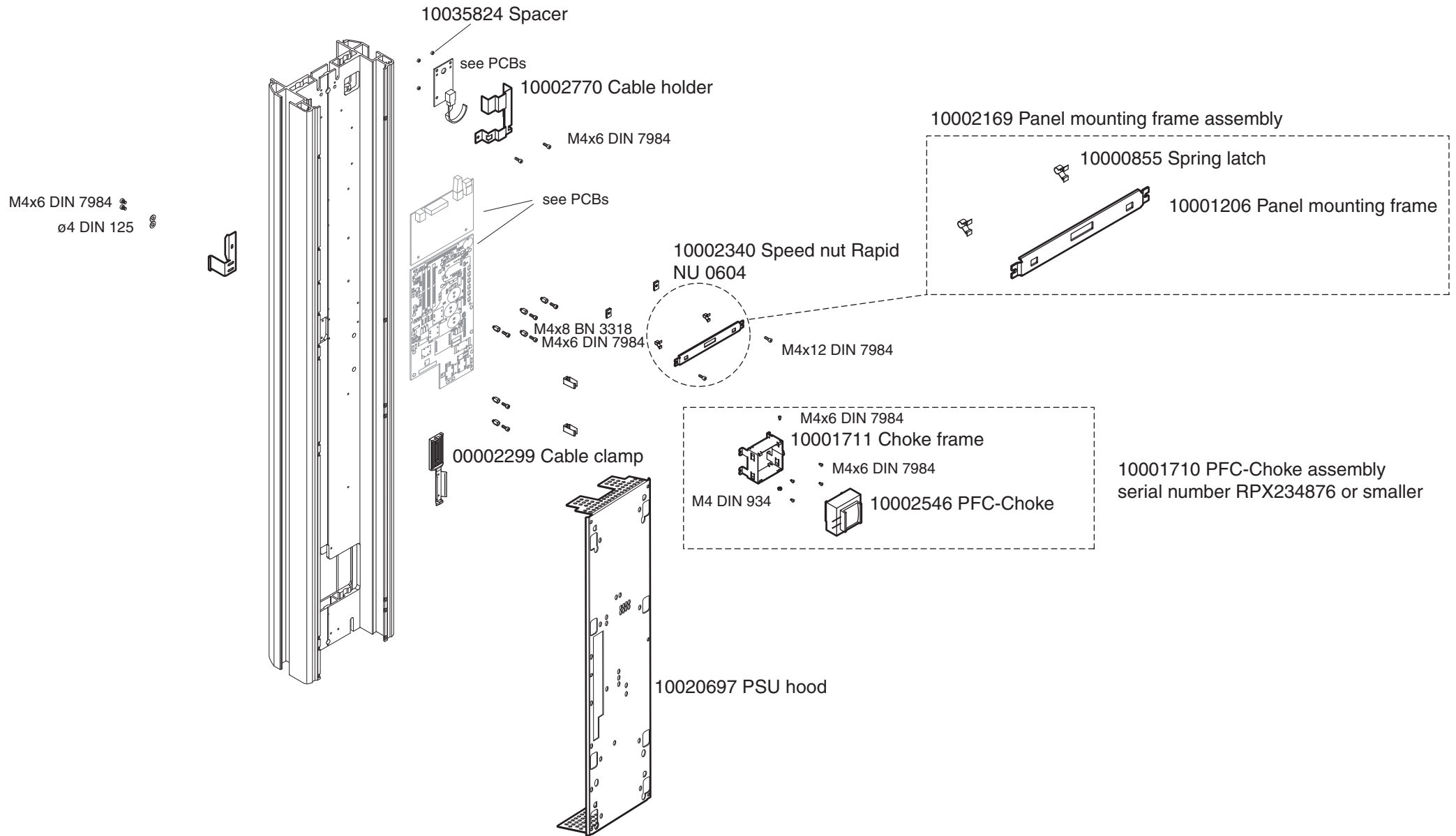


3D EMC cover assembly

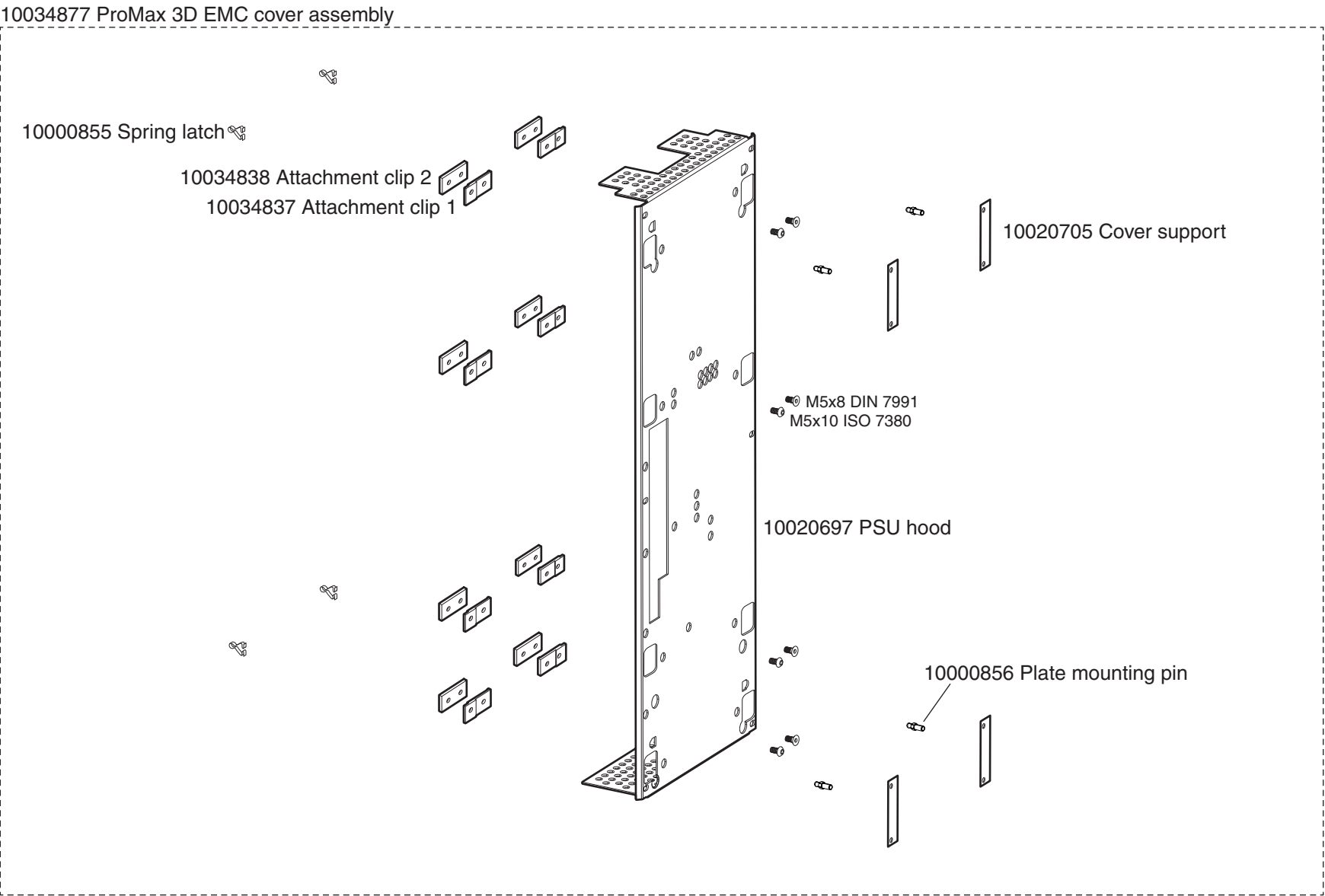

Frankfort laser

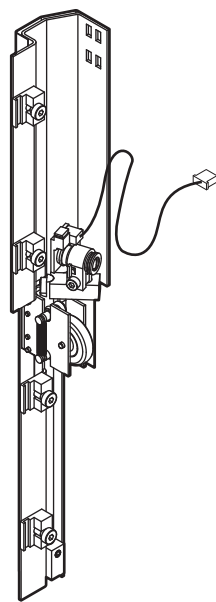

10001347 Frankfort laser

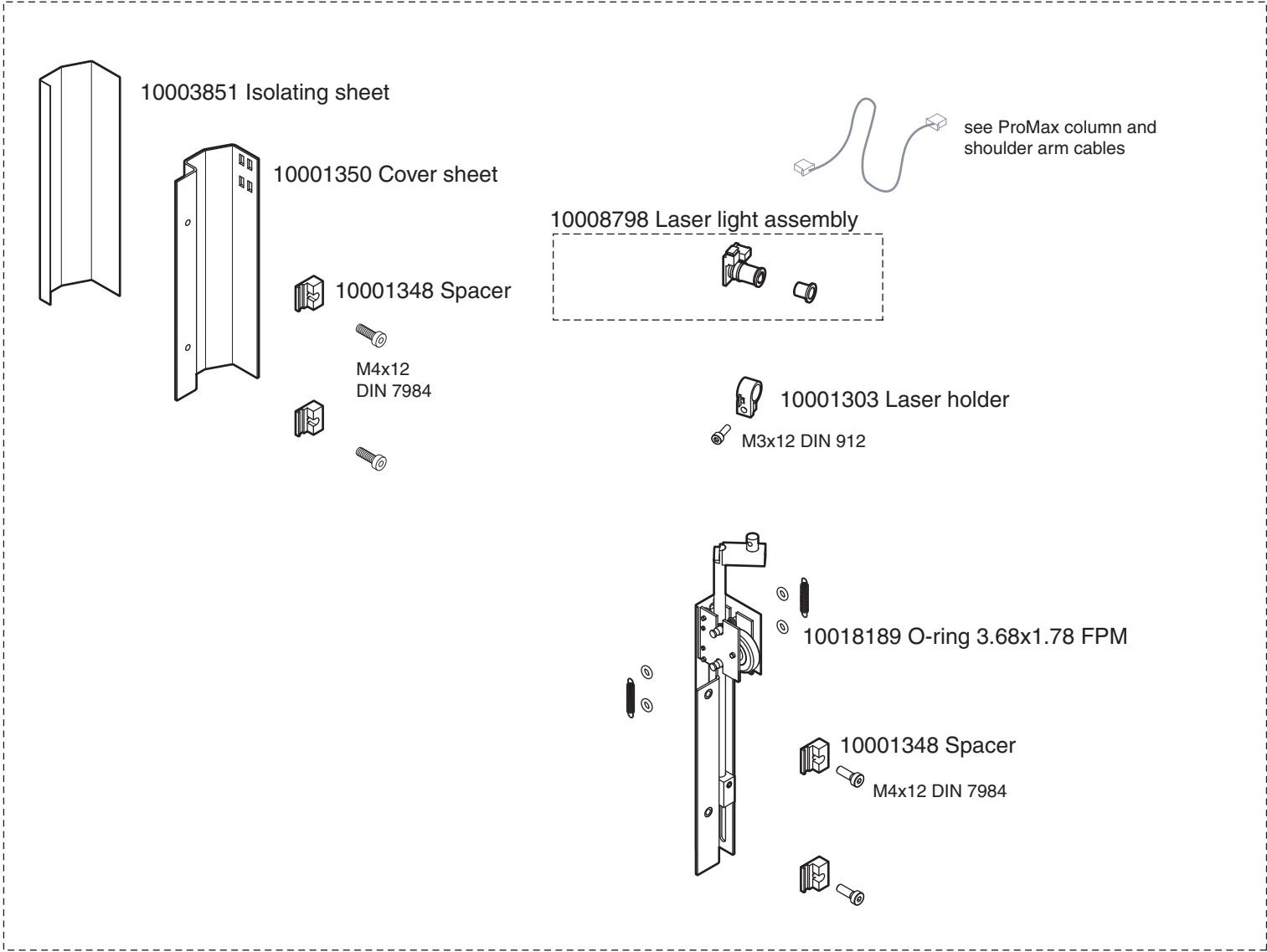

10003851 Isolating sheet

10001350 Cover sheet

10001348 Spacer

M4x12
DIN 7984

10008798 Laser light assembly

see ProMax column and
shoulder arm cables

10001303 Laser holder

M3x12 DIN 912


10018189 O-ring 3.68x1.78 FPM

10001348 Spacer

M4x12 DIN 7984

Column motor, 3D Plus, 3D Mid, 3D Max



10021255 Column motor assembly (used in 3D and 3D s units from ser. no TPX357567 and in 2D and film units from ser. no RPX 243163b)

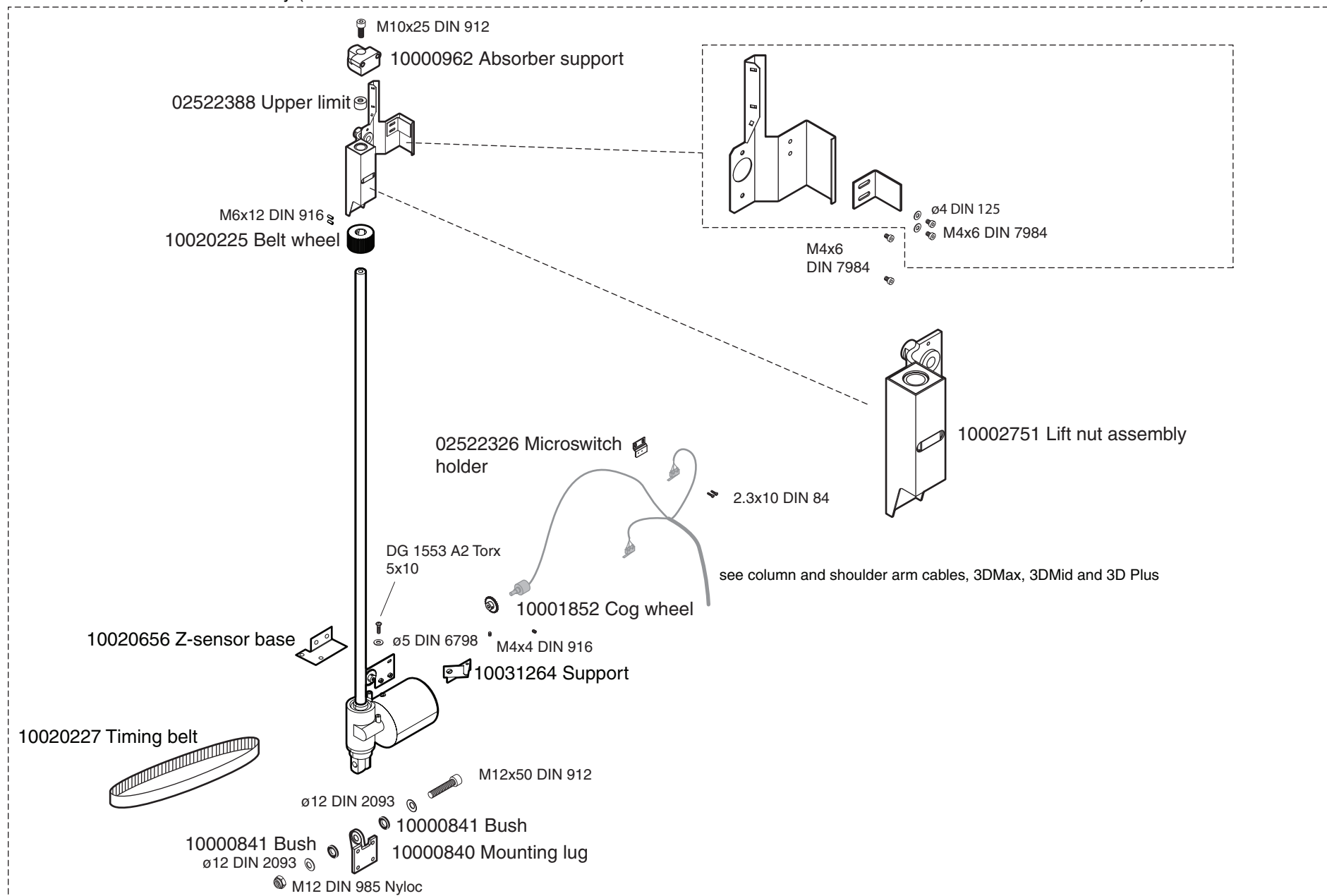

Input module




M4.8x19 DIN 7981 torx

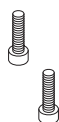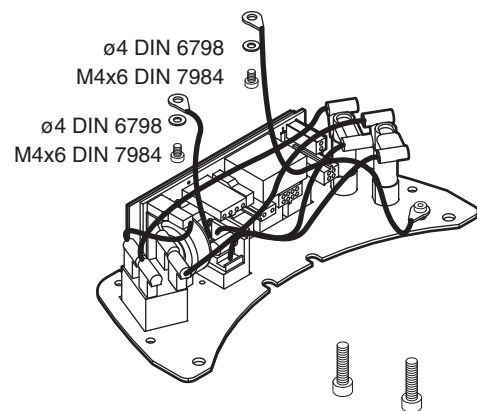


M4x16 DIN 912


M3x5 ISO 7380

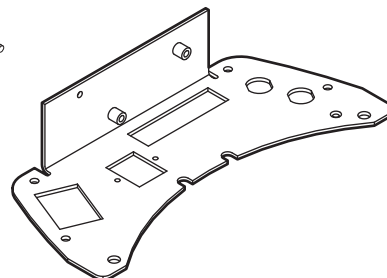

10012031 Locking plate

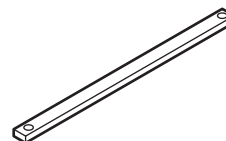


M4x6 DIN 7984
ø4 DIN 6798

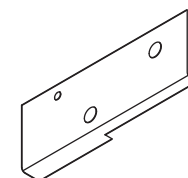

10003183 Input protective sheet

10003167 PCB holder

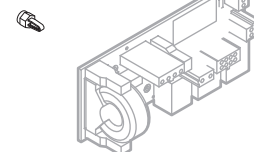


see PCBs



M4x6 DIN 7984

10001146 Connector, male STASE13

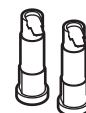

10031808 Bel Stewart 331010-1PC-LT,
RJ-45 Dummy Plug, no contacts

10000352 Fuse holder 0031.1693 (Schurter)

M3x5 ISO 7380



10001147 Locking frame




EU: 10007706 Fuse 8AT 6.3x32, type 195100 ELU
USA/Japan: 10007707 Fuse 16AT 6.3x32, type 195100 ELU



00431780 Fuse holder cover

10034385 Mains switch

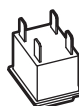


ProMax x-ray unit


COLUMN

Fixed input module, ProMax V2

NOTE! Planmeca ProMax including new power supply cable and input module has a letter 'V' in the serial number (RPXV..., TPXV..., NTPV...).

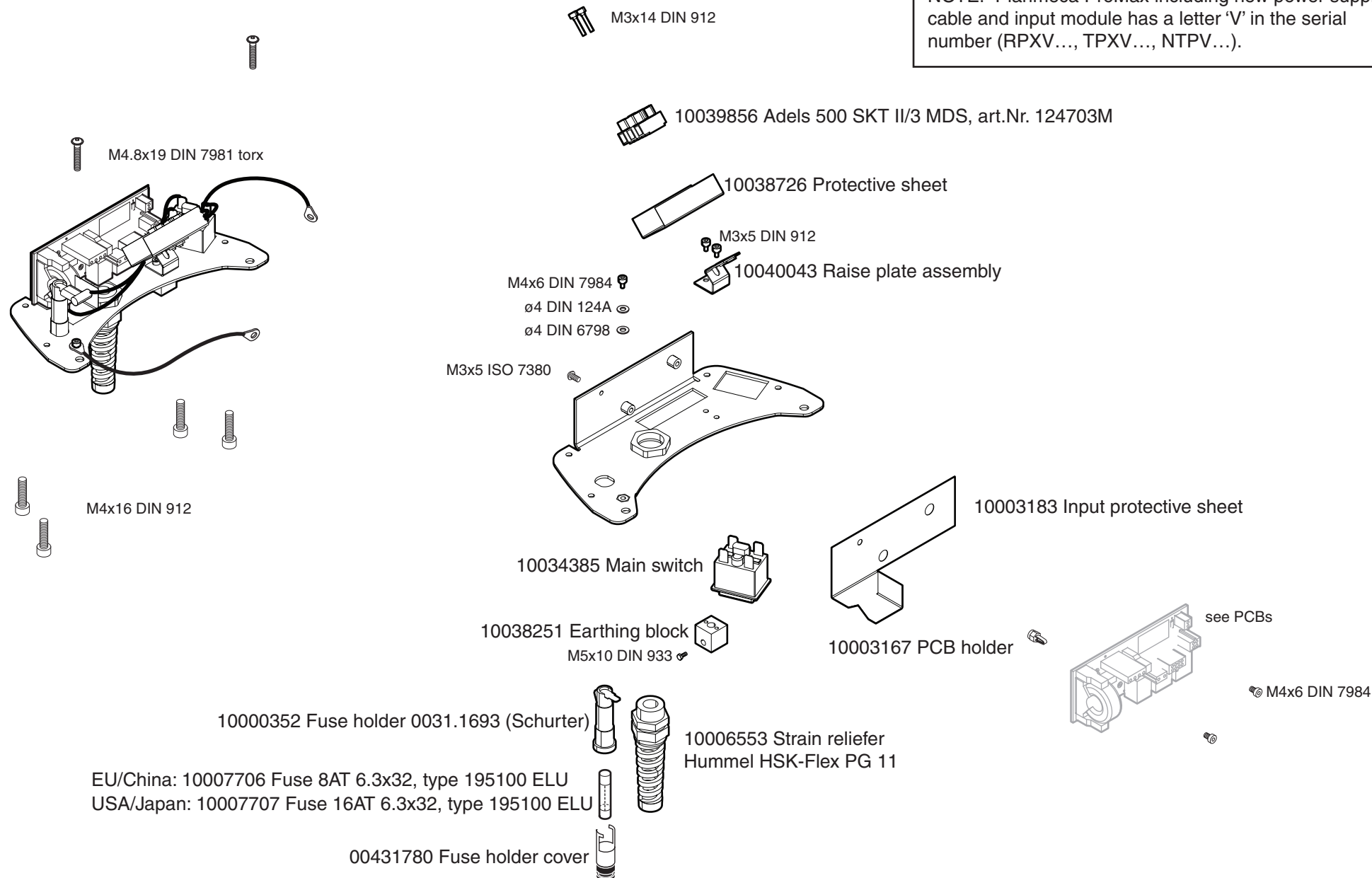


Input module cables

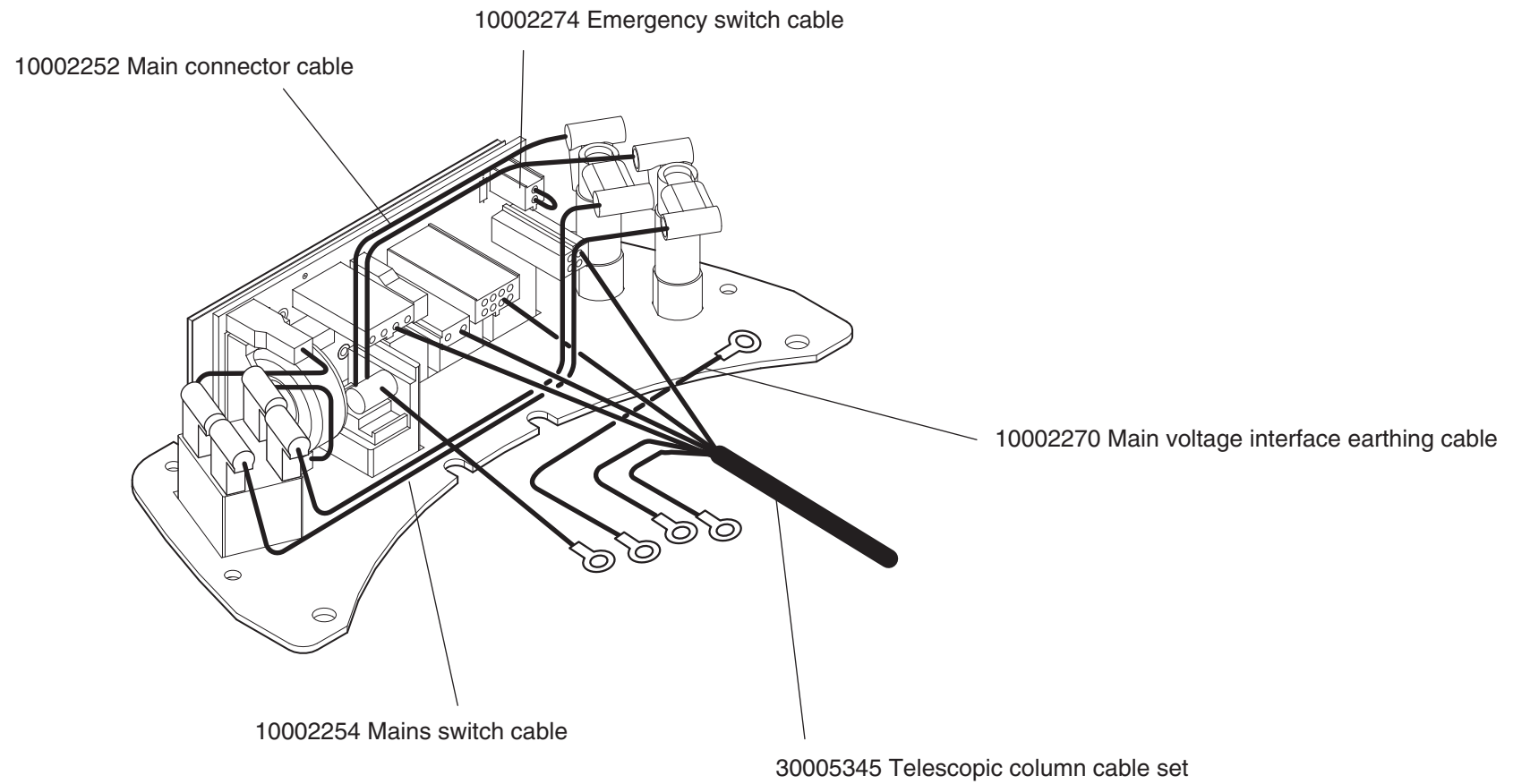


Fixed input module cables


NOTE! Planmeca ProMax including new power supply cable and input module has a letter 'V' in the serial number (RPXV..., TPXV..., NTPV...).

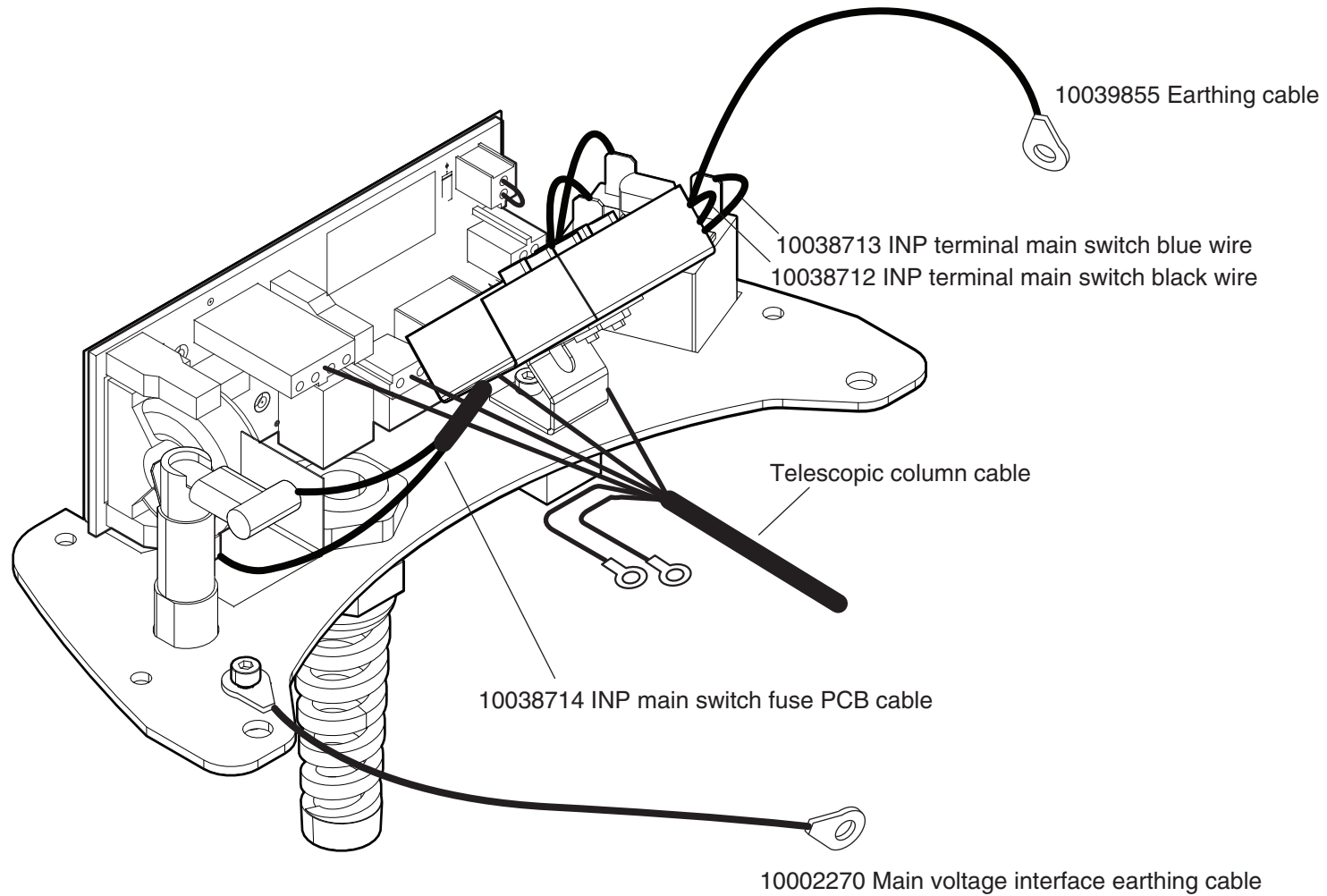

Column upper front and rear panels

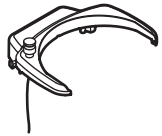

10004750 Column top cover with emergency switch

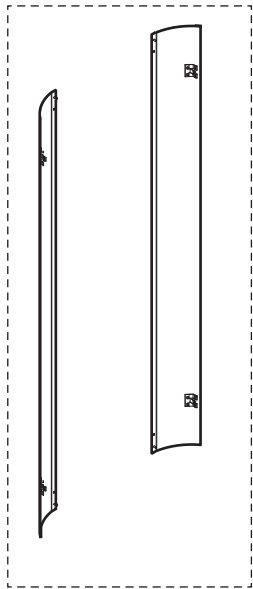


10039816 Promax rear panel assembly pair




10031836 ProMax 3D ceph front cover (Models with ser. number higher than TPX357567) (Suitable for use with ProMax 3D EMC-cover 10034877, see *ProMax 3D EMC-cover assembly, retrofit only*)



10031835 ProMax 3D front cover (Models with ser. number higher than TPX357567) (Suitable for use with ProMax 3D EMC-cover 10034877, see *ProMax 3D EMC-cover assembly, retrofit only*)


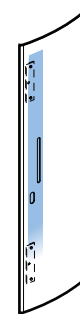


10027169 Upper front panel assembly, 3D Mid & 3D Plus with ceph (Suitable for use with ProMax 3D EMC-cover 10034877, see *ProMax 3D EMC-cover assembly, retrofit only*)

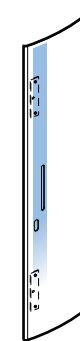


10021709 Upper front panel assembly, 3D Max, 3D Mid, 3D Plus (Suitable for use with ProMax 3D EMC-cover 10034877, see *ProMax 3D EMC-cover assembly, retrofit only*)



Column lower front panels

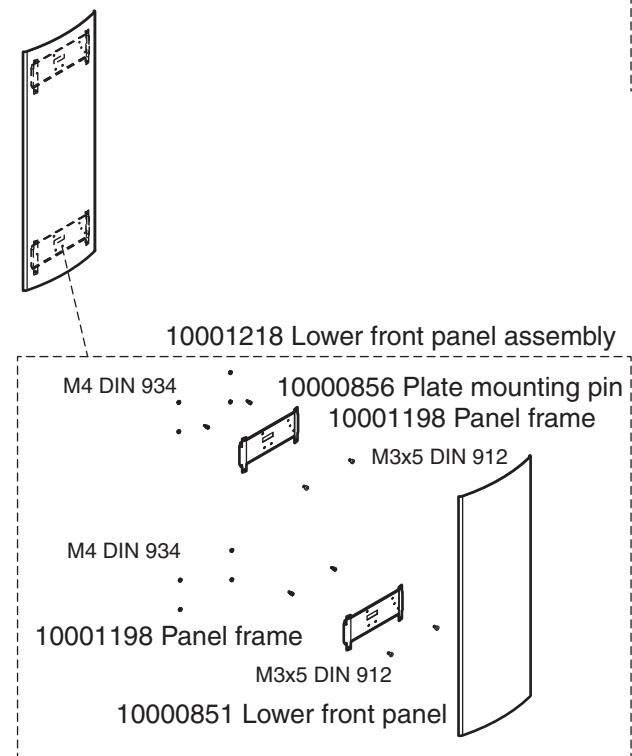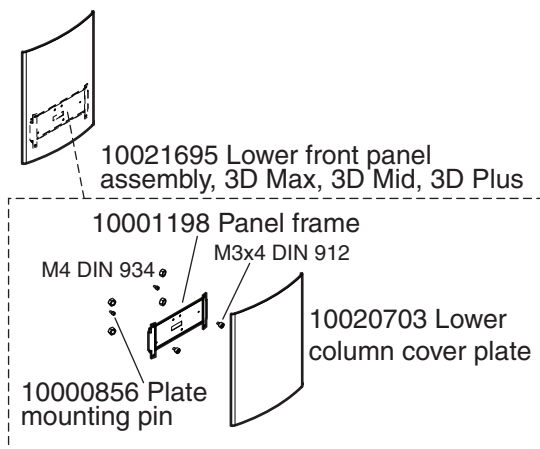


10008230 Door mounting accessory kit

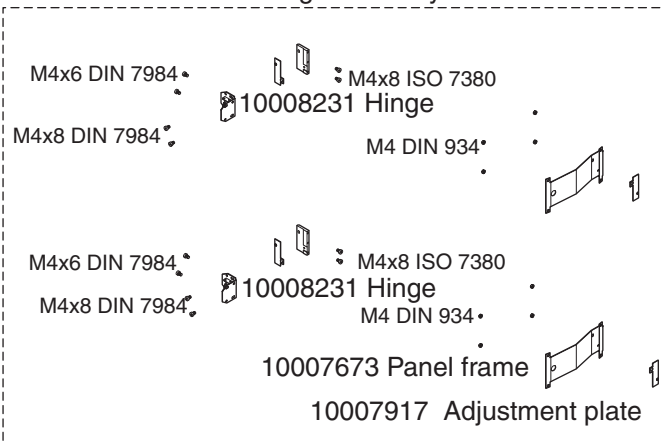

10001766 Door assembly

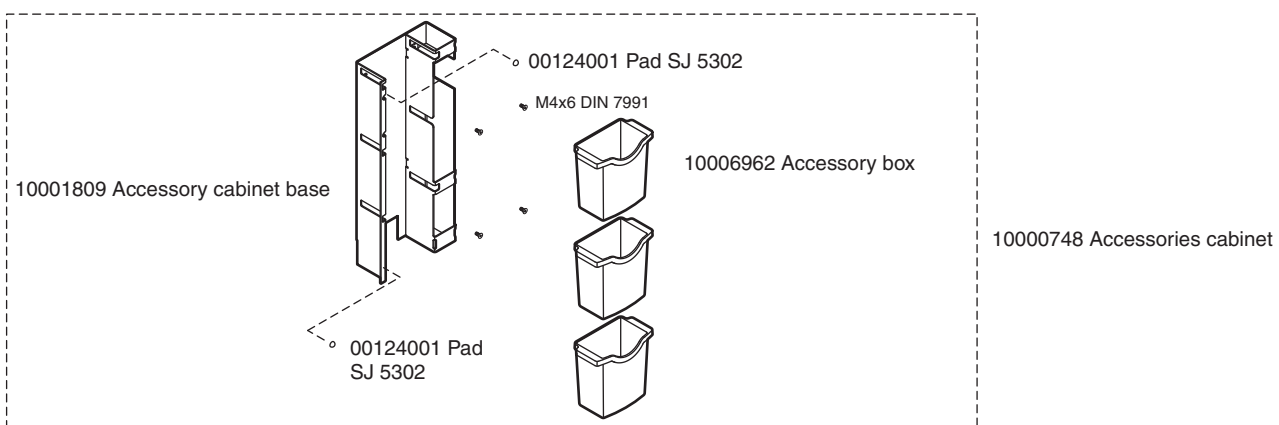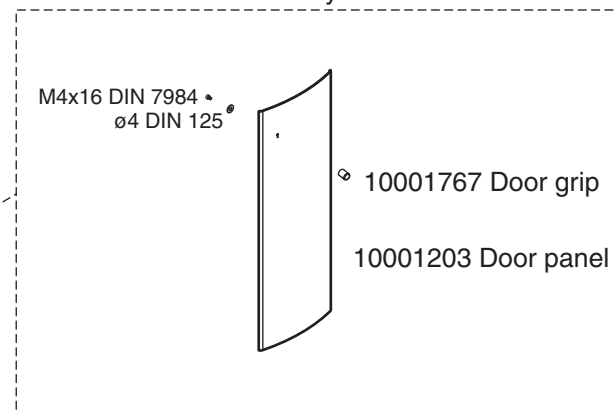

Column side covers and stickers

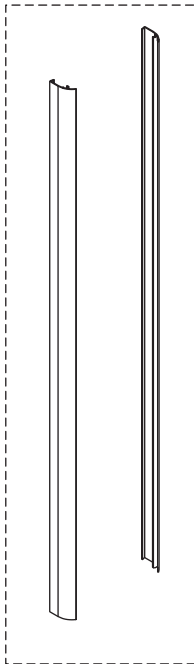


- 10004096 Column side cover set, blue
- 10033106 Column side cover set, sun
- 10033107 Column side cover set, pink
- 10033108 Column side cover set, lime
- 10033109 Column side cover set, steel


- 
- 10003647 Column sticker, blue
 - 10033079 Column sticker, sun
 - 10033080 Column sticker, pink
 - 10033081 Column sticker, lime
 - 10033082 Column sticker, steel

- 
- 10033074 Tube head sticker, sun
 - 10033075 Tube head sticker, pink
 - 10033076 Tube head sticker, lime
 - 10033077 Tube head sticker, steel

Column and shoulder arm cables, 3D Plus, 3D Mid, 3D Max

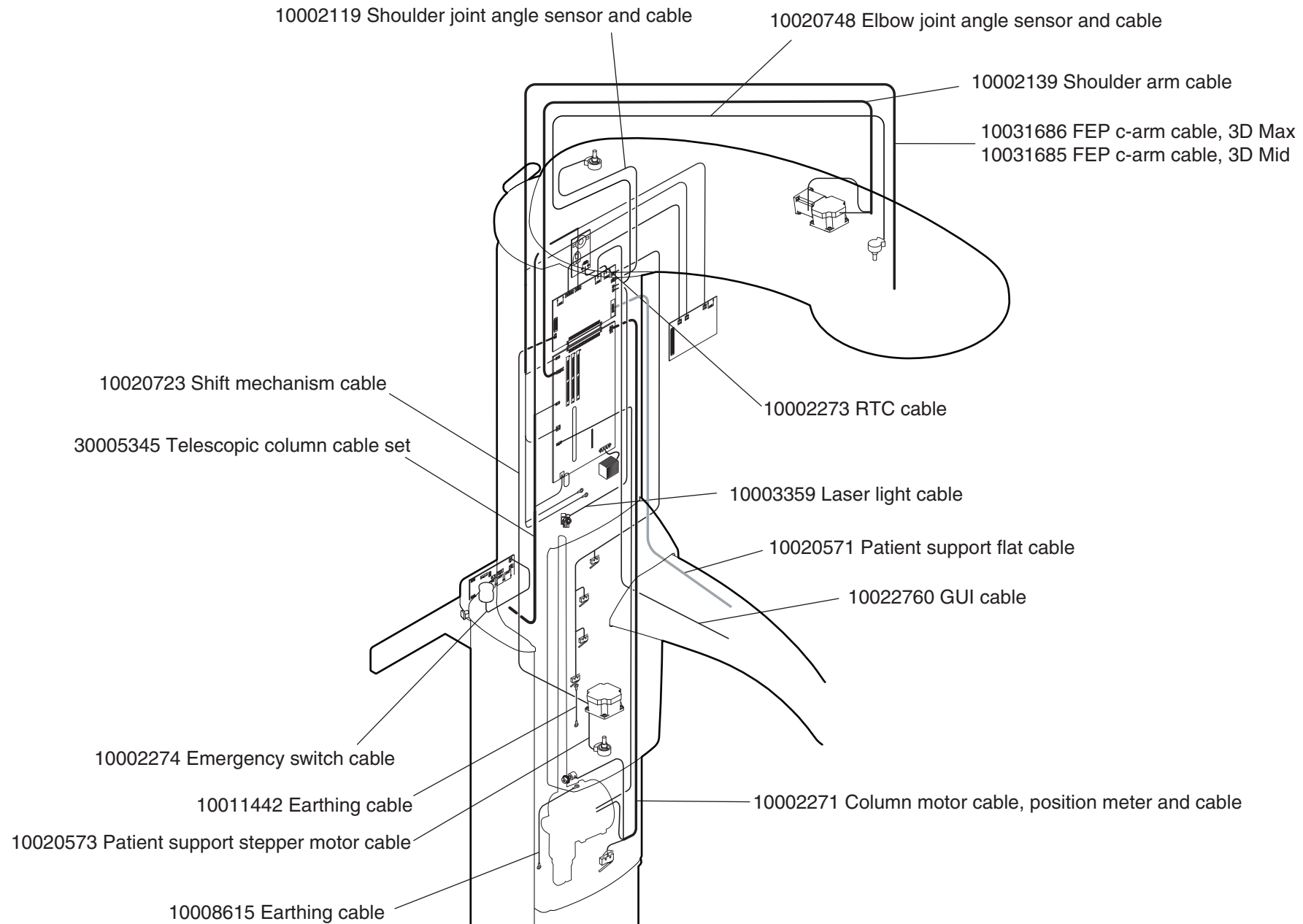


Column and shoulder arm cables, 3D

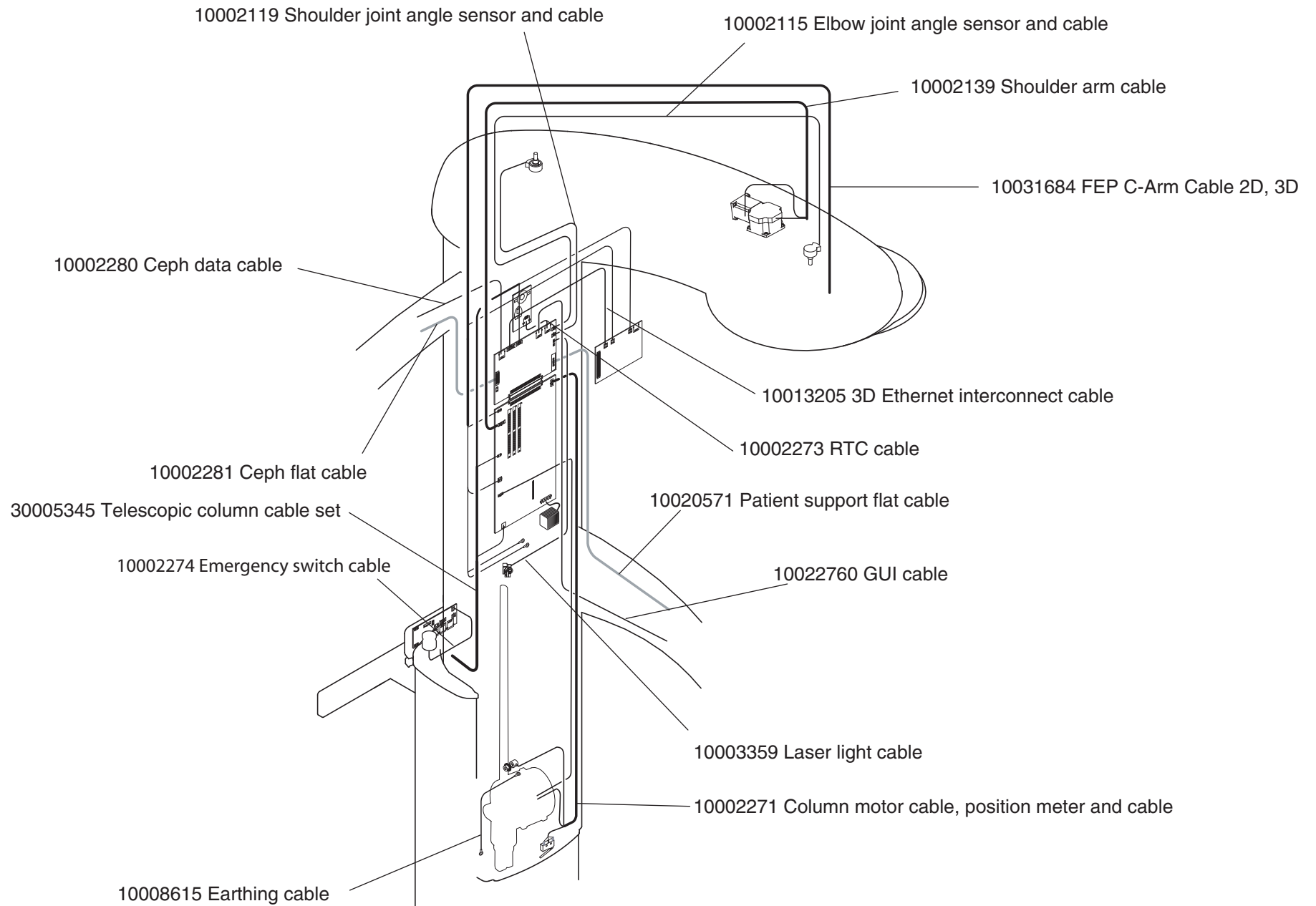


Planmeca ProMax

Patient support arm

PLANMECA

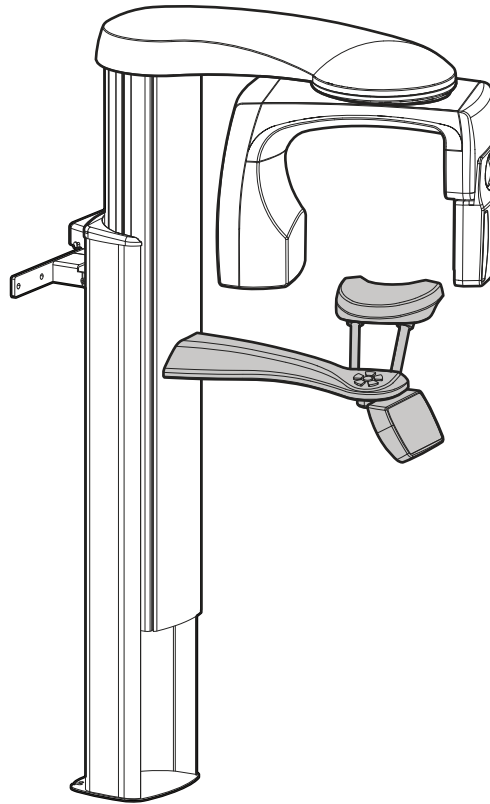


Patient support arm, 2D, 3D s, 3D Classic

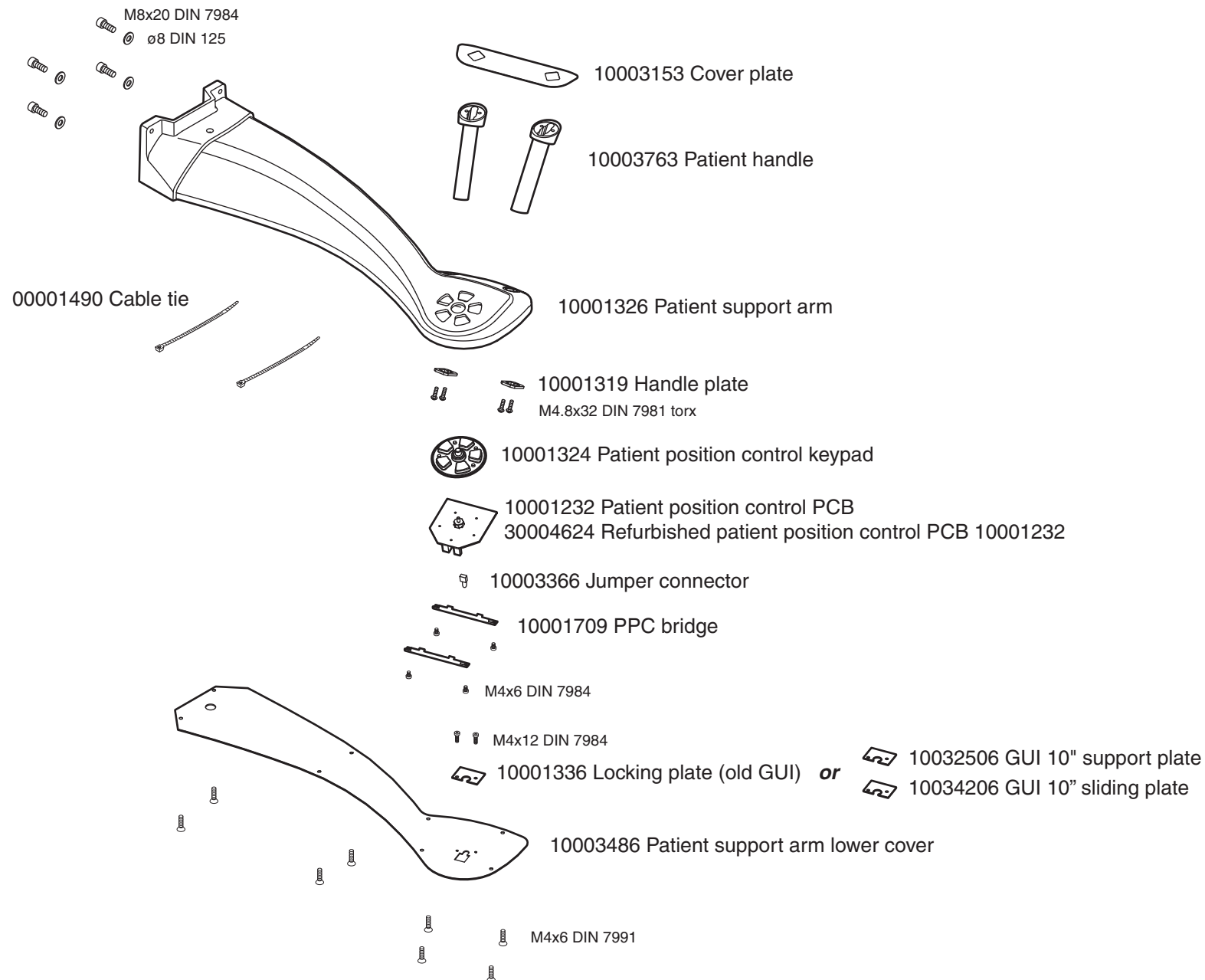


Patient support arm, 3D Plus, 3D Mid, 3D Max

10021254 Patient support moving mechanism

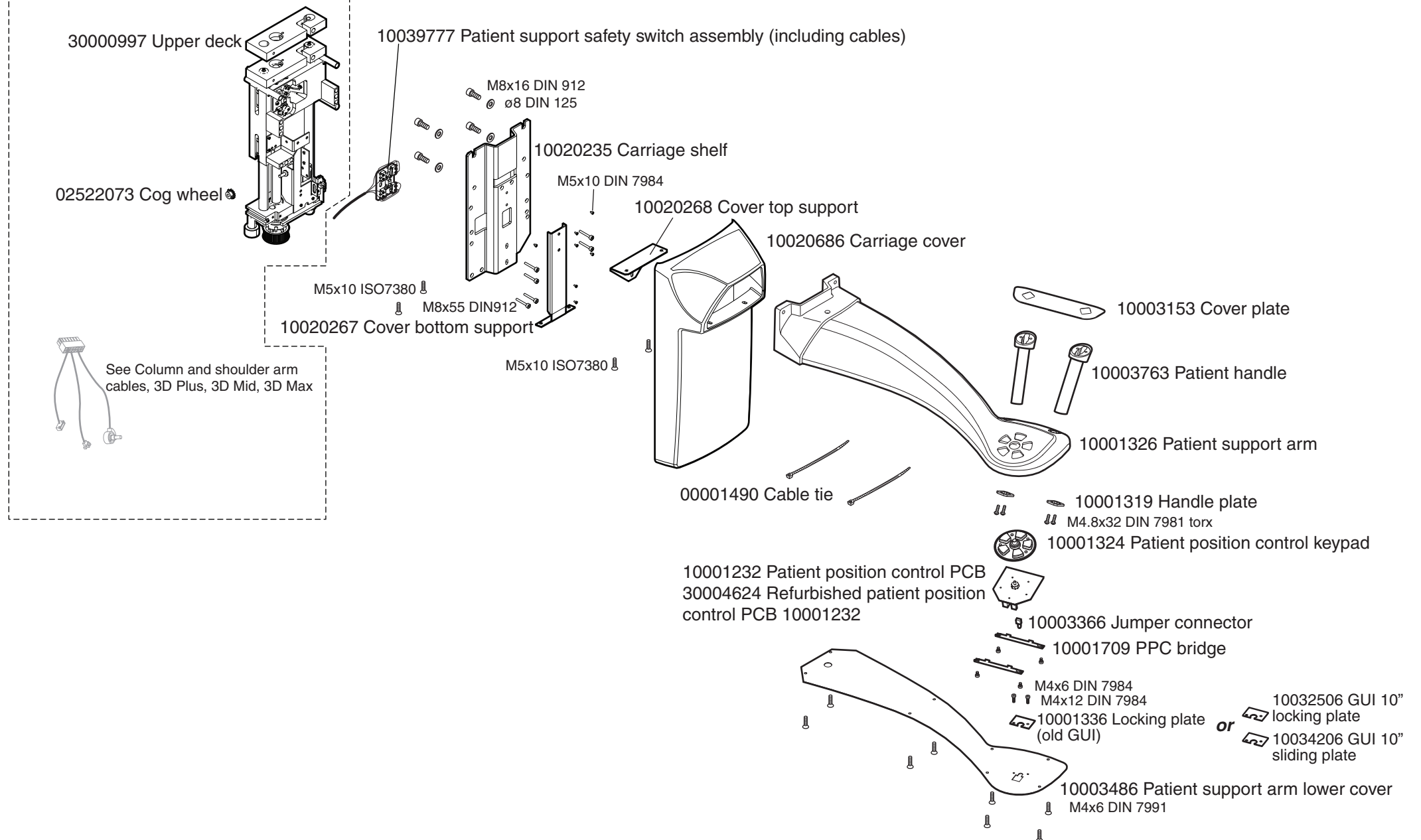


Patient support table

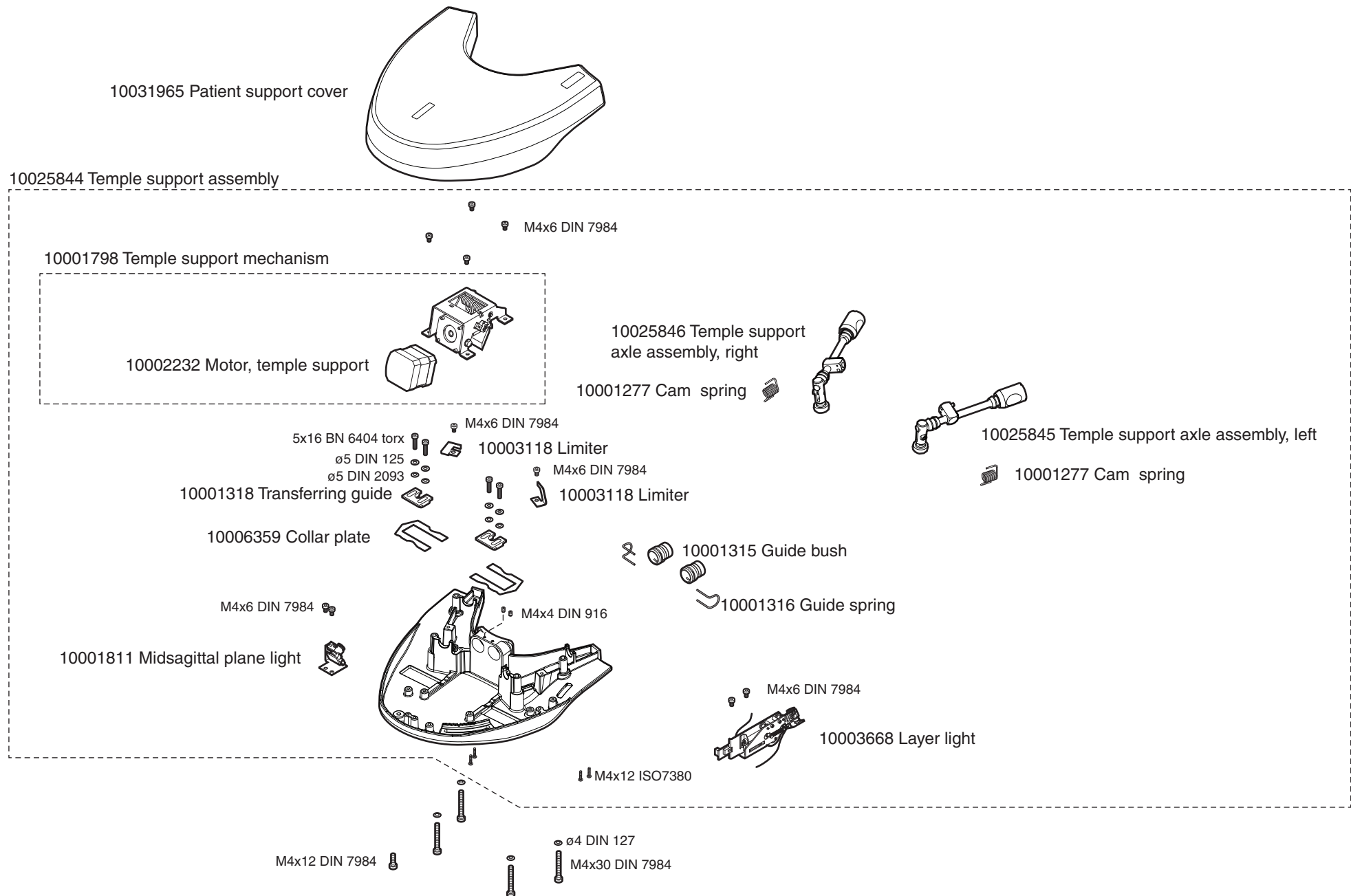


Layer light

10003668 Layer light

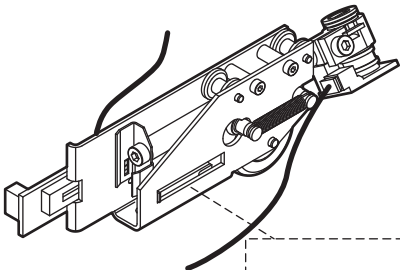


10008798 Laser light assembly

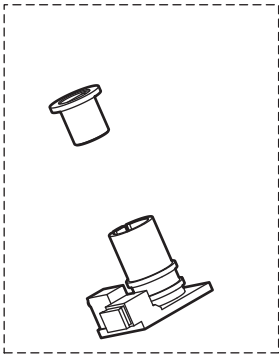


10001303 Laser holder



M3x10 DIN 912
ø3 DIN 6798

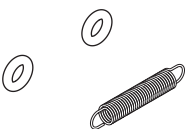


10003597 Position sensor potentiometer

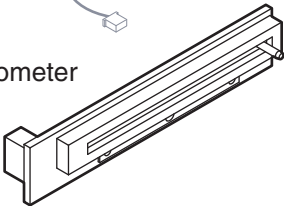

00001490 Cable tie




10018189 O-ring 3.68x1.78 FPM

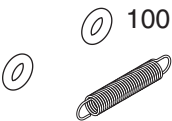


Midsagittal plane light

10001811 Midsagittal plane light

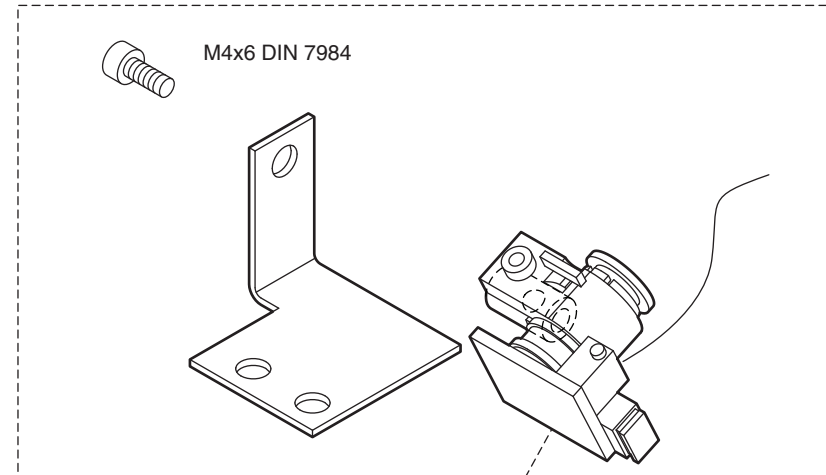


10008798 Laser light assembly

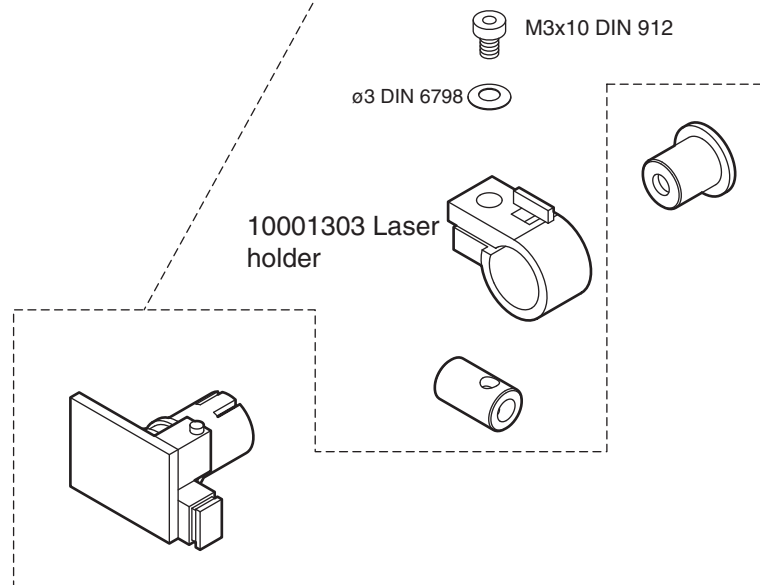



Patient supports and bite registration plate

10004051 Patient support sets


10004280 Bite piece set, 10 pcs

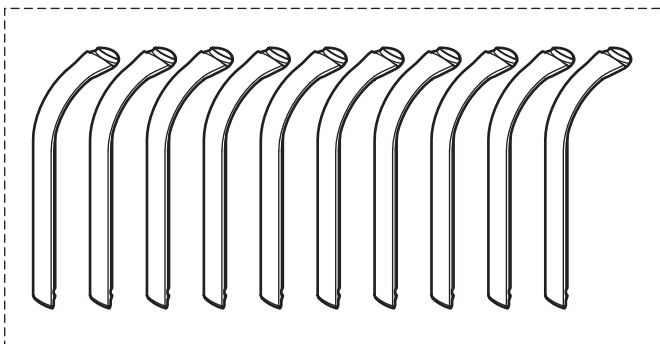


10004764 Temple support pair

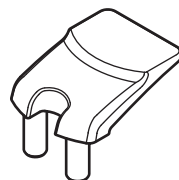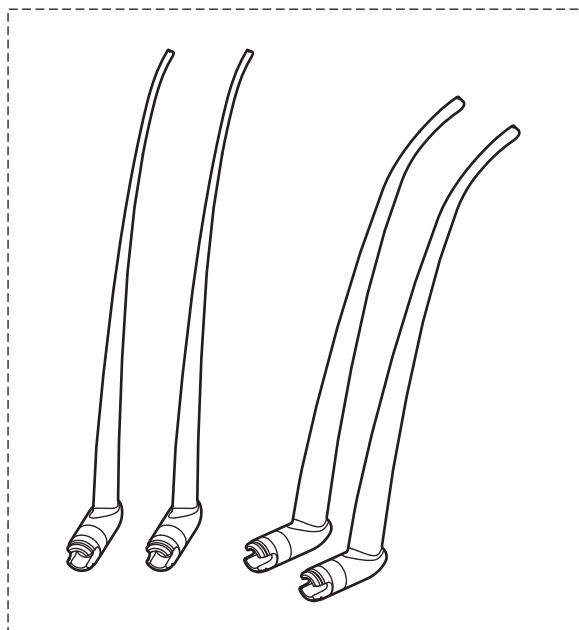


10001290 Chin rest

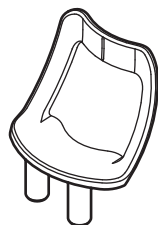

10001292 Chin support

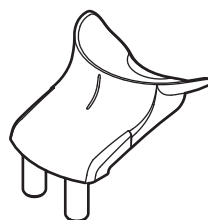


10001291 Chin cup

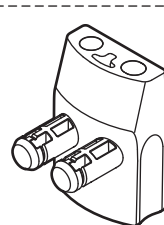

10001293 Adapter




10005182 Bite registration plate set

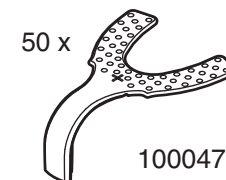


50 x

10004765 impression plate

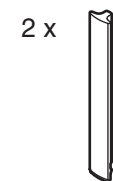

2 x


10005192 Impression plate's rod

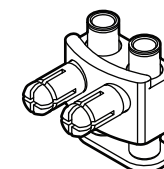

10029530 3D adapter, adjustable




10007583 Chinrest spacer

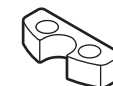

10031692 Chin support heightening piece, 3D Max ProFace


3D patient support

30005390 3D imaging patient support set

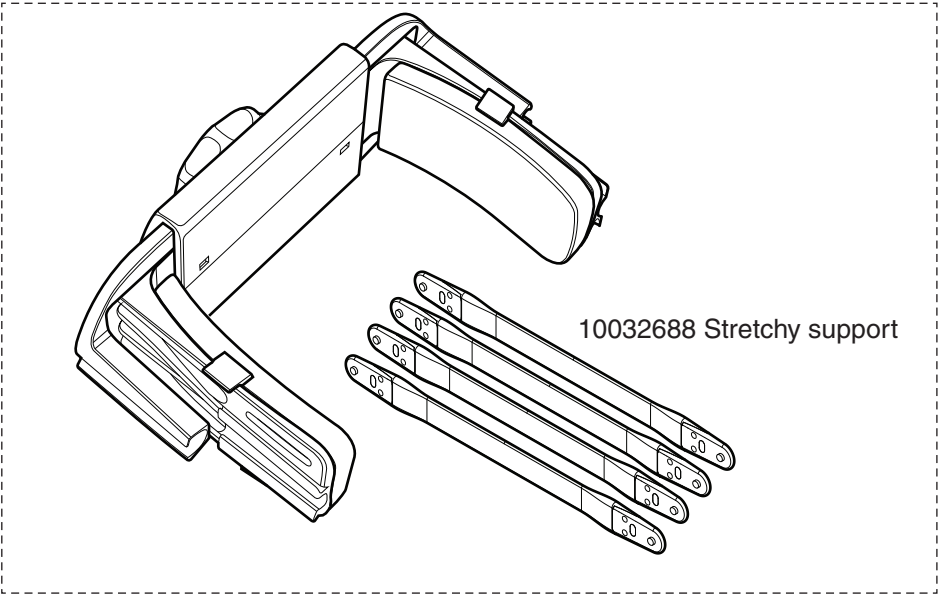


10032688 Stretchy support

10028030 3D patient support rod, right

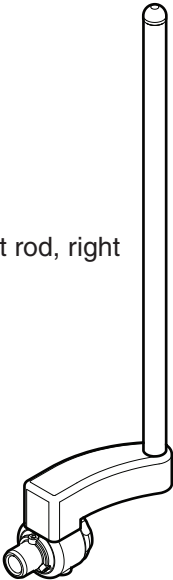


10028031 3D patient support rod, left

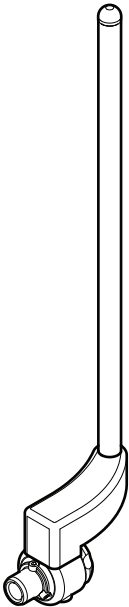

Patient support retrofit kits


10034355 Patient support upgrade kit (for upgrading a “mask type” support)

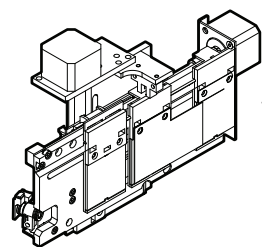

10023718 Primary collimator Va Cu



10027947 Tube head laser assembly,
3D Mid, 3D Plus

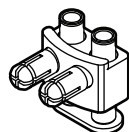


10029530 3D adapter,
adjustable



10001291 Chin cup (2 pcs.)

10039227 ProMax tube head covers

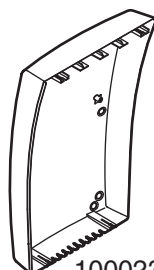


10002340 Speed nut
Rapid NU 0604

M4x6 DIN 7984


M4x12 DIN 7984

10002497 Base spring

M4x8 DIN 7984

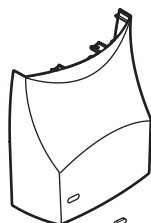


30005390 3D imaging patient support set

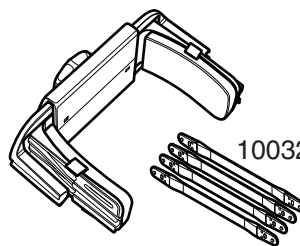

10032688 Stretchy support

10031965 Patient support cover

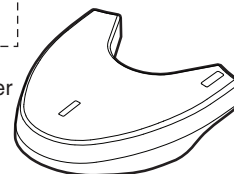

10025844 Temple support assembly

10001798 Temple support mechanism

M4x6 DIN 7984

10002232 Motor, temple support

10025846 Temple support
axle assembly, right

5x16 BN 6404 torx
ø5 DIN 125
ø5 DIN 2093

M4x6 DIN 7984

10003118 Limiter

M4x6 DIN 7984

10003118 Limiter

10001318 Transferring guide

10006359 Collar plate

10001315 Guide bush

10001316 Guide spring

M4x6 DIN 7984

10001811 Midsagittal
plane light

M4x4 DIN 916

M4x6 DIN 7984


M4x12 ISO7380

10003668 Layer light


10025845 Temple support axle
assembly, left

10001277 Cam spring

10028030 3D patient support rod, right

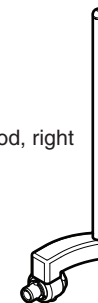

10028031 3D patient support rod, left

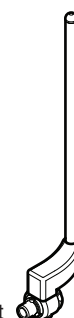

ProTouch 3 user interface and retrofit kit

NOTE! For information about Planmeca ProMax ProTouch user interface retrofit, see technical bulletin PX-128A at Planmeca dealer support

30001449 ProTouch 3.0 retrofit

30001645 ProTouch 3.0 spare part kit

10037959 ProTouch installation accessory bag



10001336 Locking plate




10034206 GUI 10" Sliding plate



10032506 GUI 10" support plate



10030354 Sliding bushing

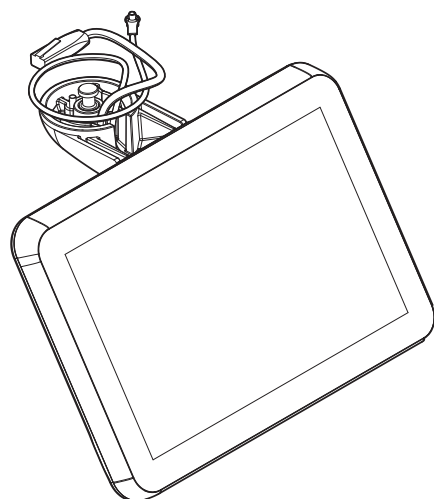

M4x10 DIN912



ø4,3/d2 9 DIN125




M4 DIN934




00002269 Cable anchor



10026808 Cat6 feed-through coupler FM 692

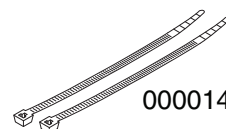

00001490 Cable tie 2,5mm x 100mm, PLT1M-M

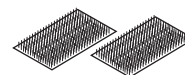

00012108 Sticker 25mmx15mm +0.5
(Dual lock SJ3560)




10029312 Ethernet interface PCB, 4D

30004628 Refurbished 4D Ethernet Interface PCB 10029312



10013205 3D ethernet interface PCB cable


Patient support arm and user interface cables

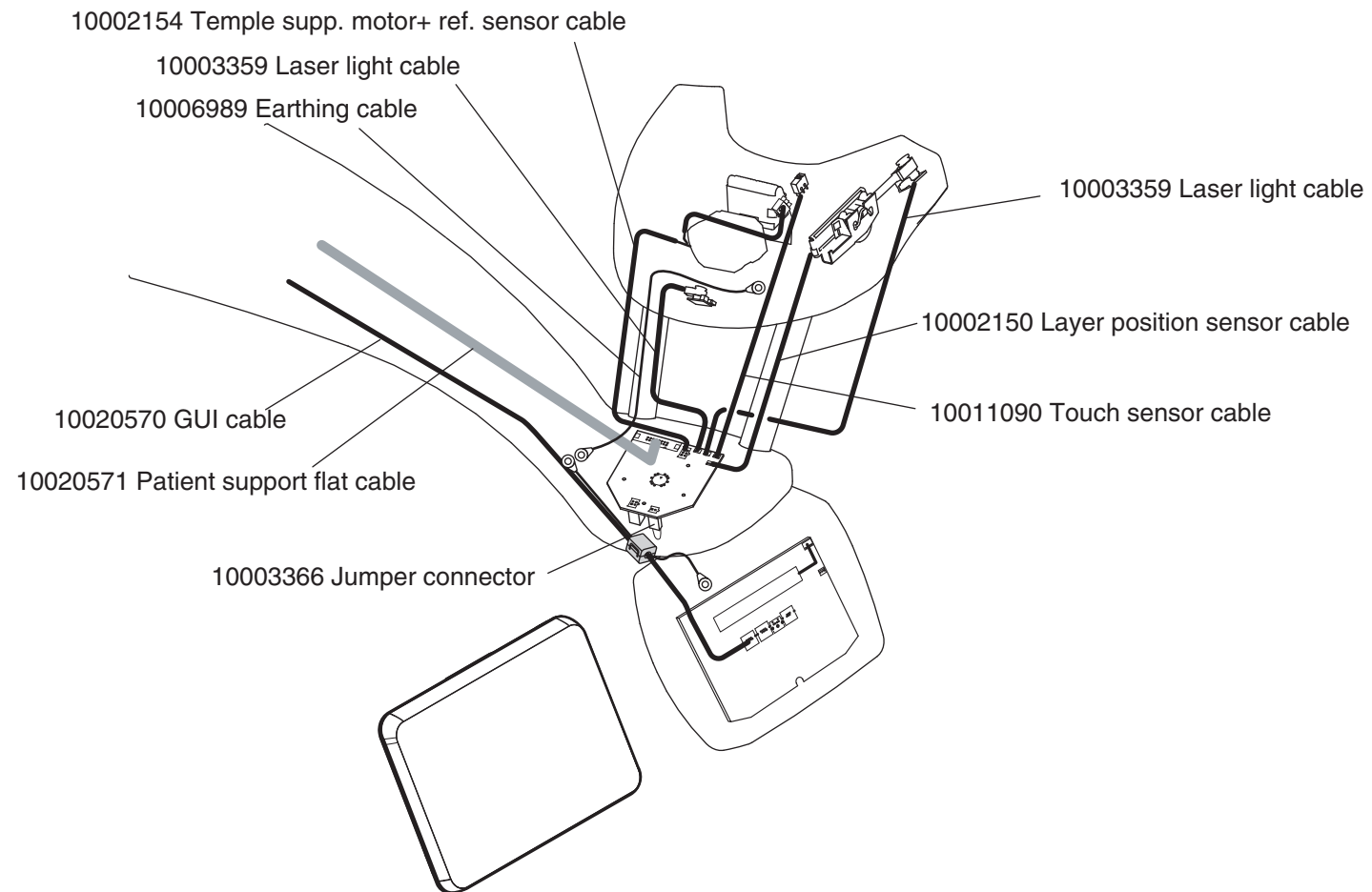


Planmeca ProMax

Upper arm


PLANMECA

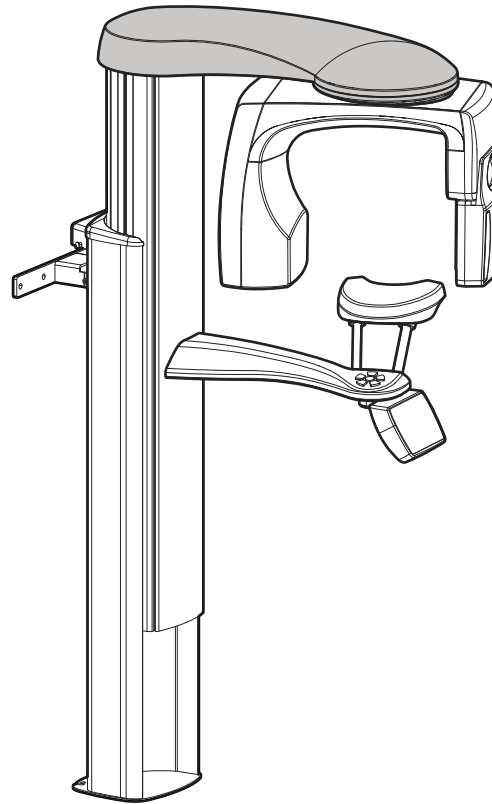


SCARA2- Shoulder arm parts

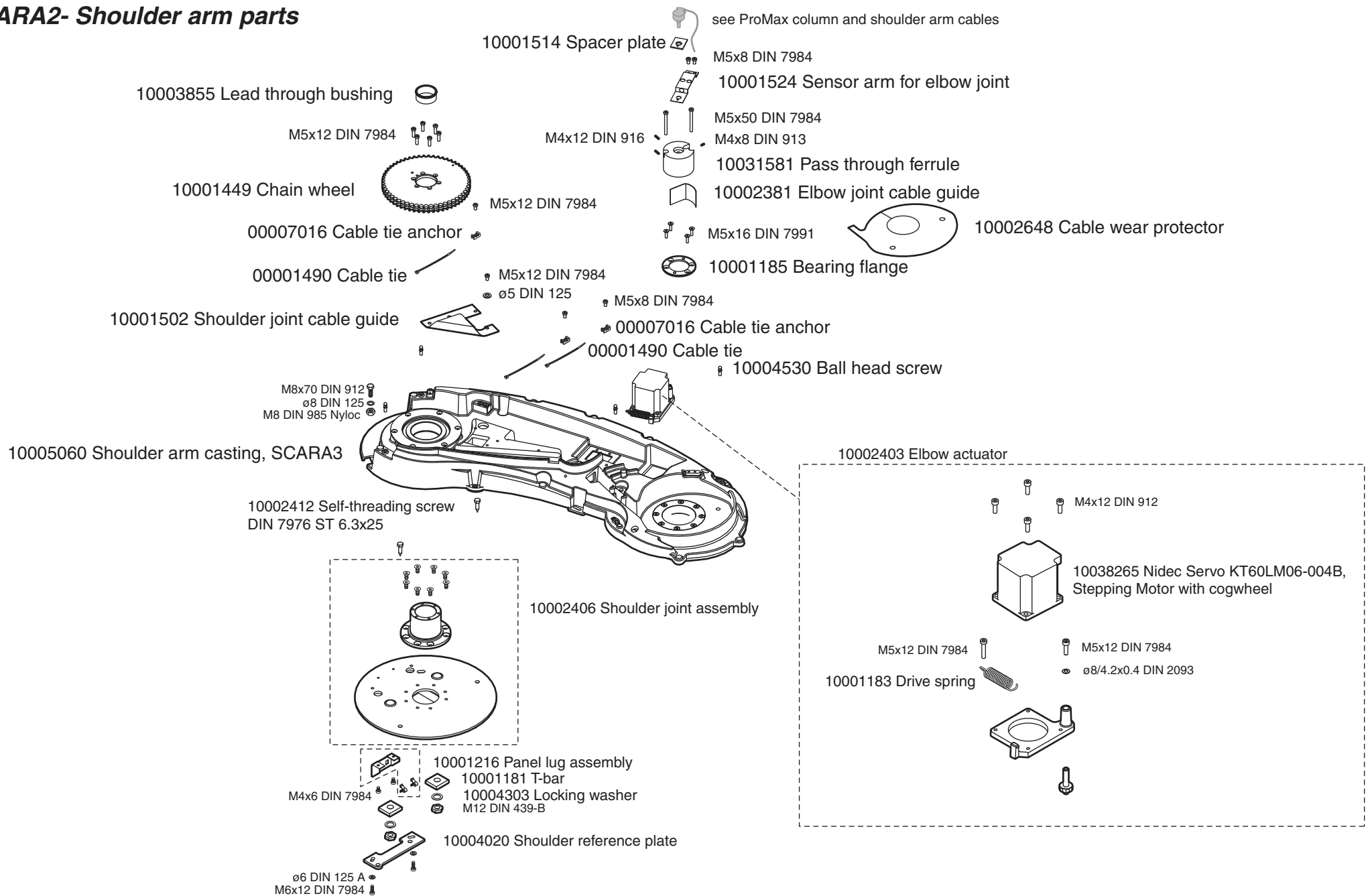









Shoulder arm cover and elbow arm

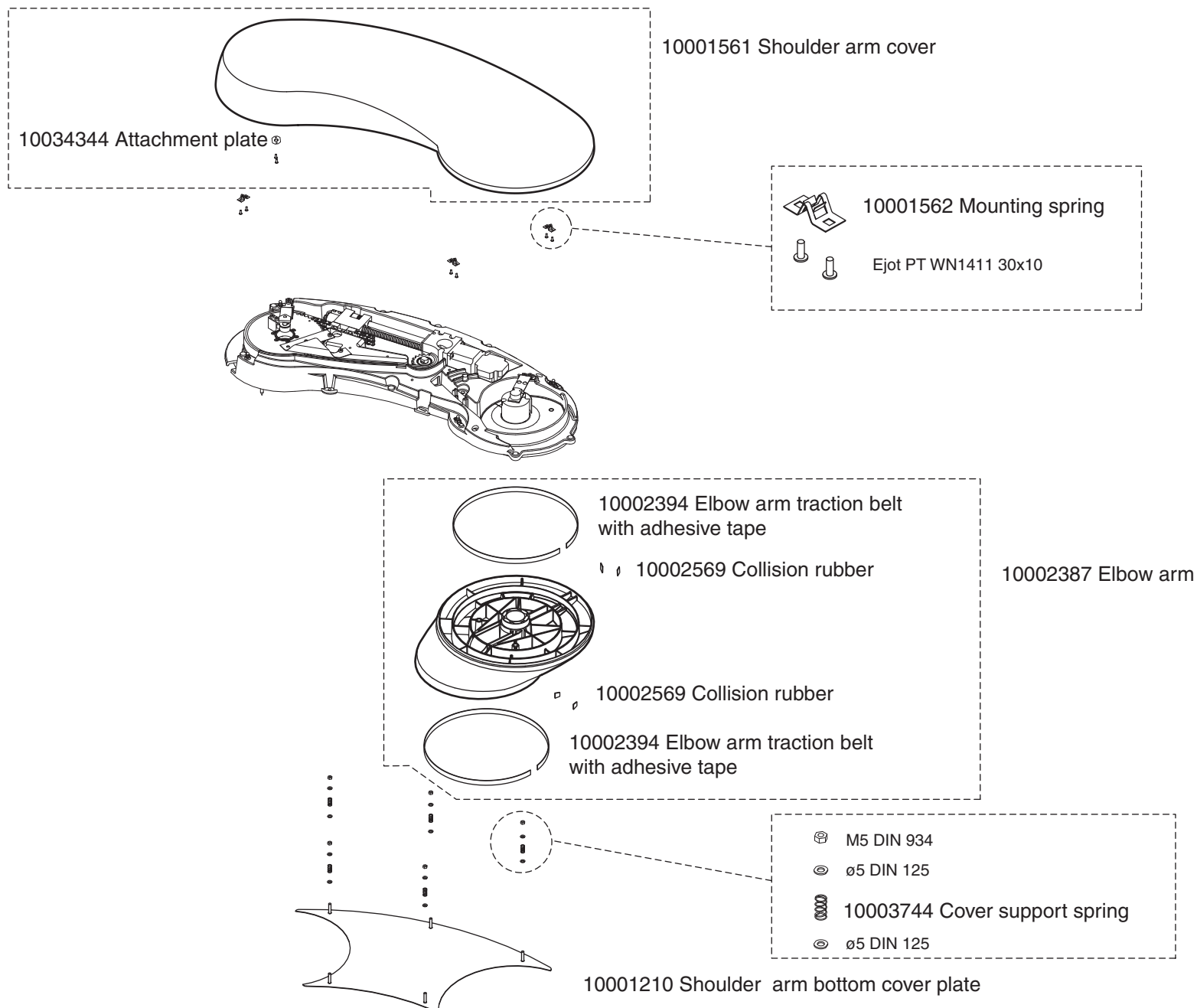


SCARA3- Linear actuator

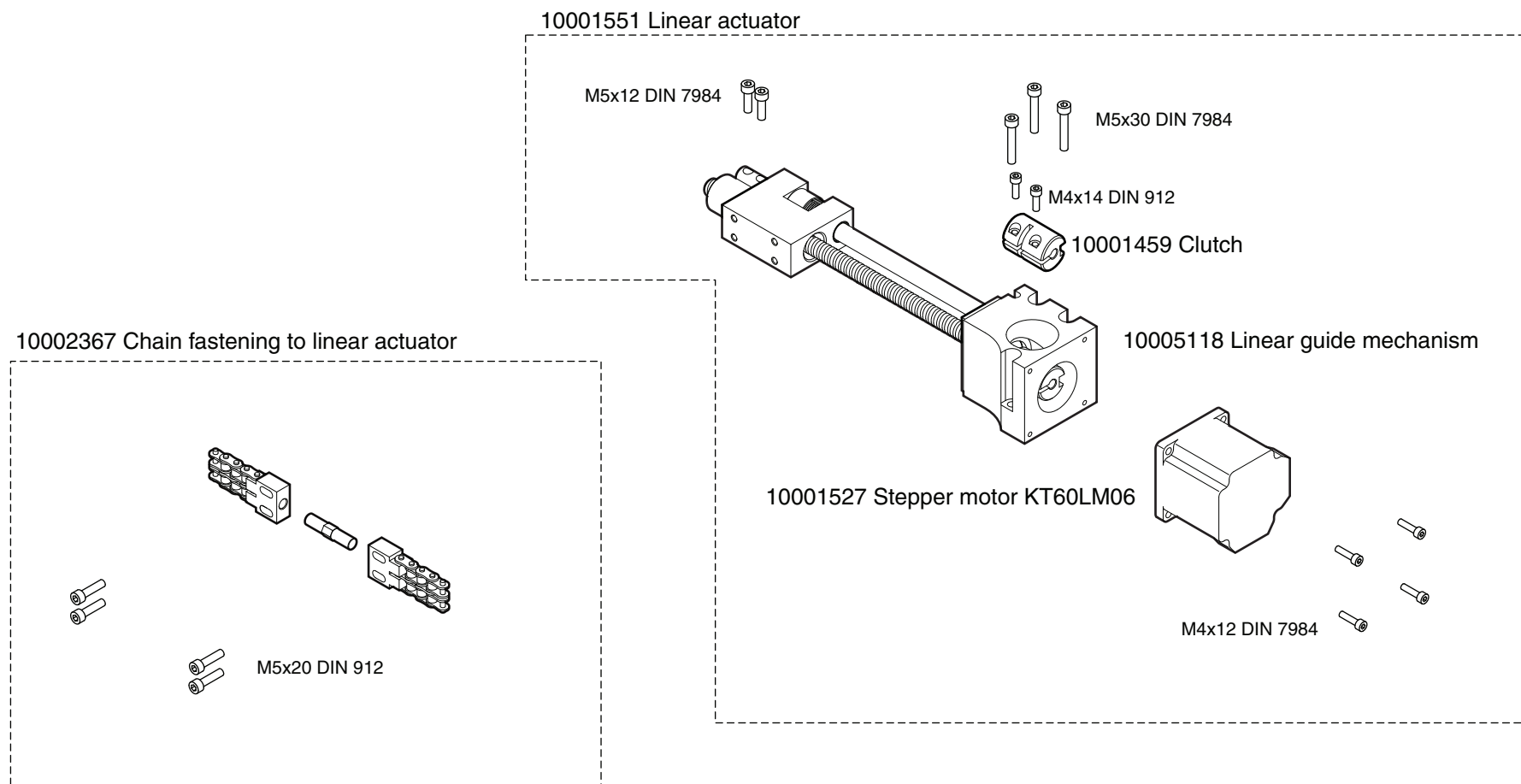


10002373 Fastening of shoulder joint sensor

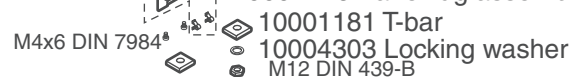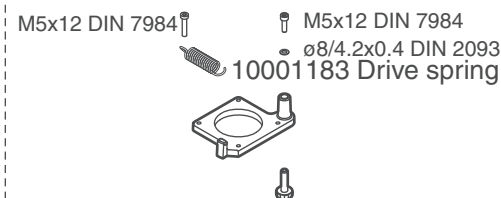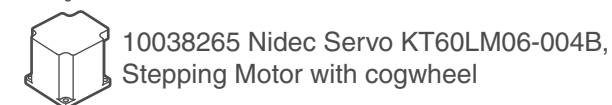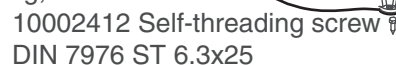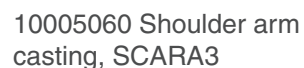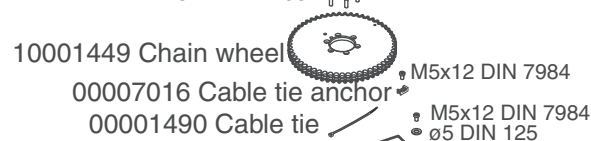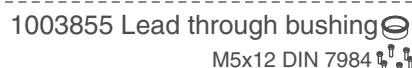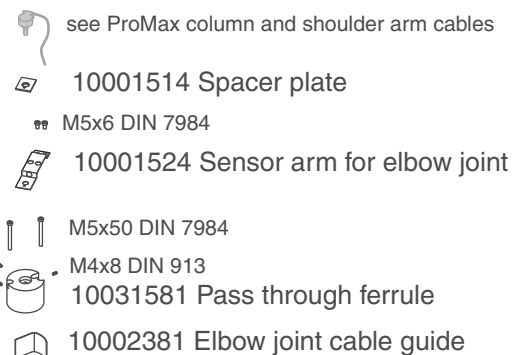


SCARA3- Shoulder arm parts, 3D Plus, 3D Mid

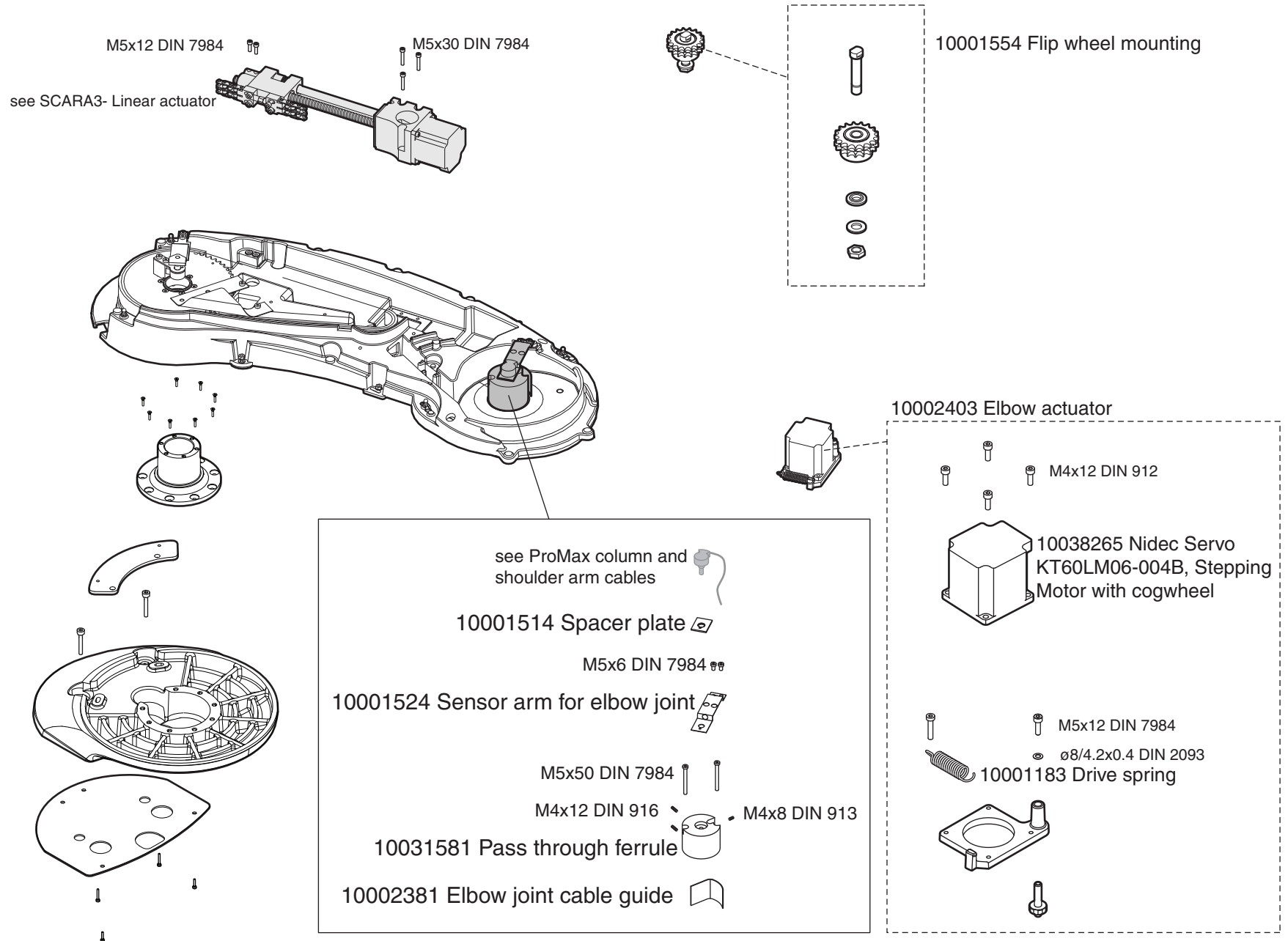


Shoulder arm cover and elbow arm, 3D Plus, 3D Mid

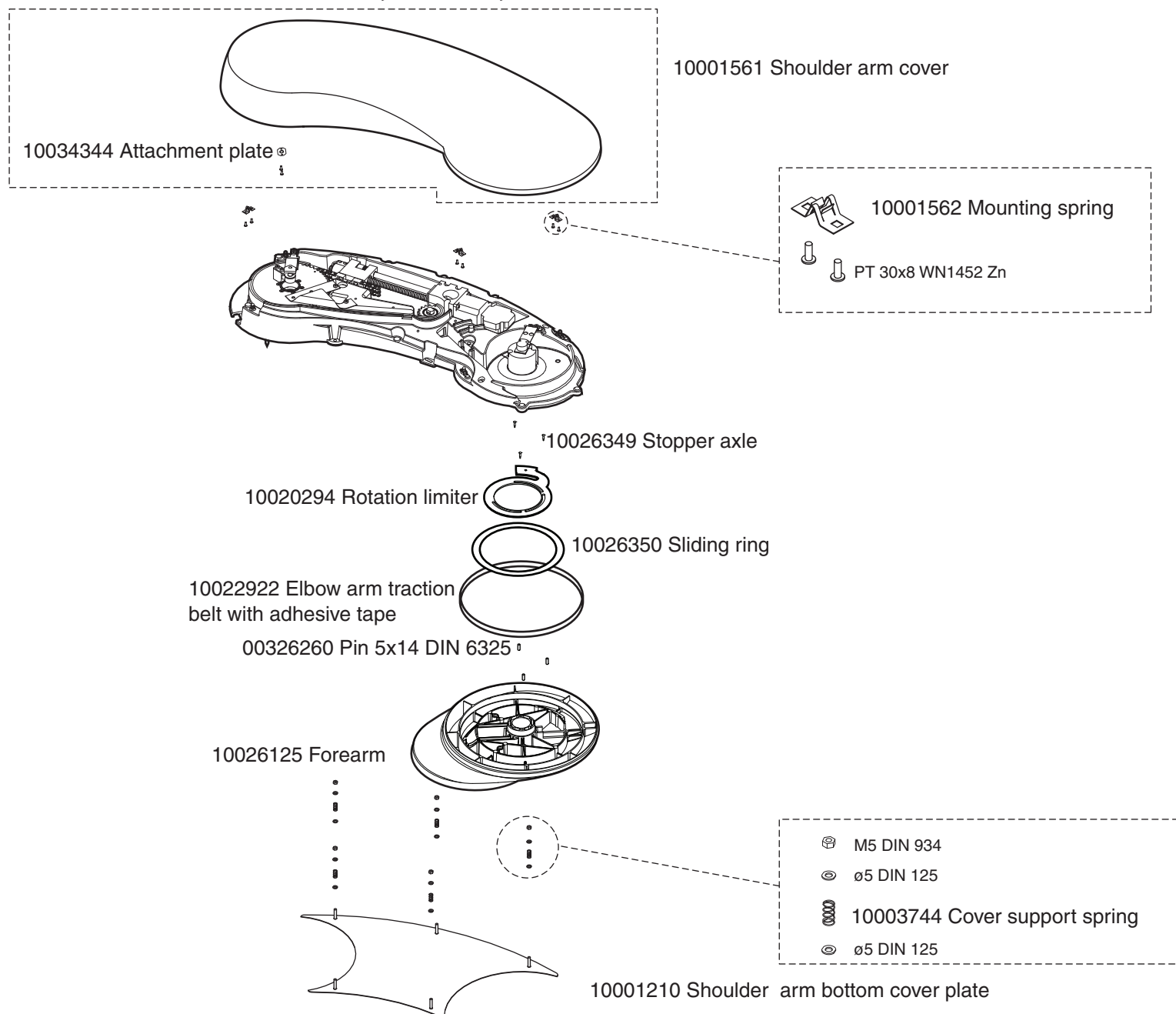

SCARA3- Shoulder arm parts, 3D Max

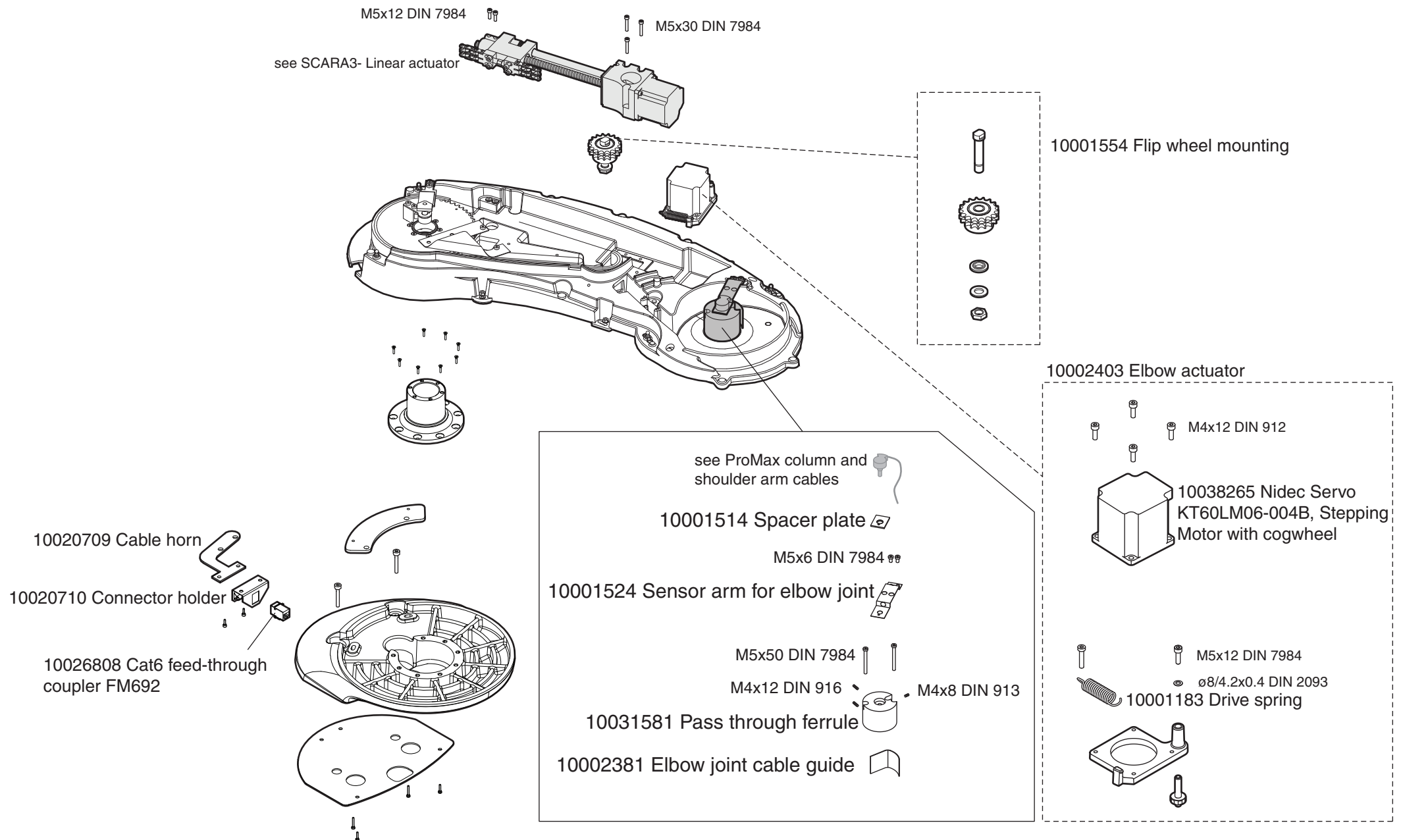


Shoulder arm cover and elbow arm, 3D Max

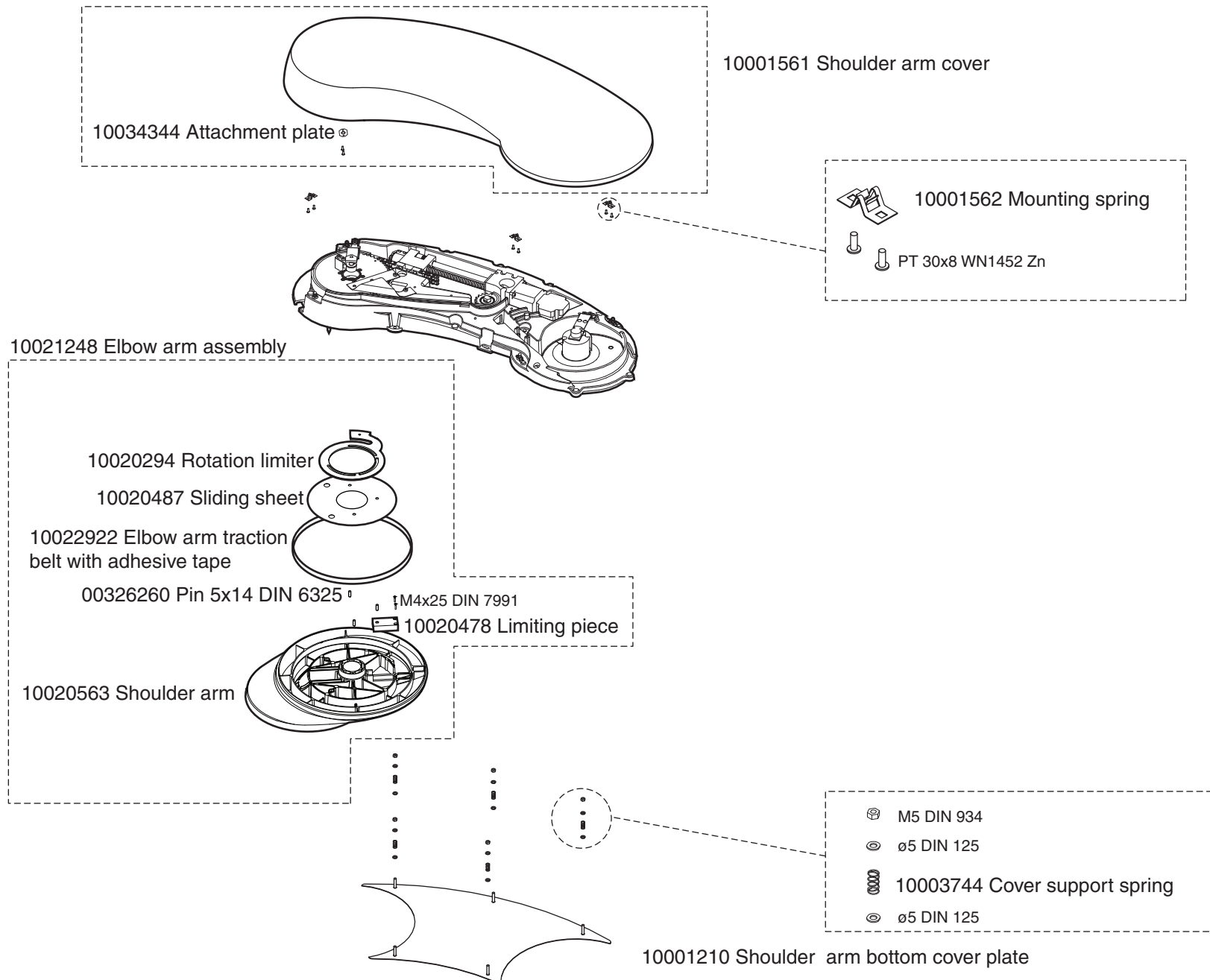


Planmeca ProMax C-arm


PLANMECA

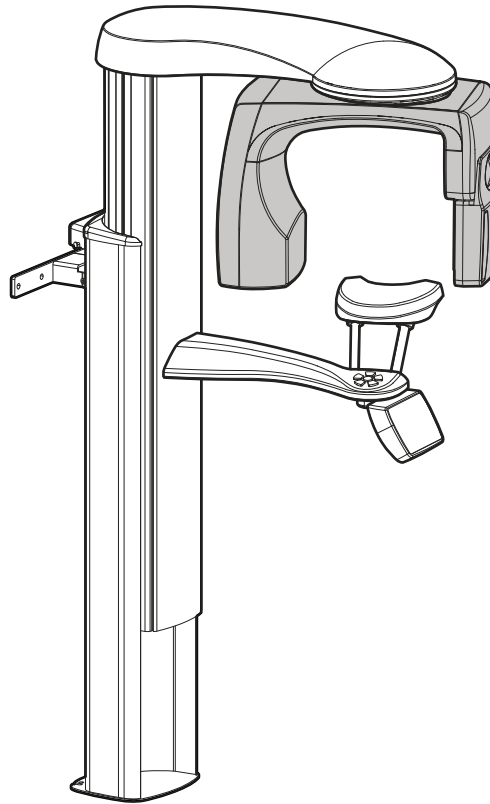


C-arm, 2D, 3D s, 3D Classic

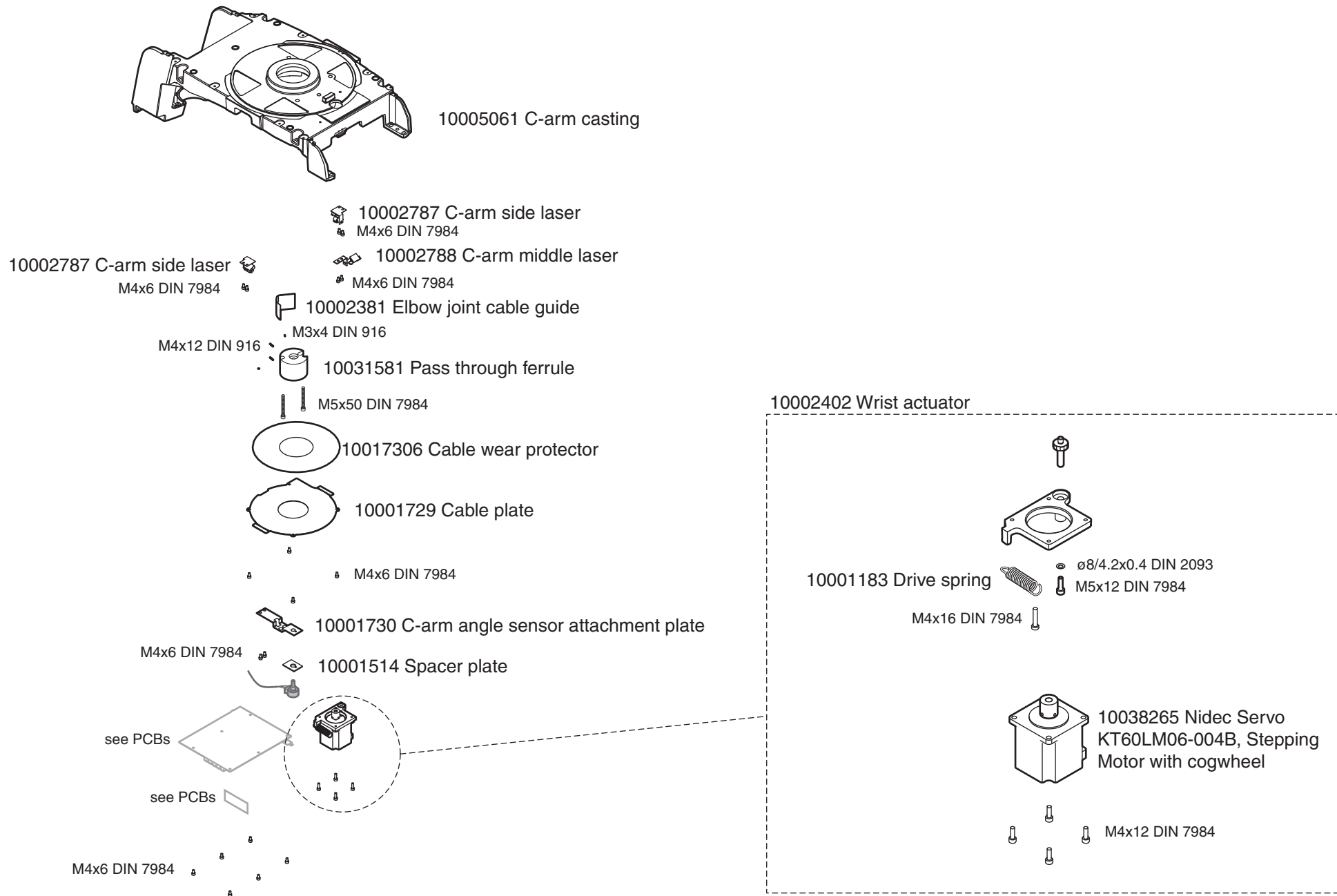


C-arm, 3D Plus, 3D Mid

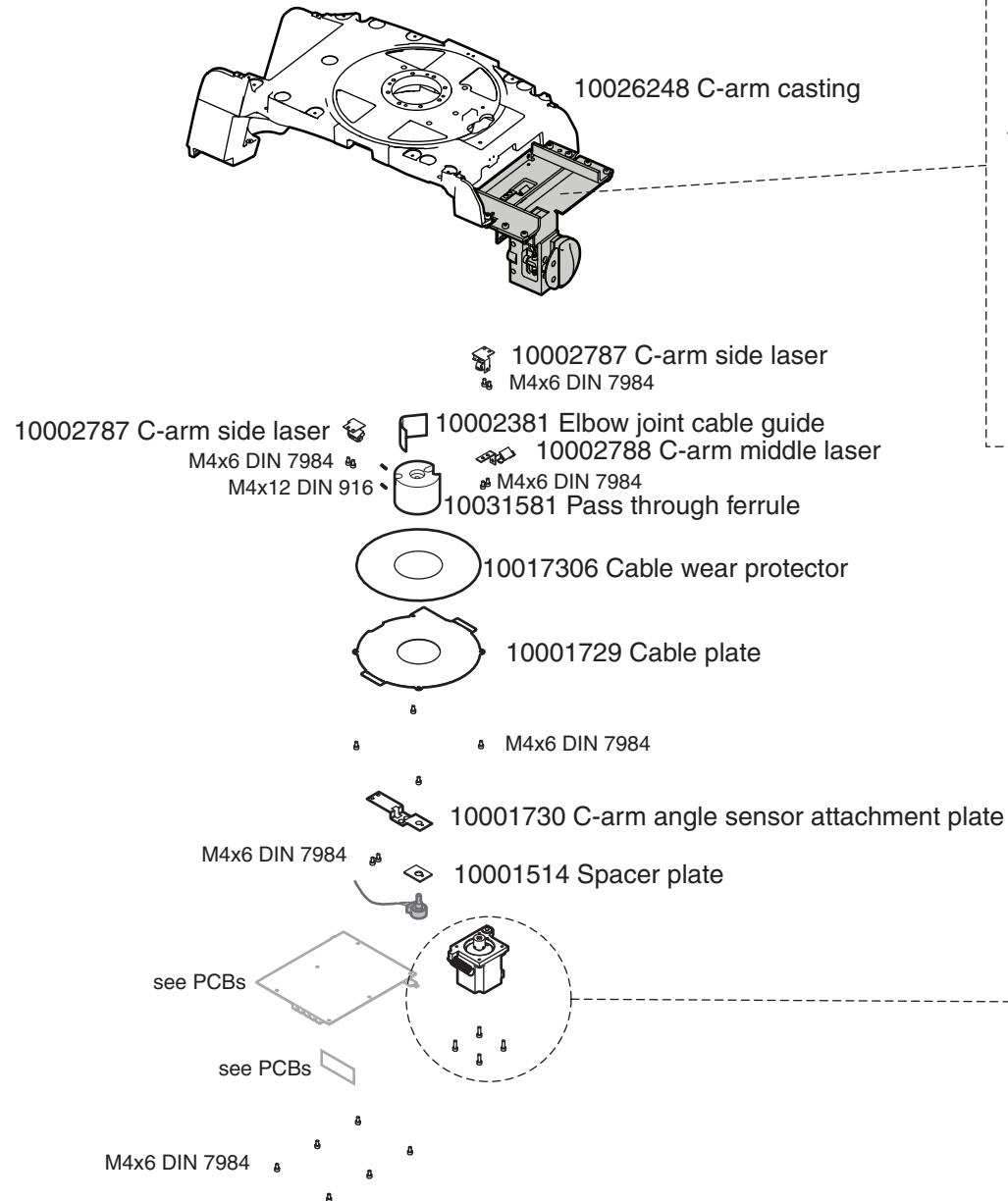


C-arm extension

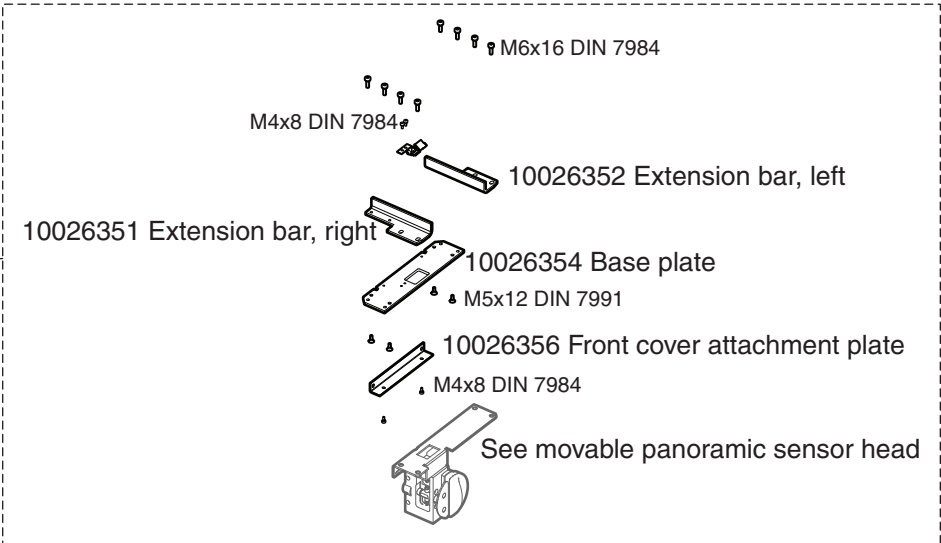


10002402 Wrist actuator

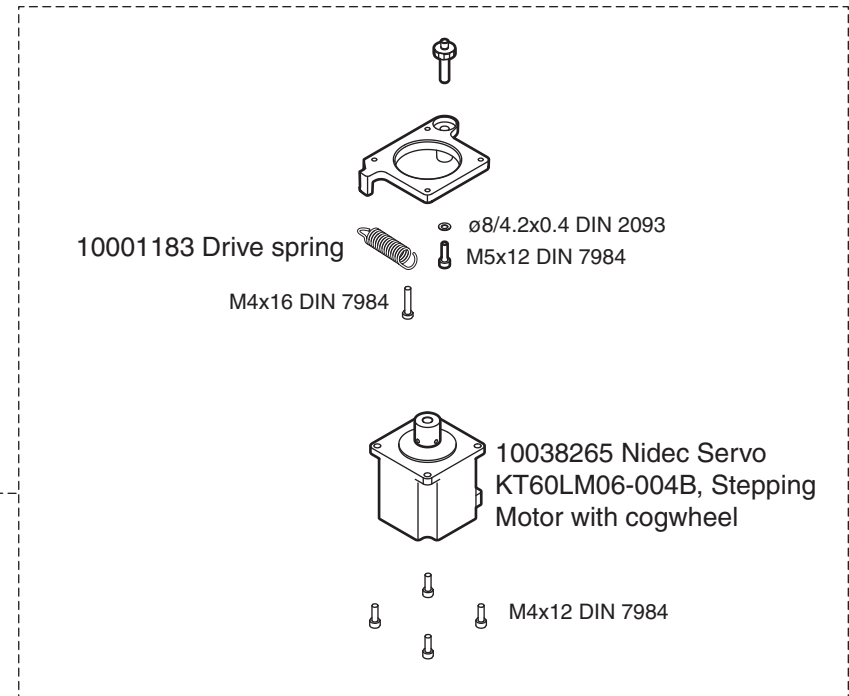


C-arm, 120kV, 3D Plus, 3D Mid

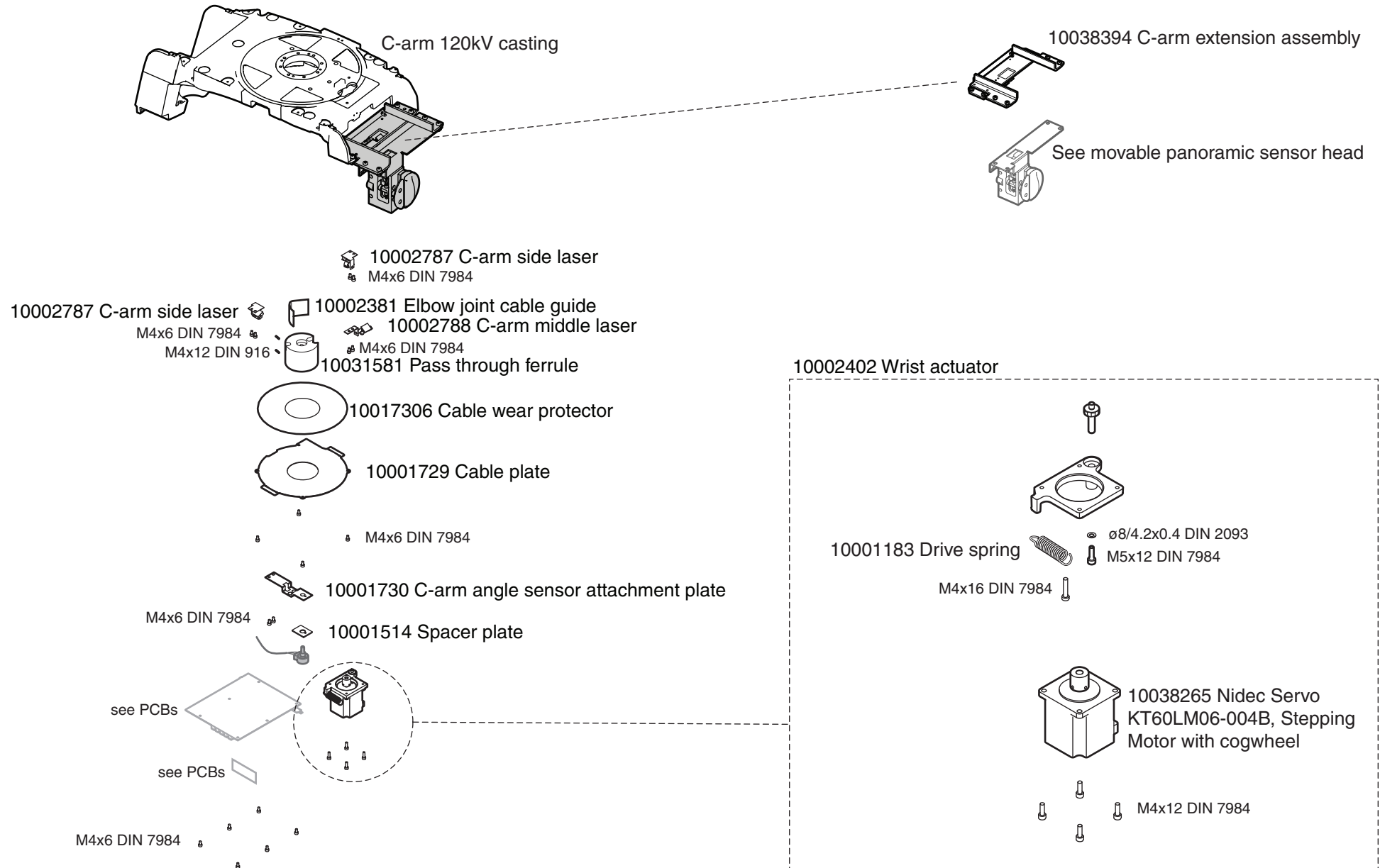


C-arm, 3D Max

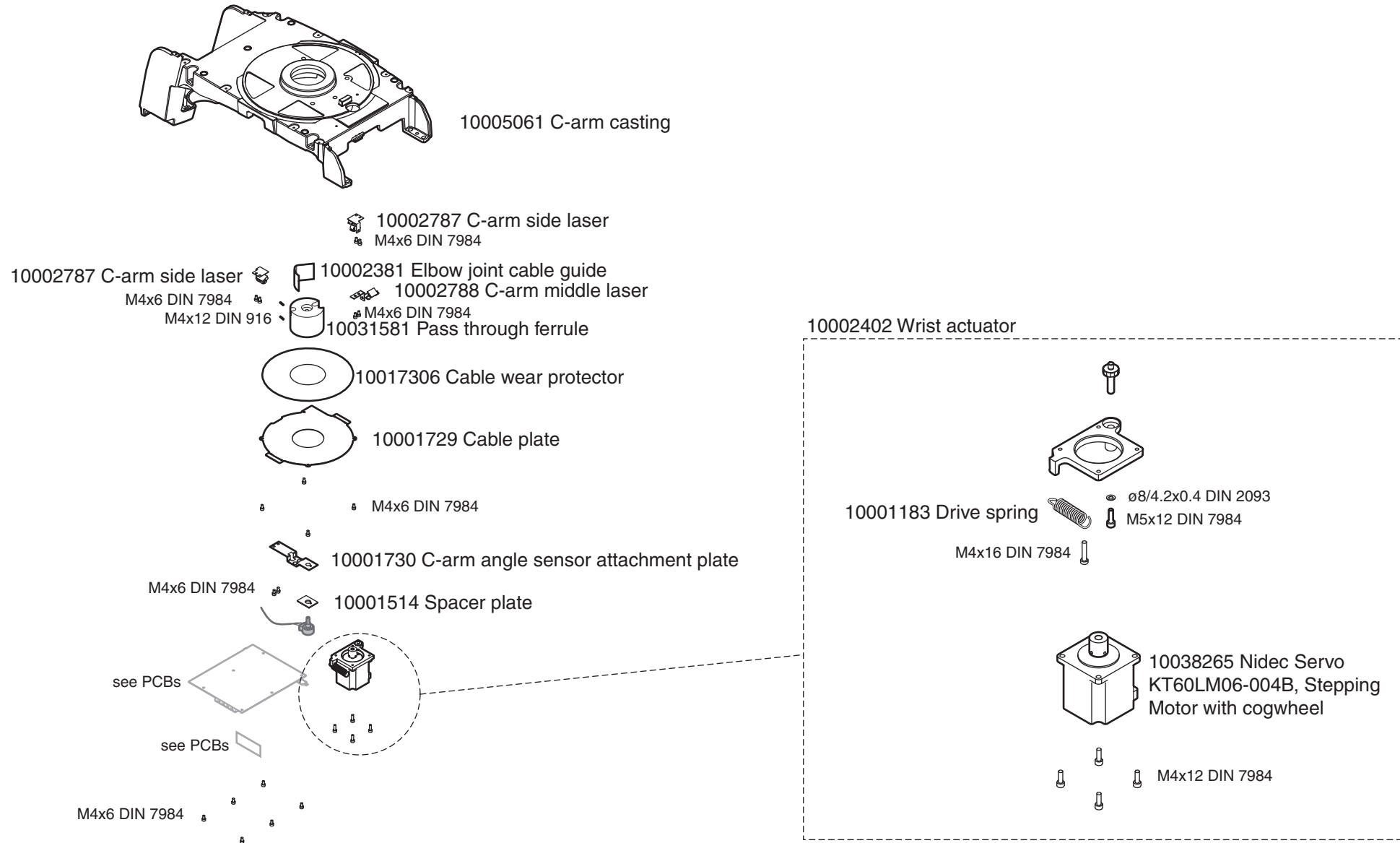


C-arm, 120kV, 3D Max

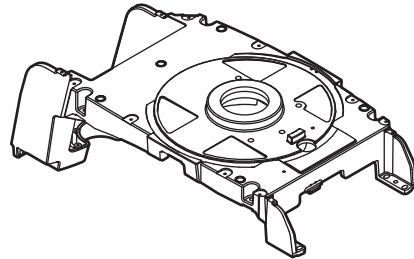


C-arm casting 120kV 3D Max

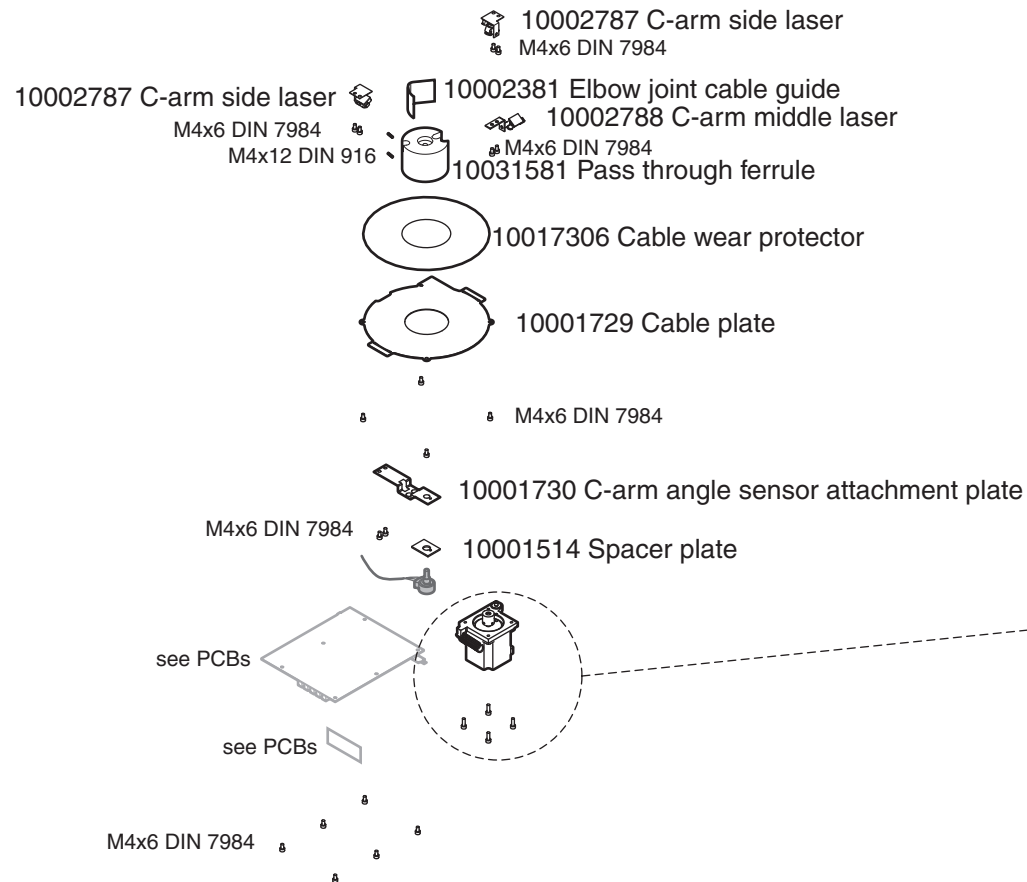


10002402 Wrist actuator

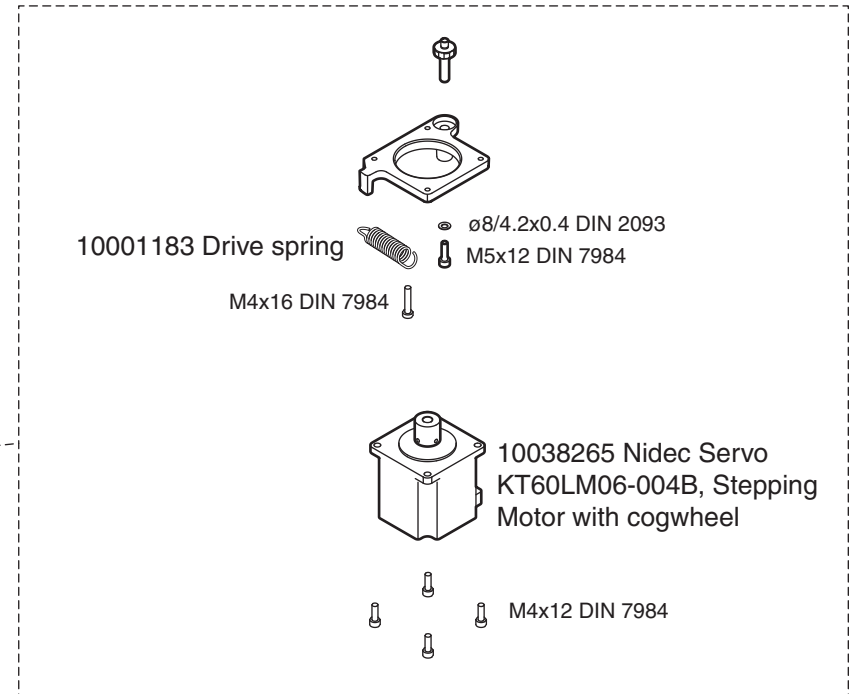



Laser lights

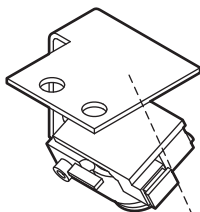


10002787 C-arm side laser

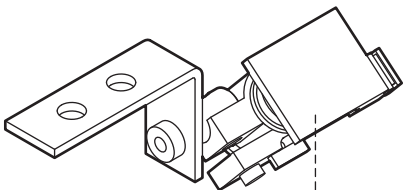


10002788 C-arm middle laser

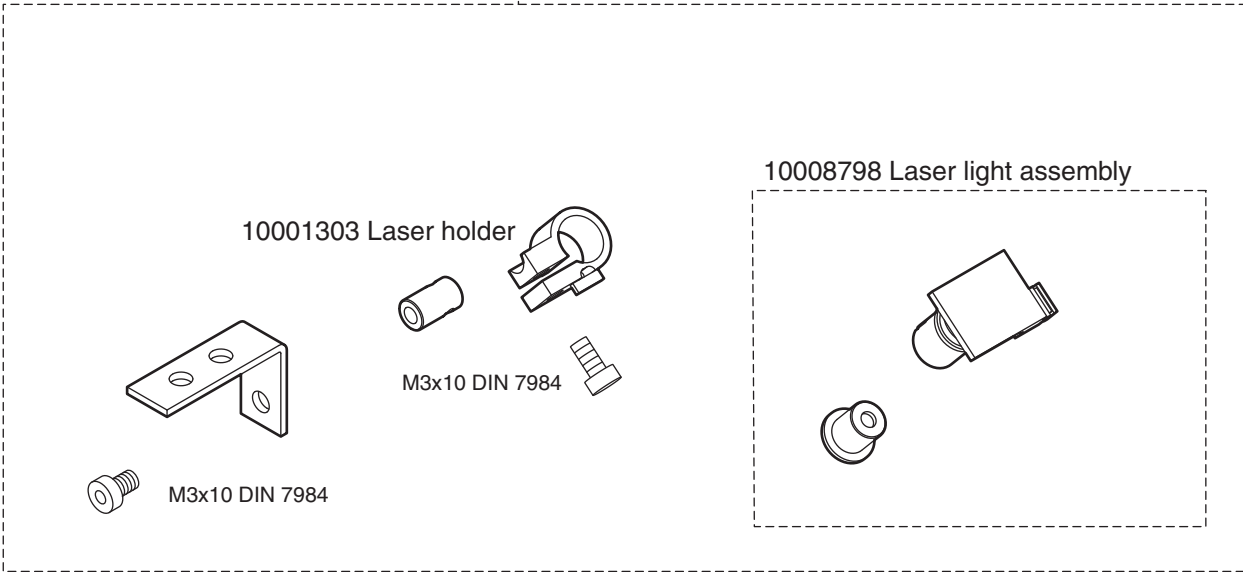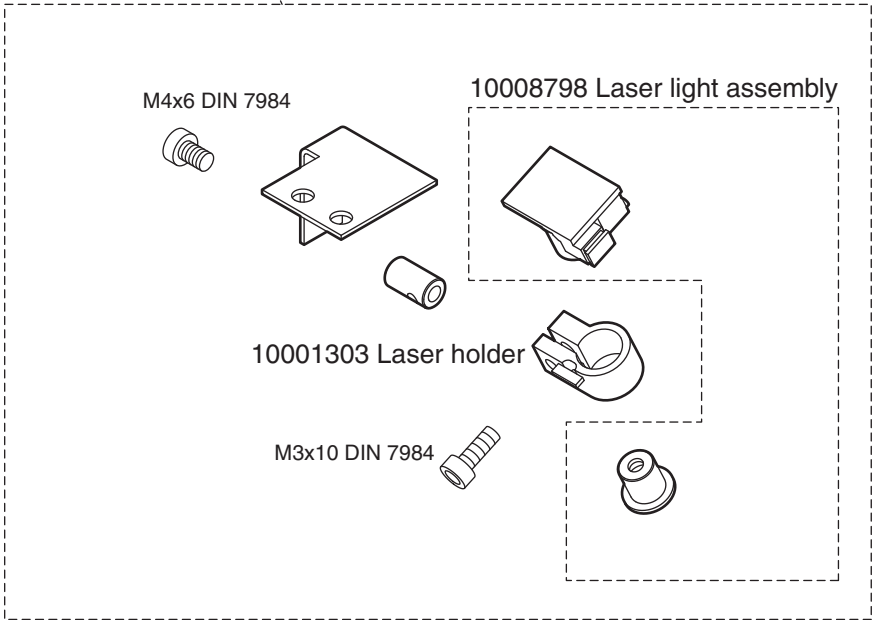


C-arm cables, 2D, 3D s, 3D Classic, 3D Plus, 3D Mid

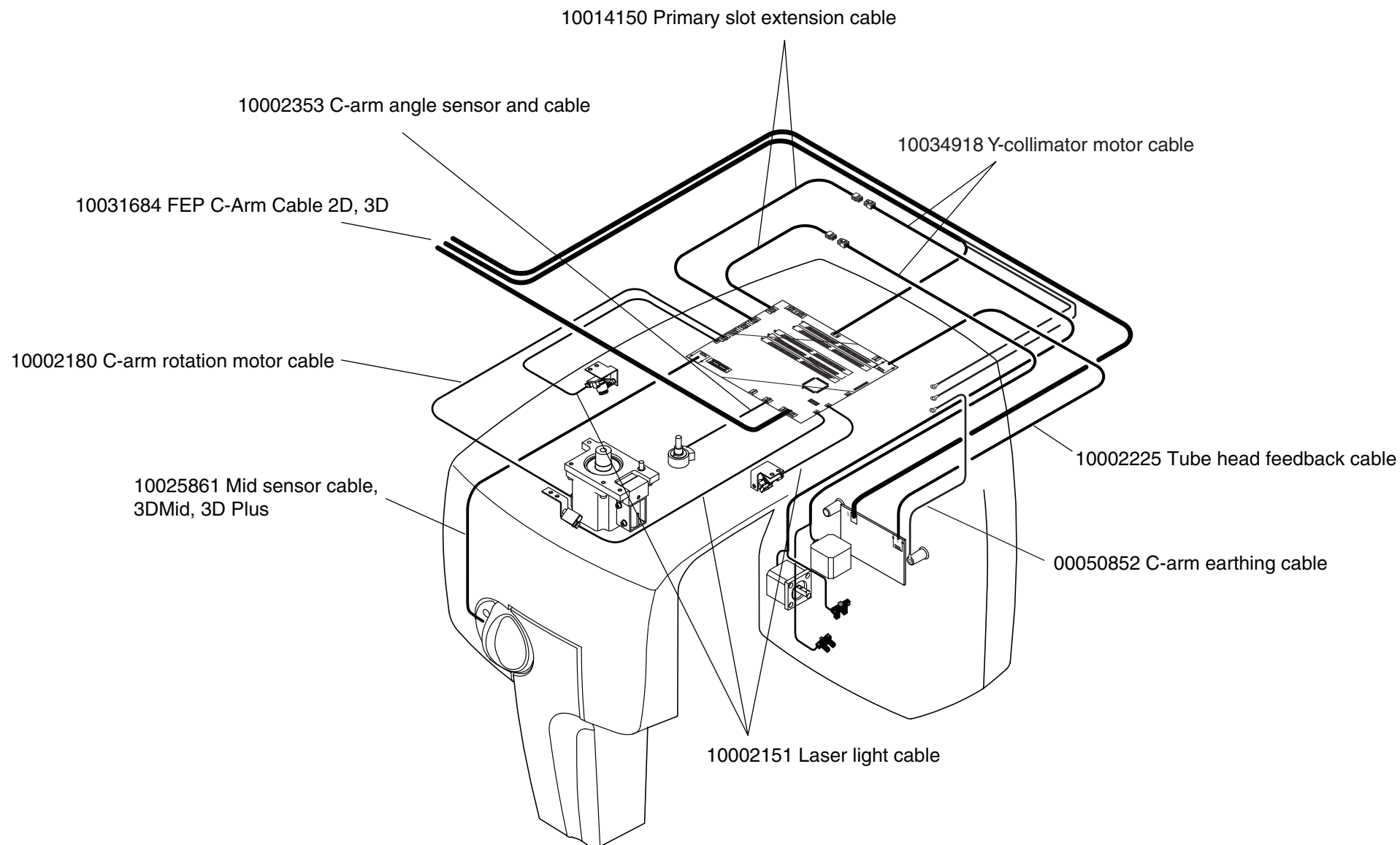


C-arm cables, 3D Max

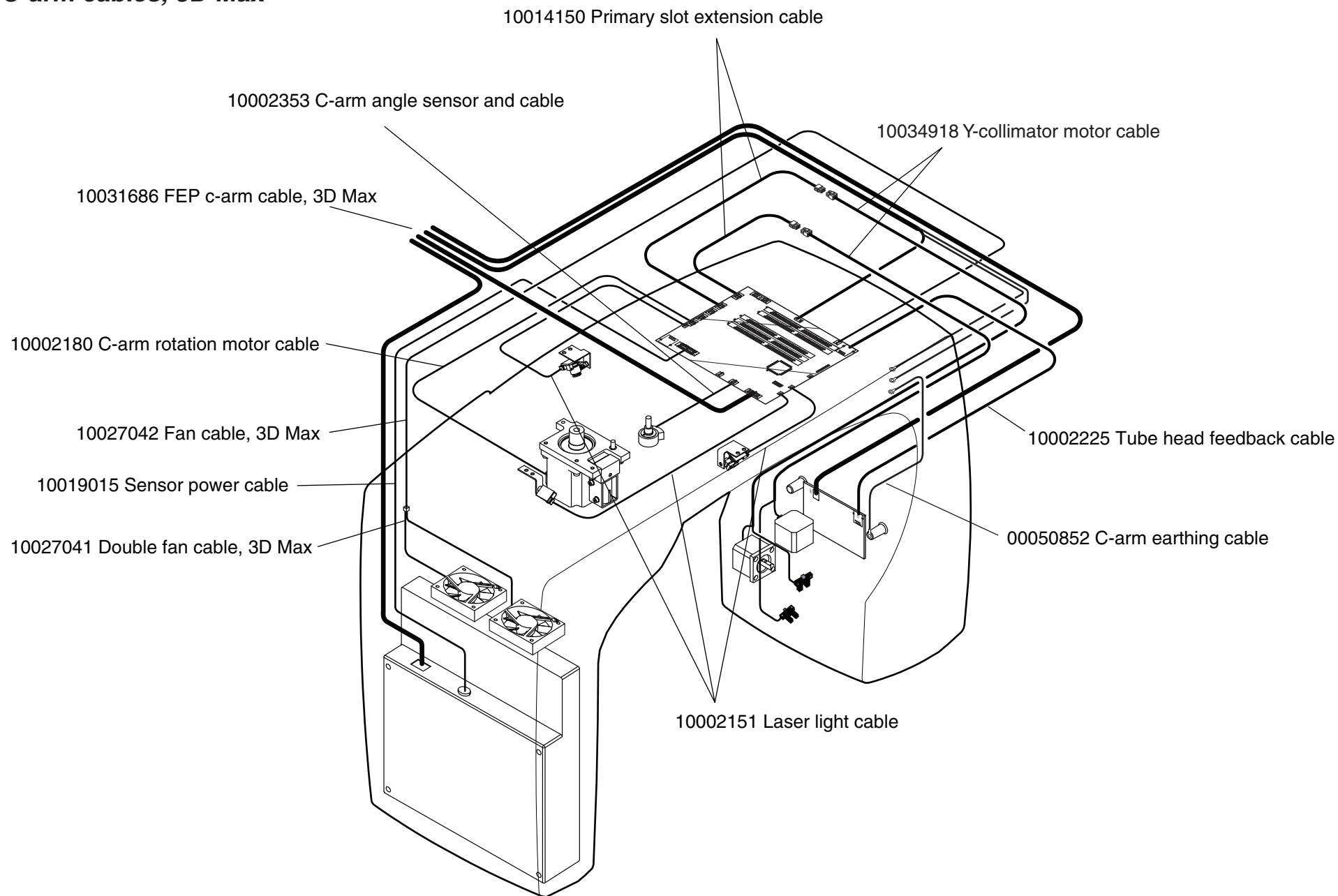


Covers, 2D, 3D s, 3D Classic

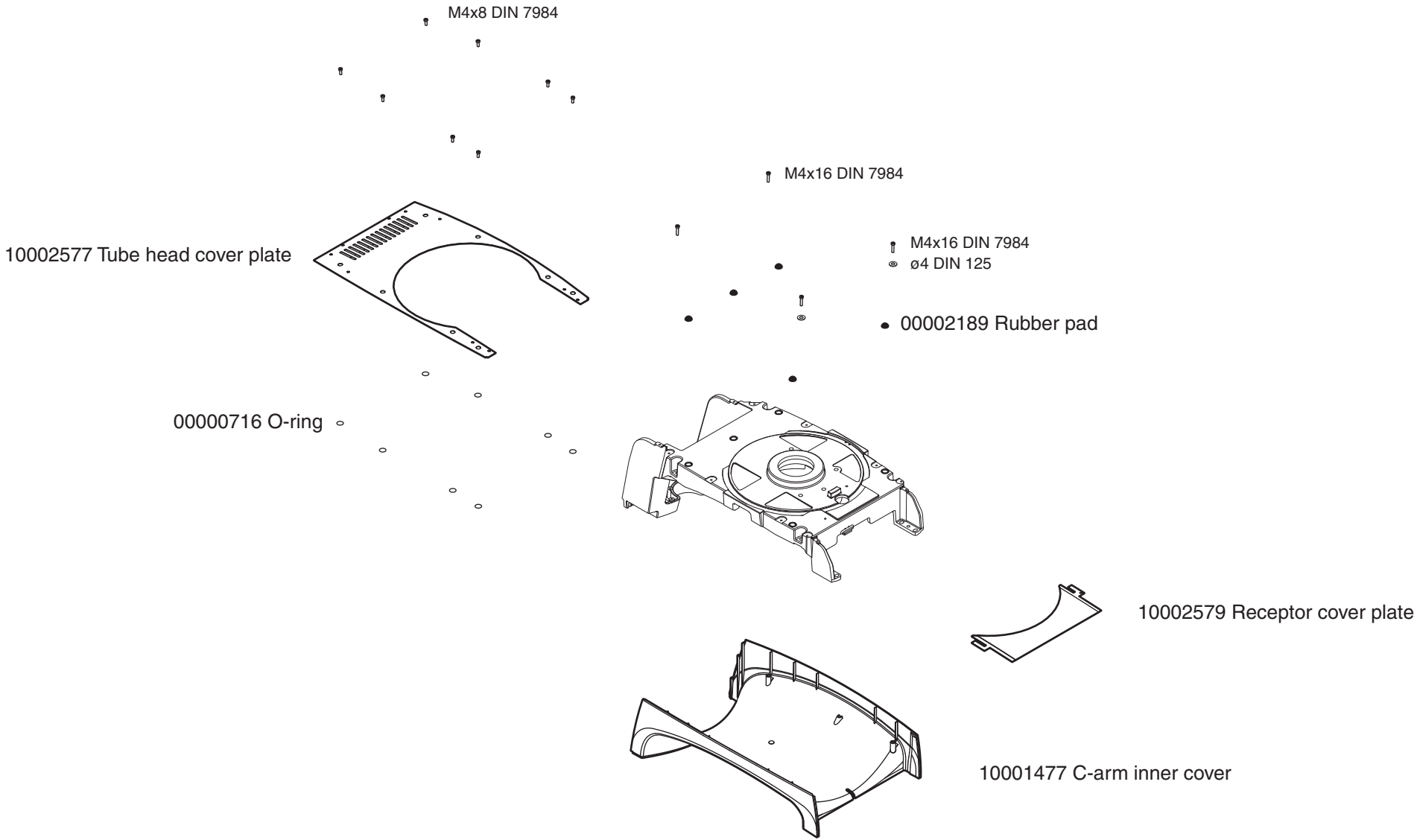


Covers, 3D Plus, 3D Mid

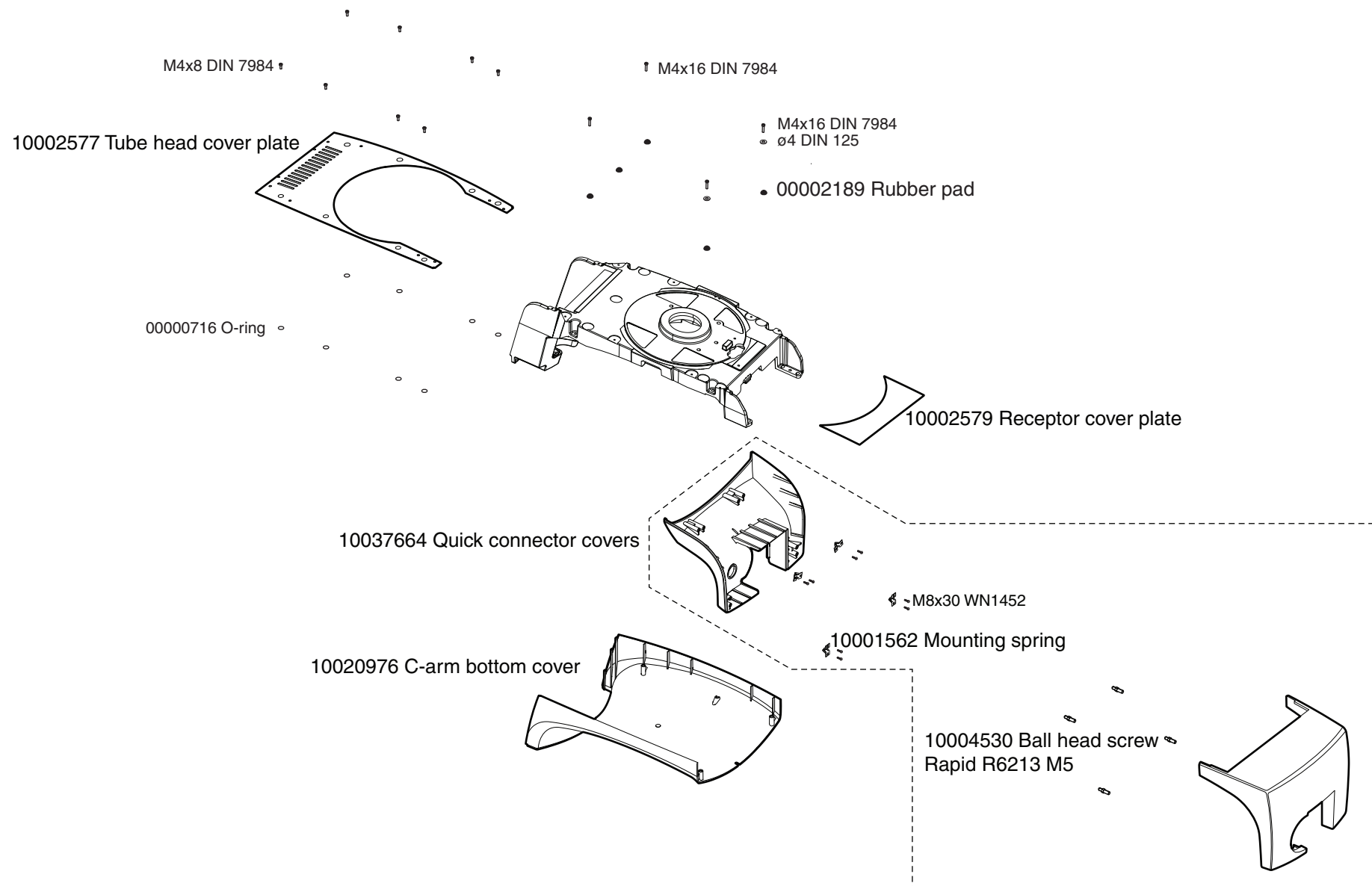


Covers, 3D Max

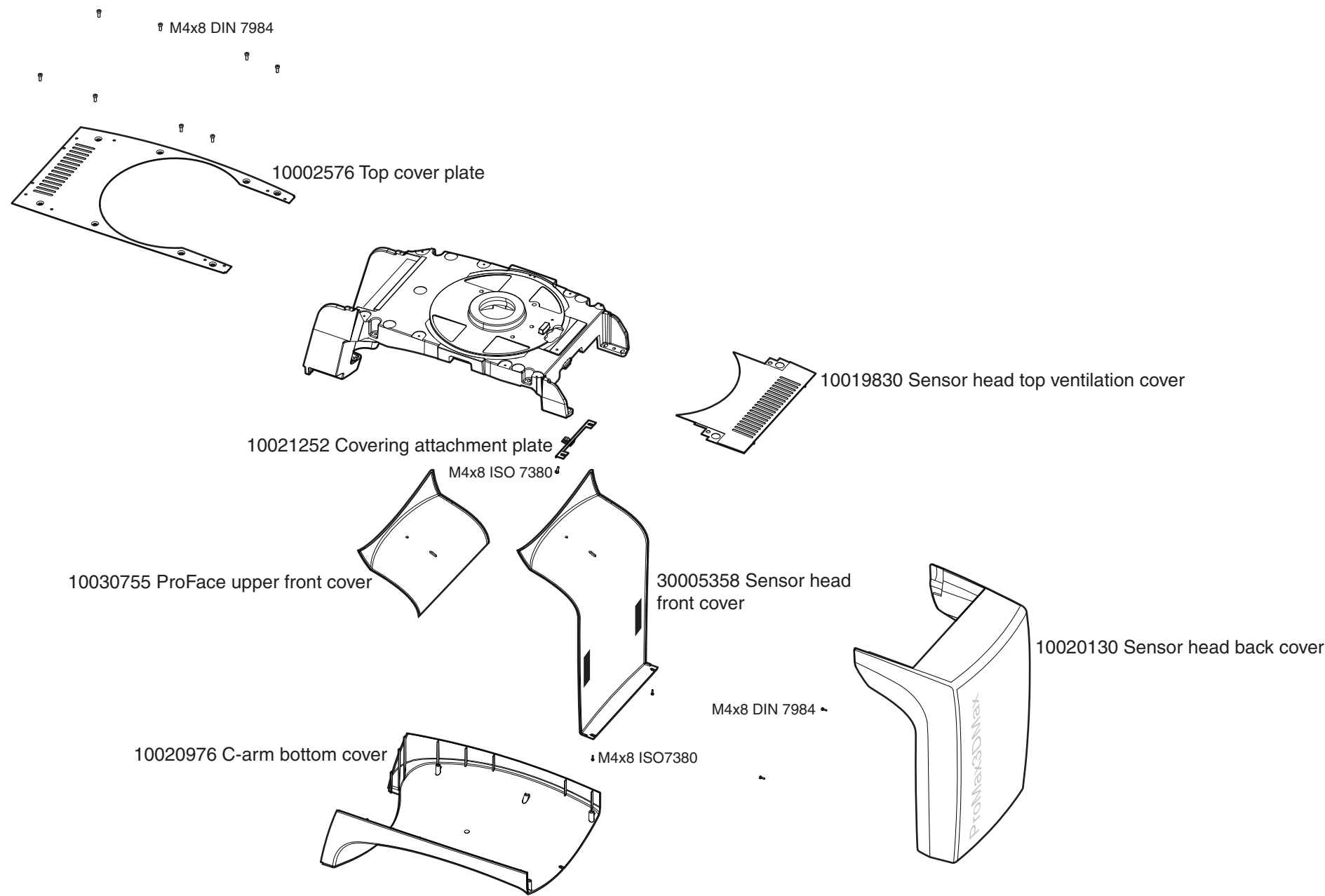

Sensor head connector

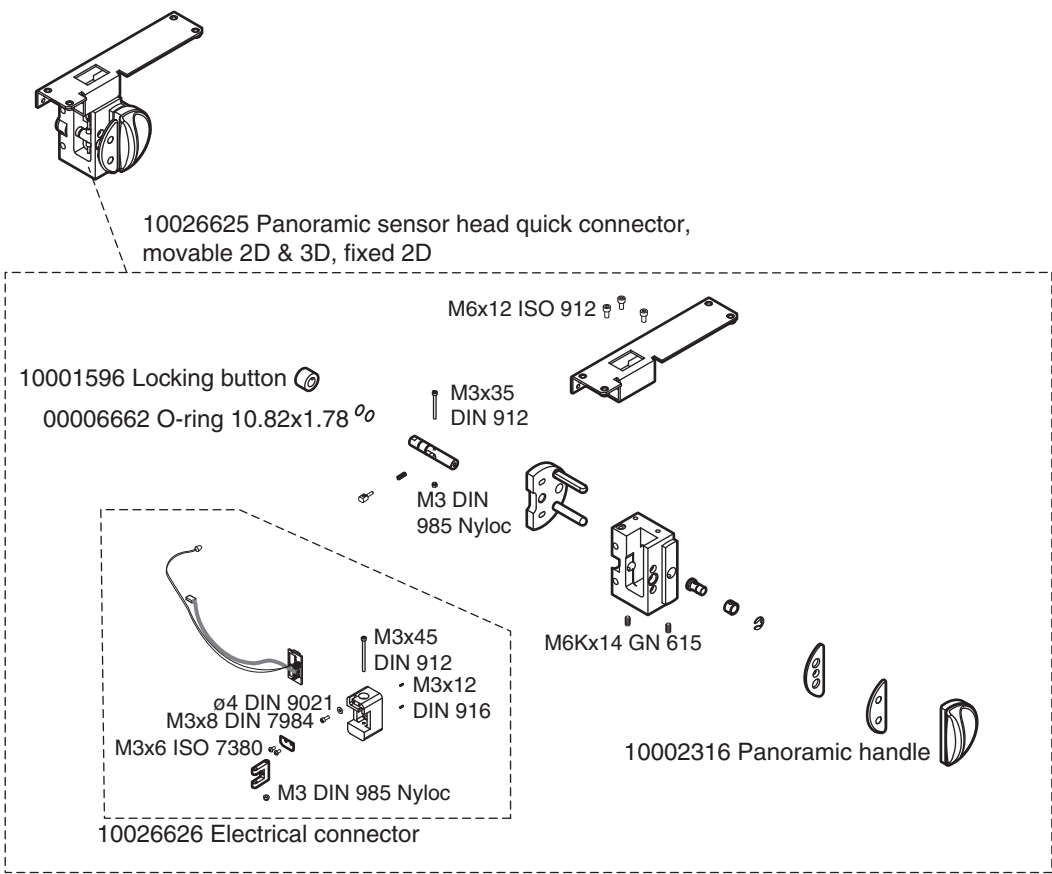


Sensor head connector covers

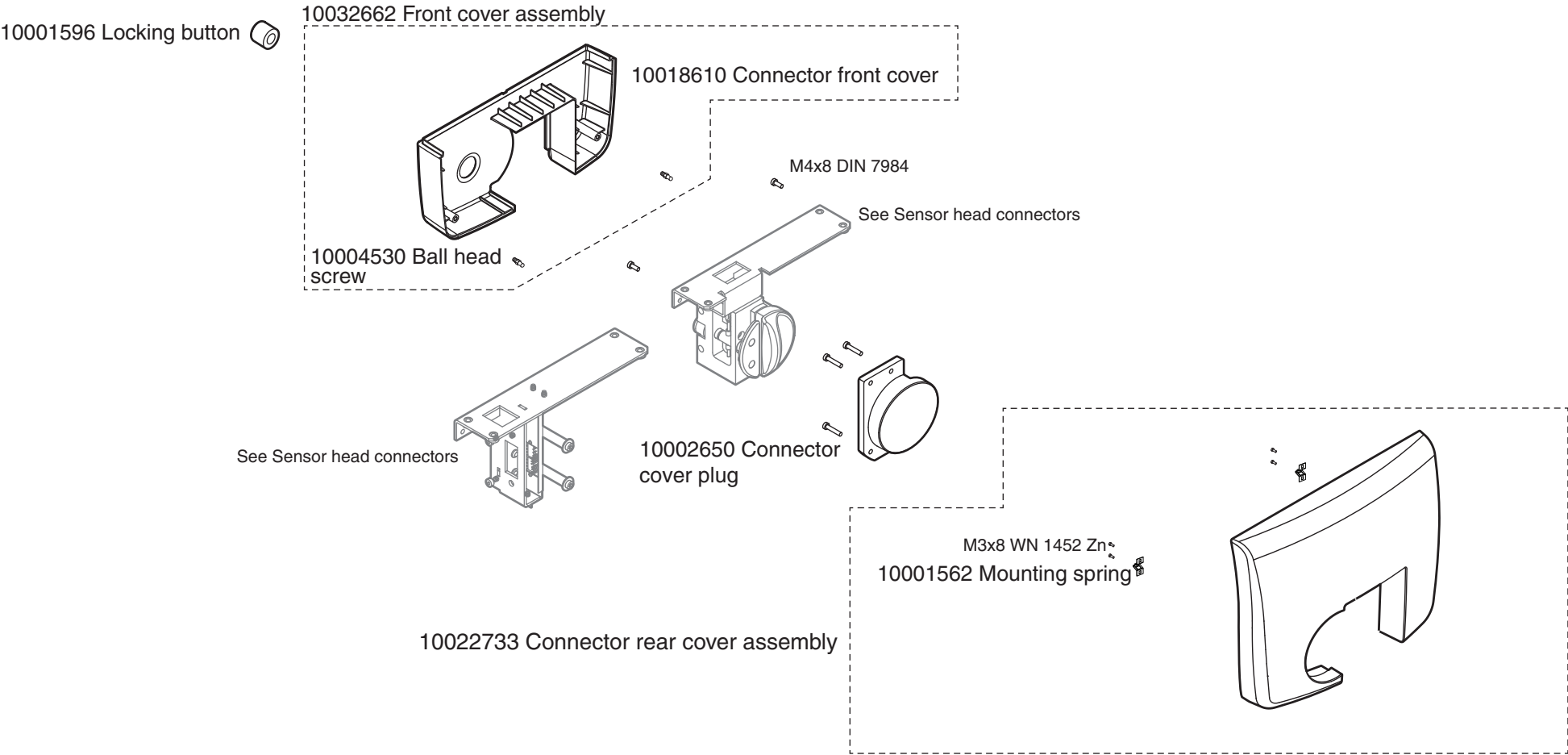

Planmeca ProMax


Tube head

PLANMECA

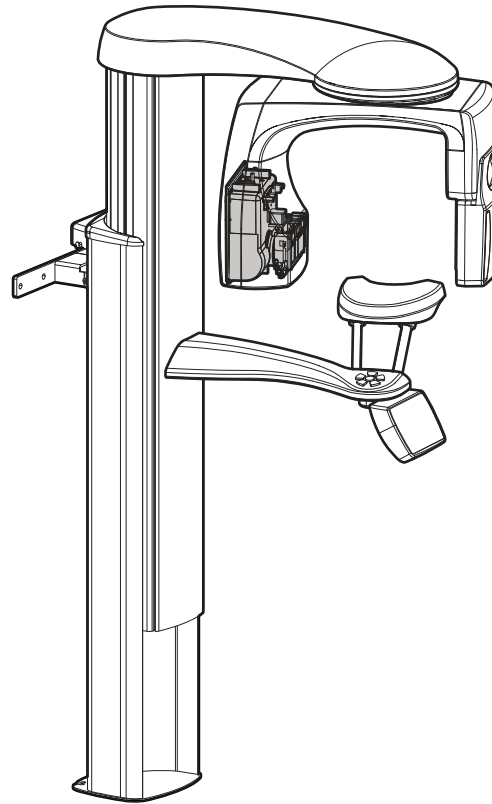

Tube head, fixed


10039227 ProMax tube head covers

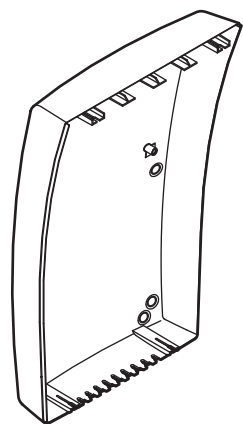

M4x12 DIN 7984
10002497 Base spring

M4x16 DIN 7984

see Primary collimator,
Primary collimator Cu 3D or
Primary collimator 3D Max

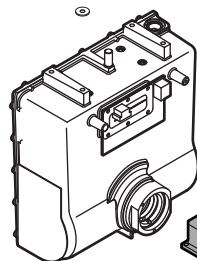

10031219 ProMax 2D/3D tube head assembly, fixed and rotating
10021228 3D Max Tube head assembly
10002431 2D/Film tube head assembly

M4x8 DIN 7984

NOTE! For information about replacing tube head covers, see technical bulletin PX-117 at Planmeca dealer support


M4x6 DIN 912

10004446 Th-screw

10016595 Fan & adapter plate

M4x20 DIN 912

10016609 Fan

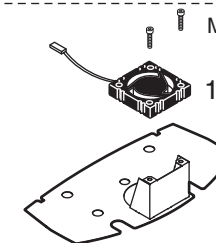


NOTE! For Planmeca ProMax tube head versions and compatibility information, see technical bulletin PX-076A at Planmeca dealer support

M4x8 DIN 7984


10002497 Base spring

10002340 Speed nut
Rapid NU 0604

M4x6 DIN 7984

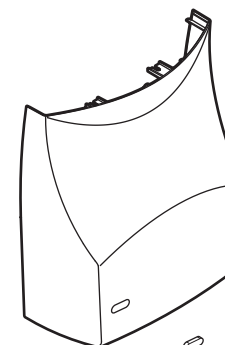

Tube head, rotating

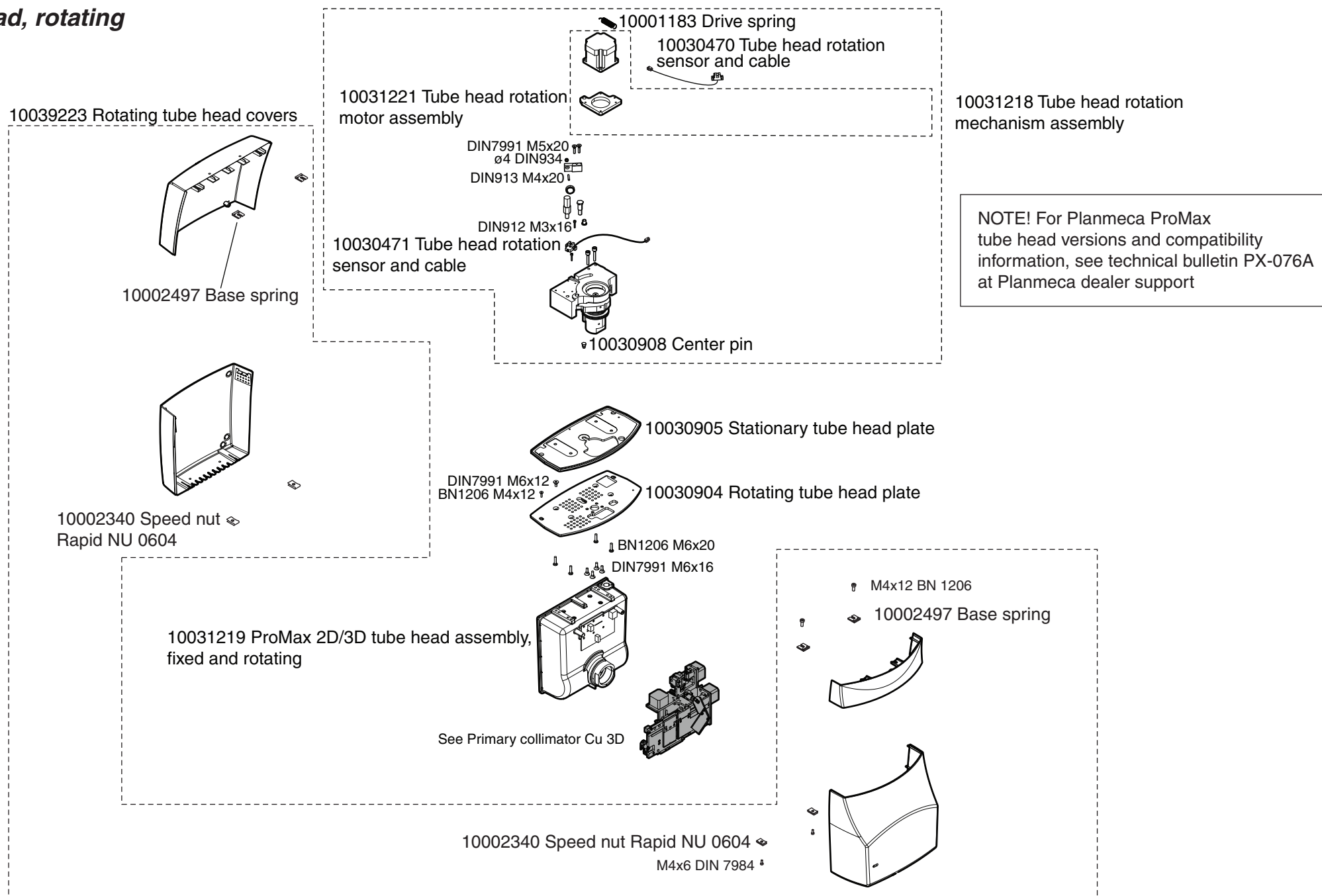

Tube head, 120kV rotating, 3D Plus, 3D Mid


10039264 Upper back cover assembly ProMax 120 kV




10039265 Lower back cover assembly Promax 120 kV



10037849 Tubehead rotation assembly 3D Mid/Plus 120 kV

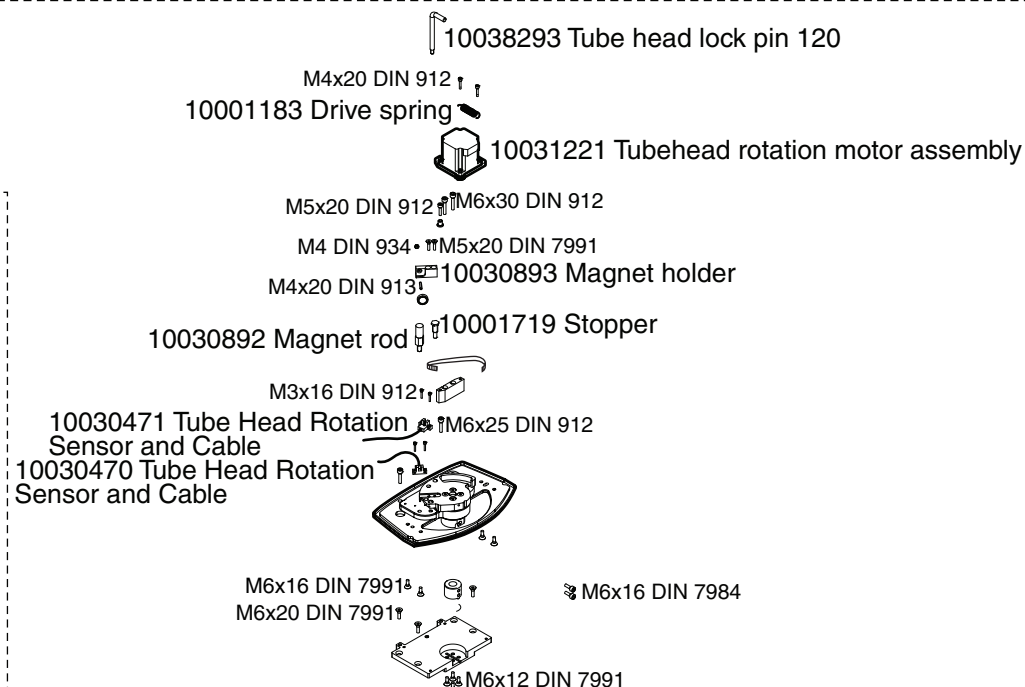


10039537 ProMax tube head 120 kV S

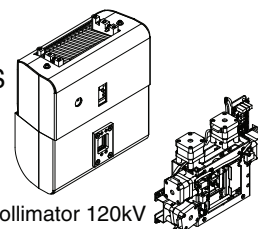


10039267 Tube head front cover assembly ProMax 120 kV


10002340 Speed nut Rapid NU 0604
M4x8 DIN 7984
10002497 Base spring
M4x8 DIN 7984

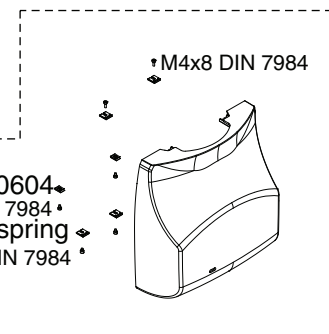

Tube head, 120kV rotating, locked, 3D Max


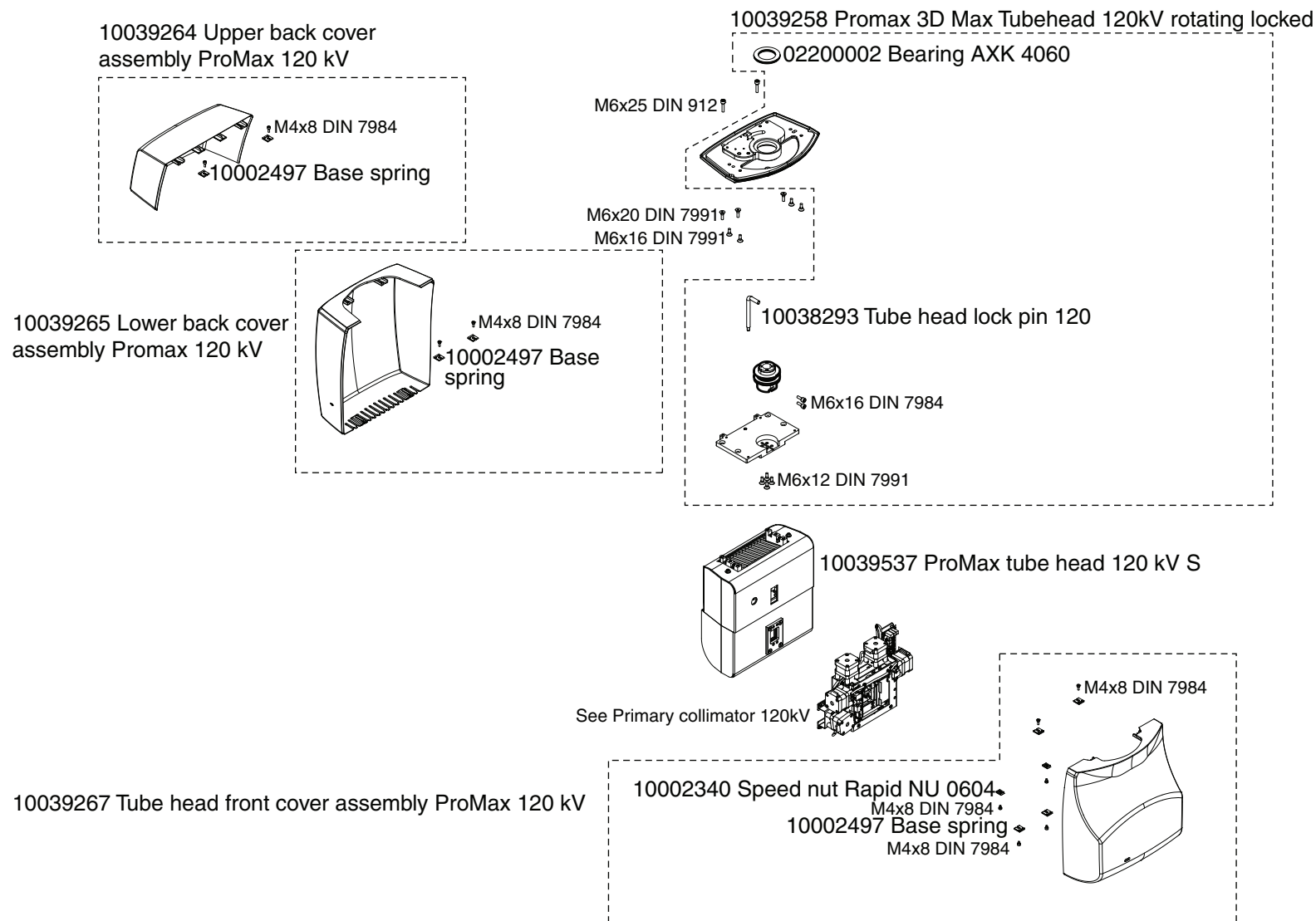


Rotating tube head cables

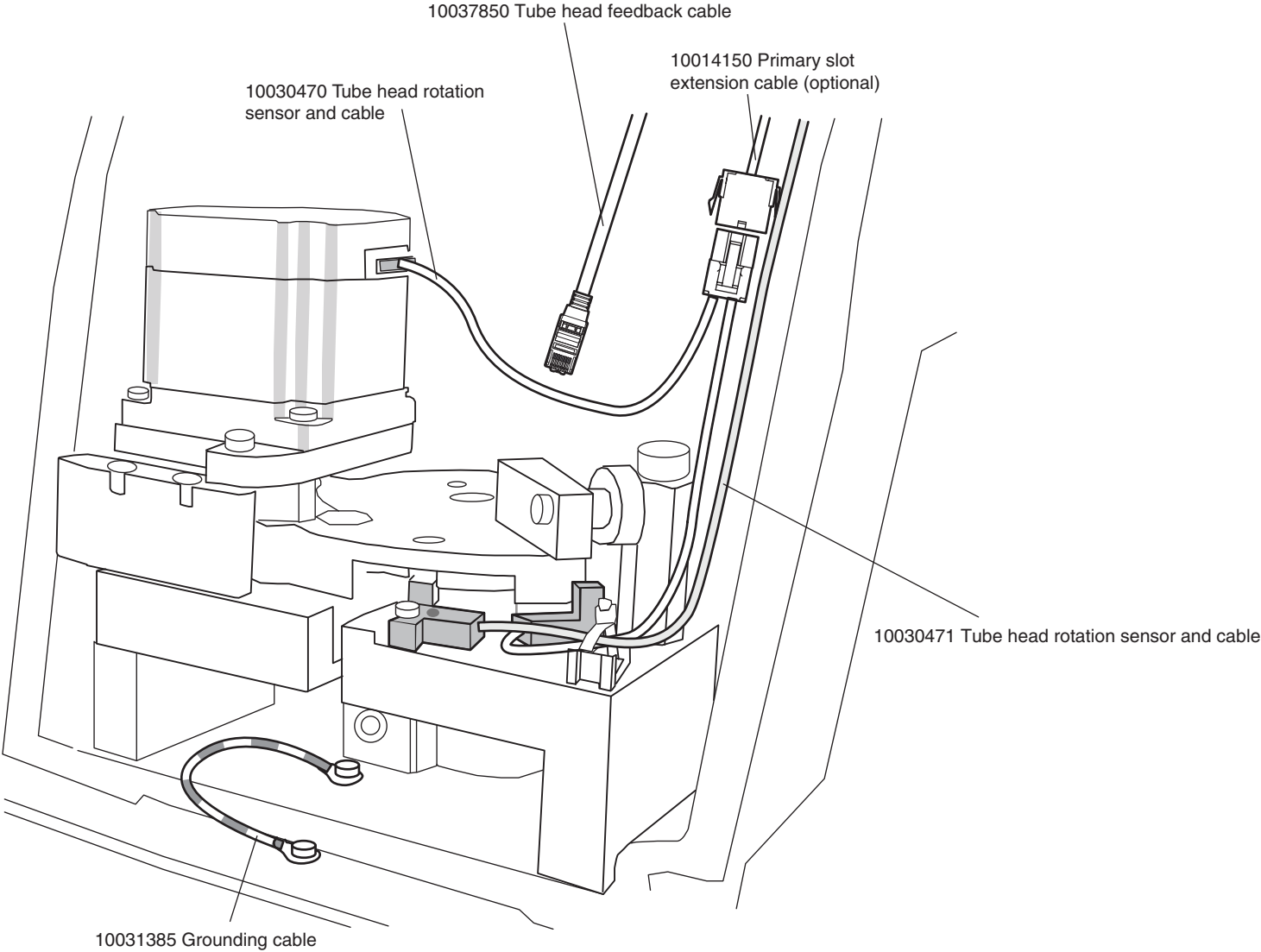


Primary collimator, 2D digital


10004091 Primary collimator, 2D digital

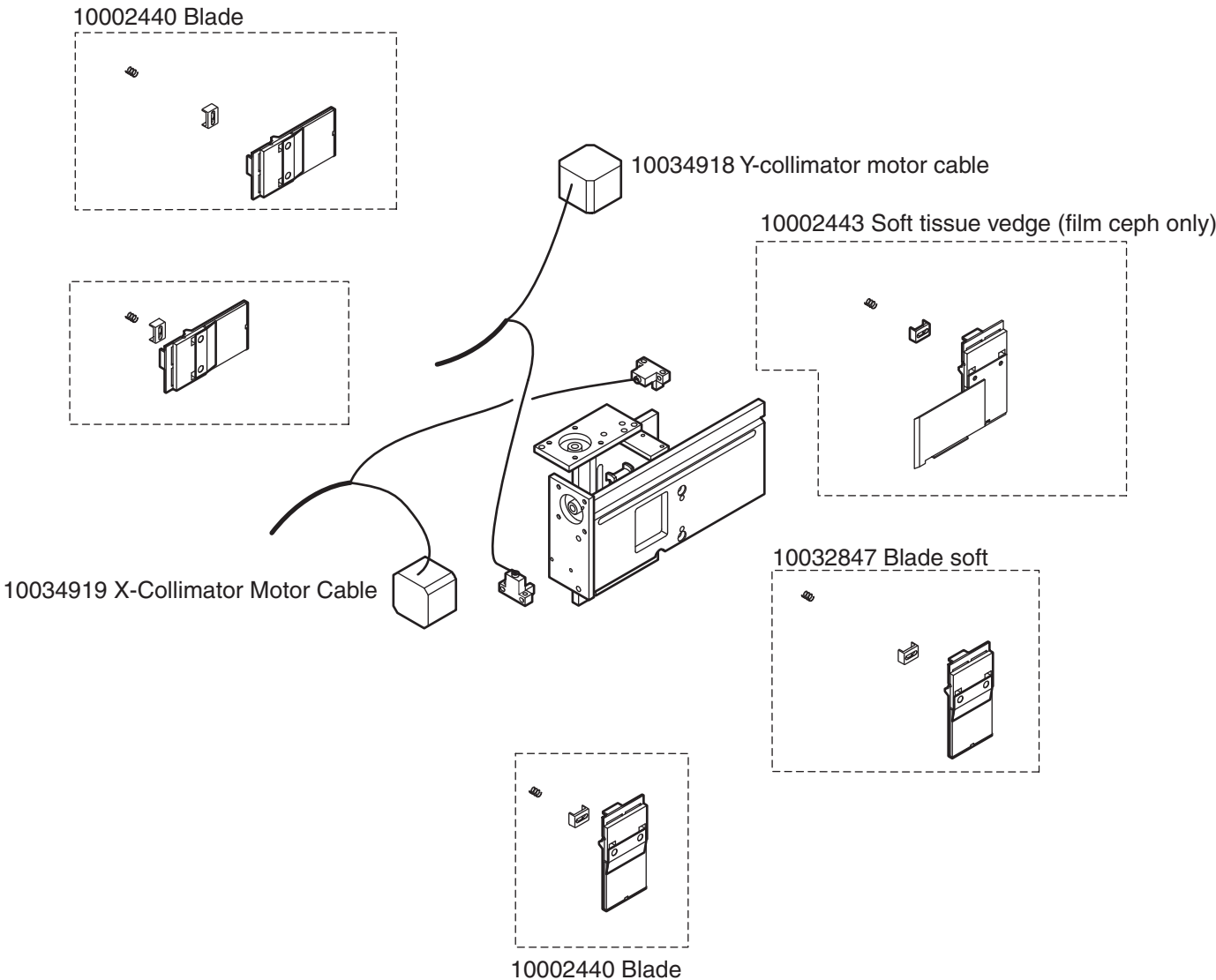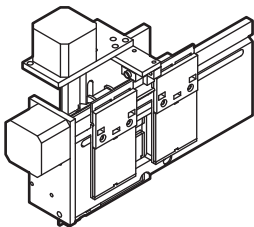

Primary collimator Cu, 3D s, 3D Classic, 3D Plus, 3D Mid

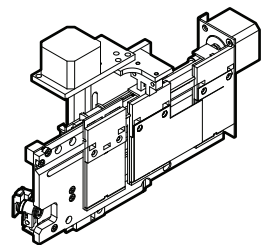


10023718 Primary collimator Va Cu

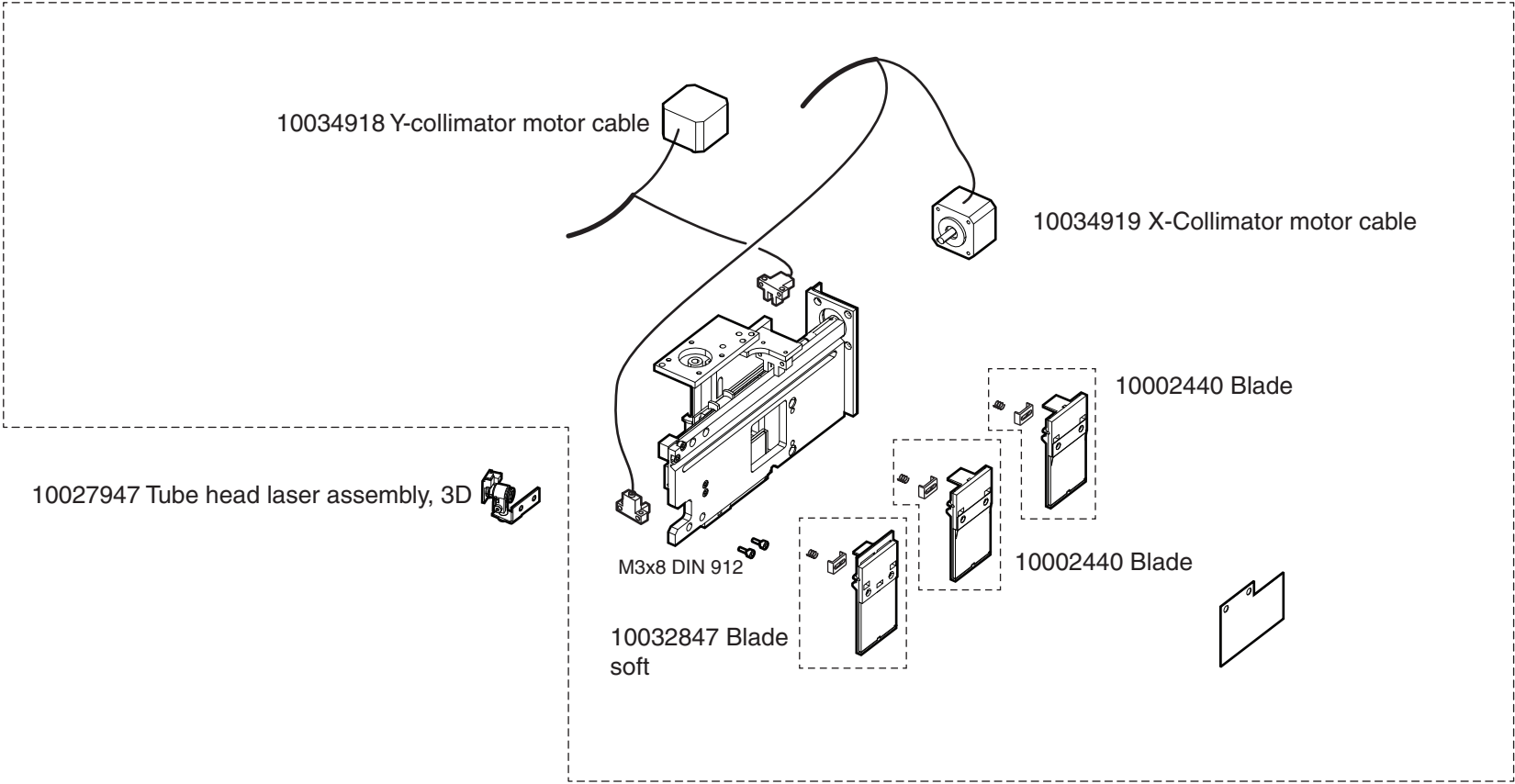


Primary collimator, 3D Max

10035729 Primary collimator 3D Max multi

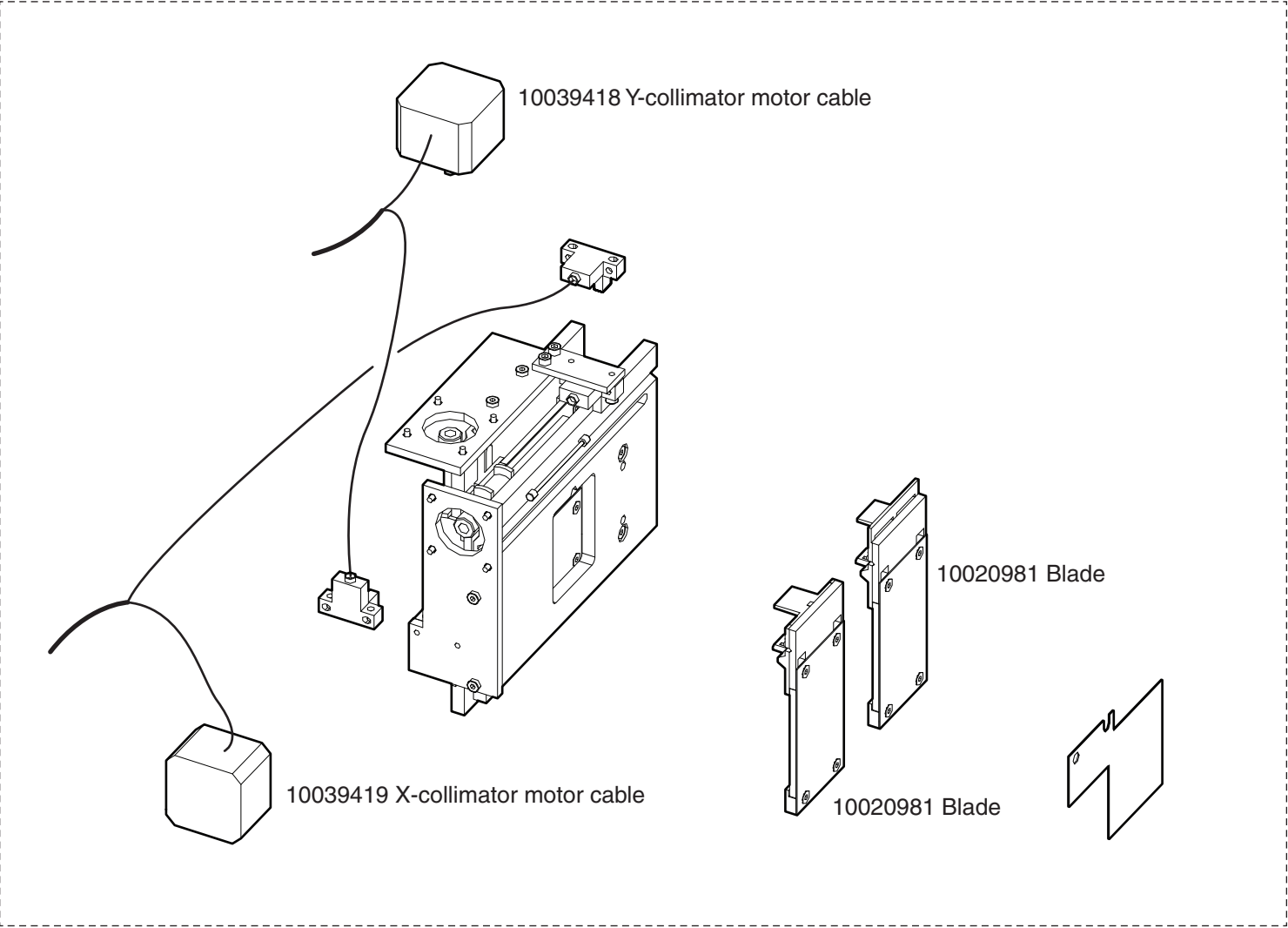


Primary collimator, 120kV

10037750 ProMax 120 kV multi collimator assembly

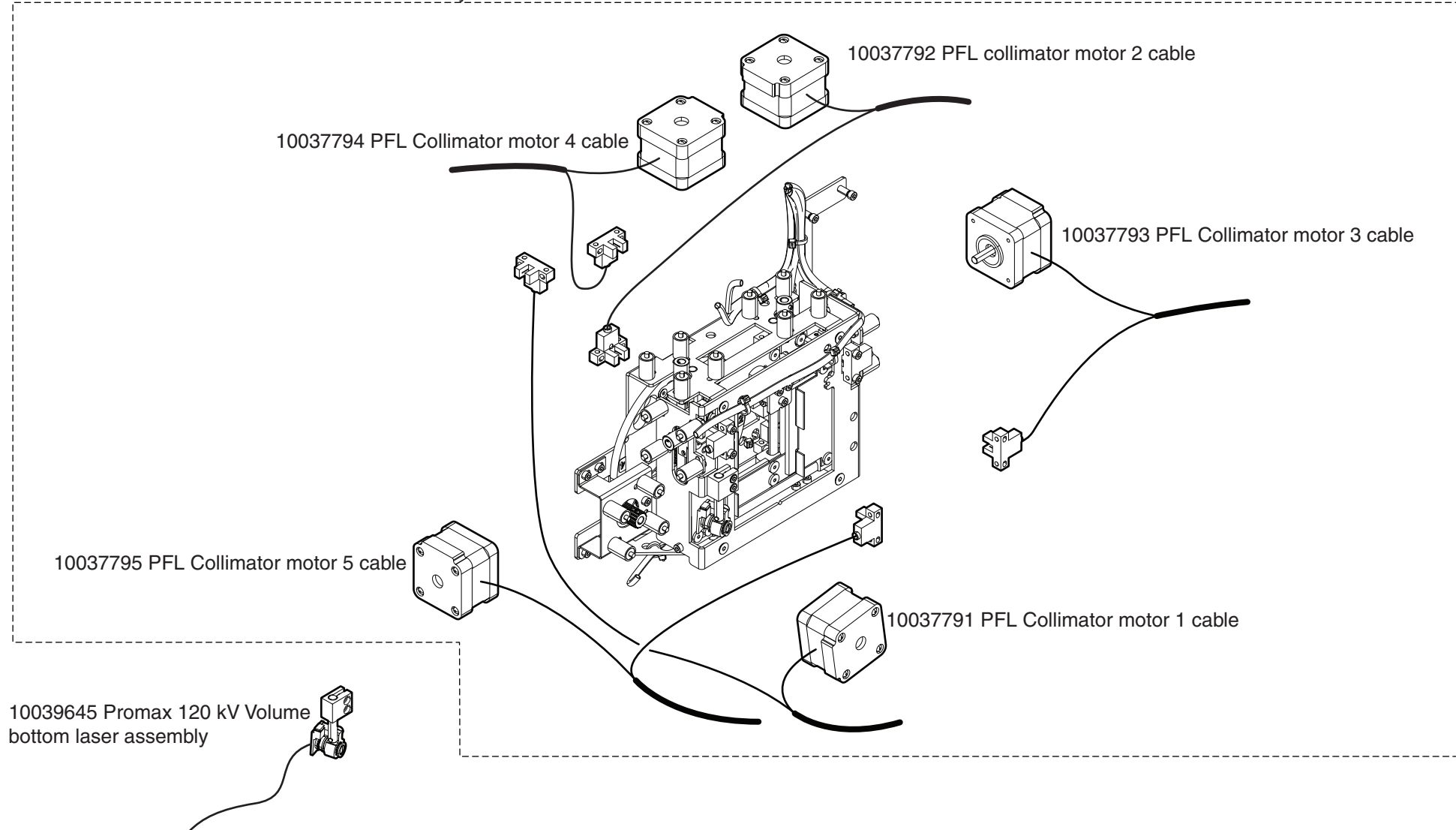



Planmeca ProMax Cephalostat

PLANMECA

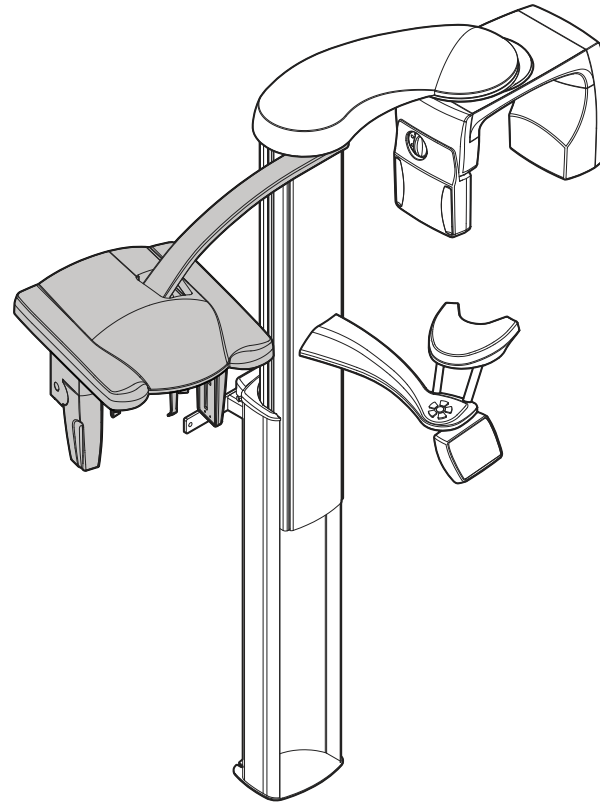


Scanning housings and arm


NOTE! For installation instructions for a Planmeca ProMax having old covers, see technical bulletin PX-118 at Planmeca dealer support

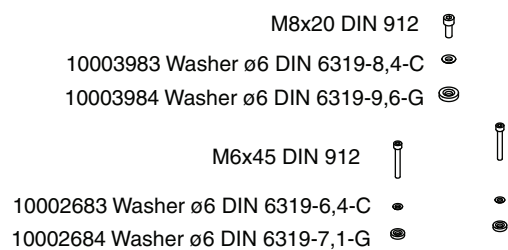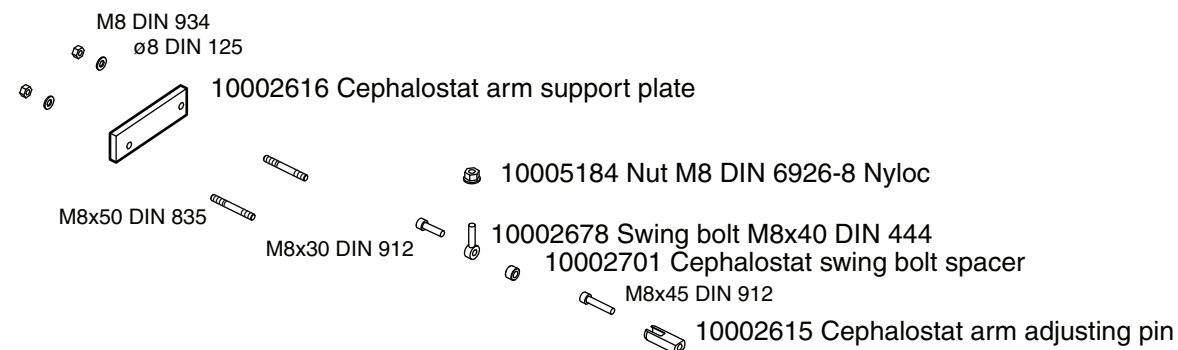


10003045 Cephalostat arm

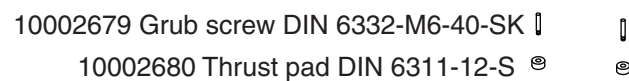

10039466 Ceph covers spare part set

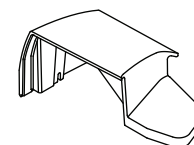


- M30x12 WN1423 Zn (7 pcs)
- M30x8 WN1451 Zn (12 pcs)
- 10001562 Mounting spring (6 pcs)
- 10004530 Ball head screw Rapid R6213 M5
- 10038289 Ceph rear cover attachment spacer
- 10037104 Extension nut 35mm (4 pcs)
- 10037105 Extension nut 50mm (2 pcs)

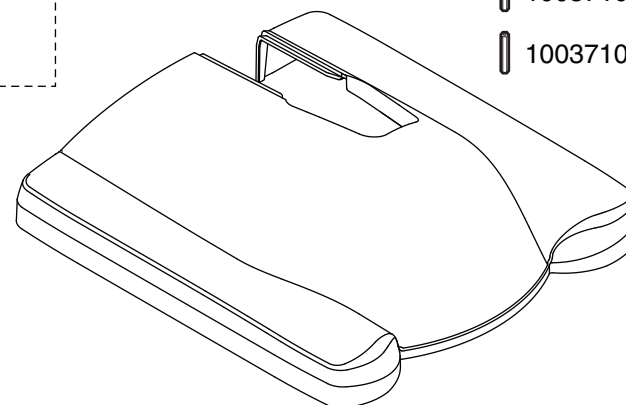


Scanning mechanism


10003682 Digital cephalostat scanning mechanism

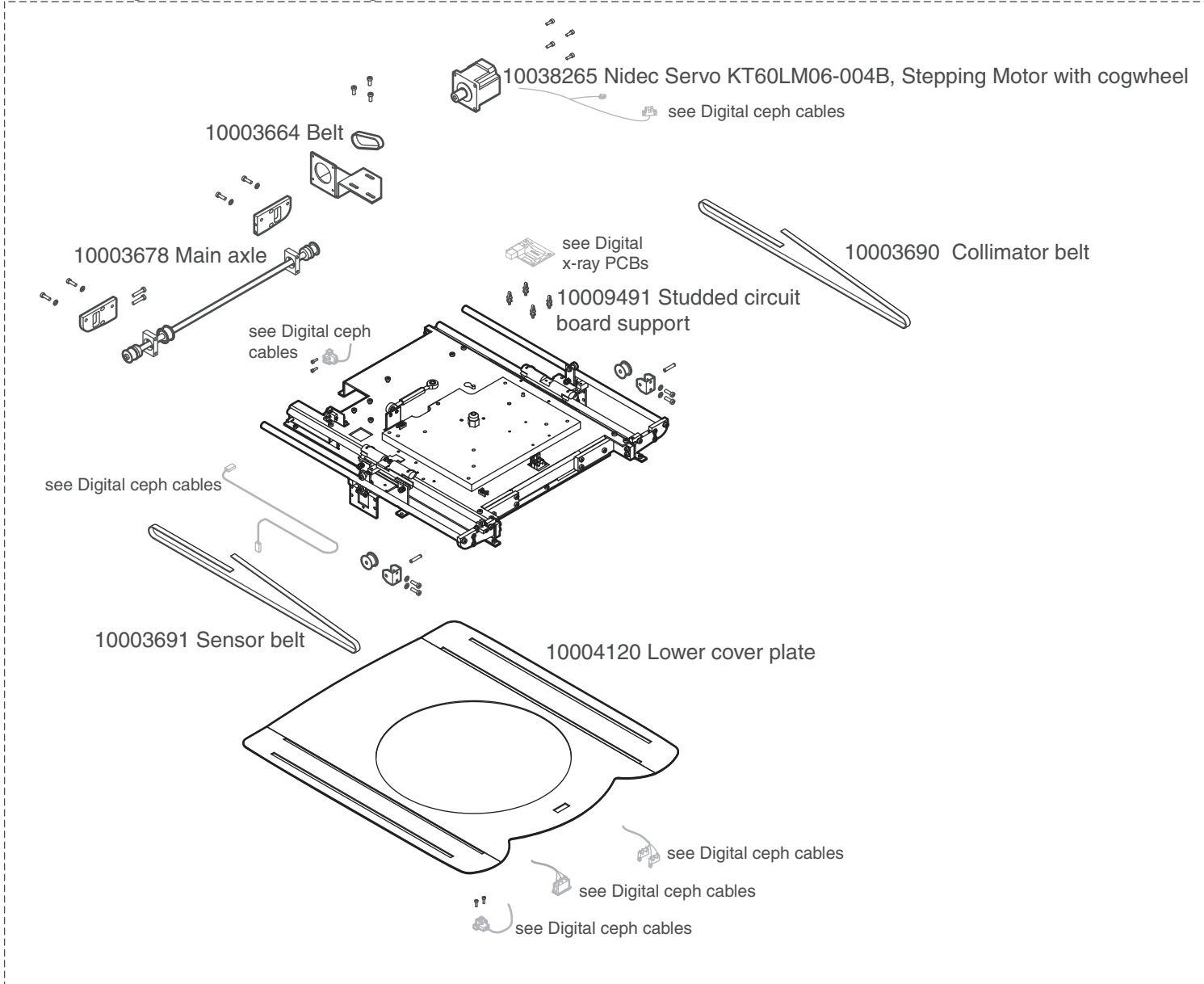


Scanning cables

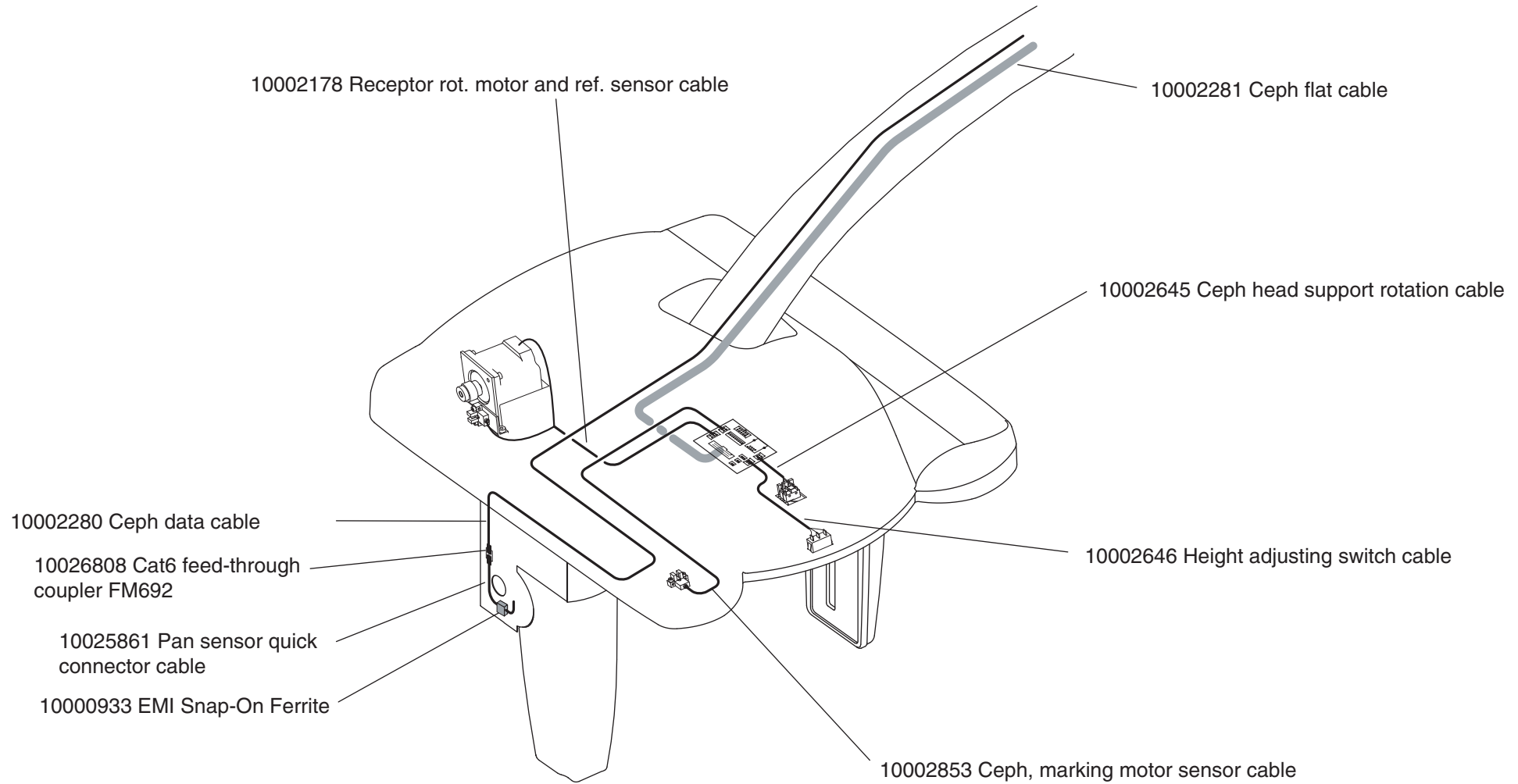

Head support


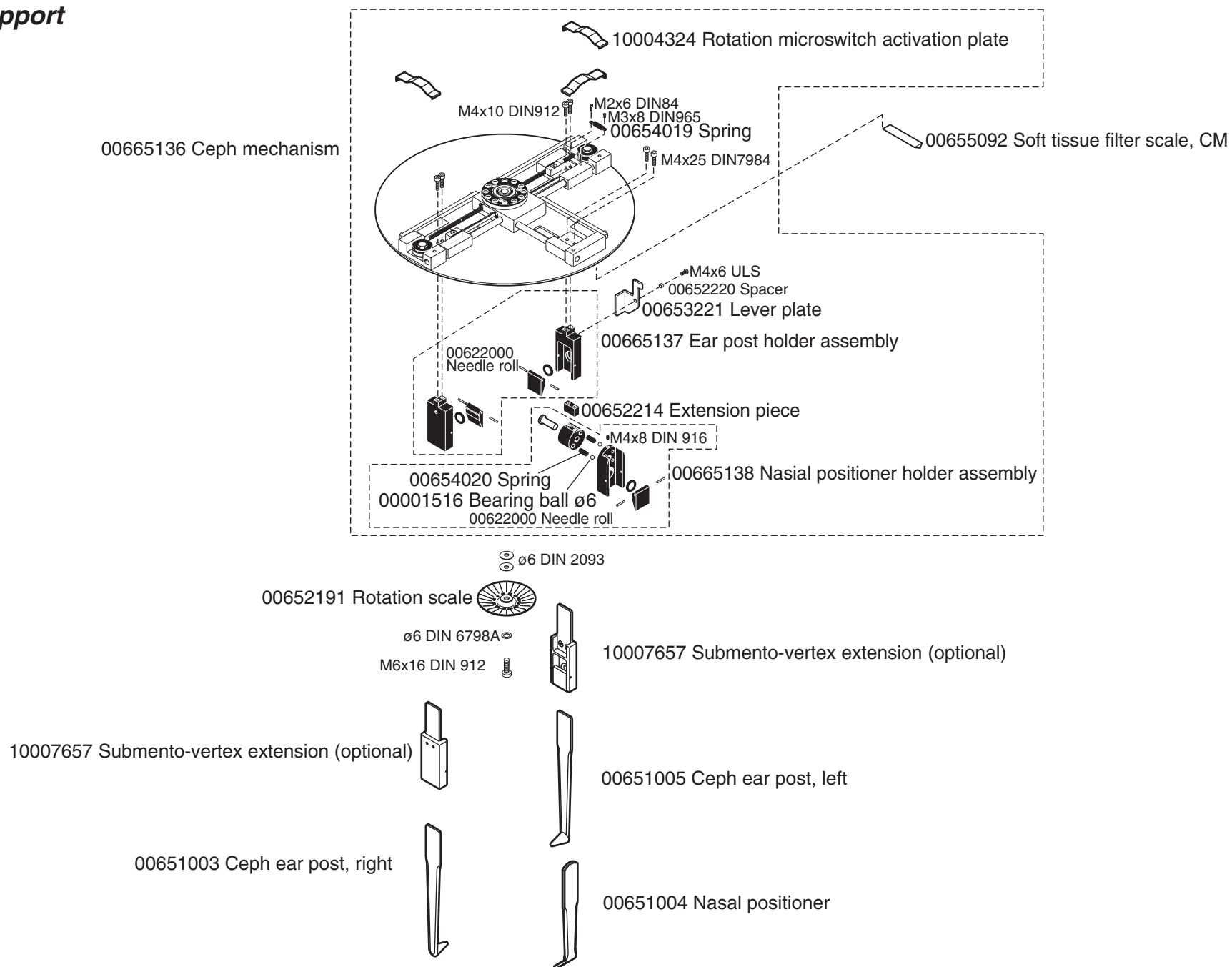


Housings and arm, ProCeph and film

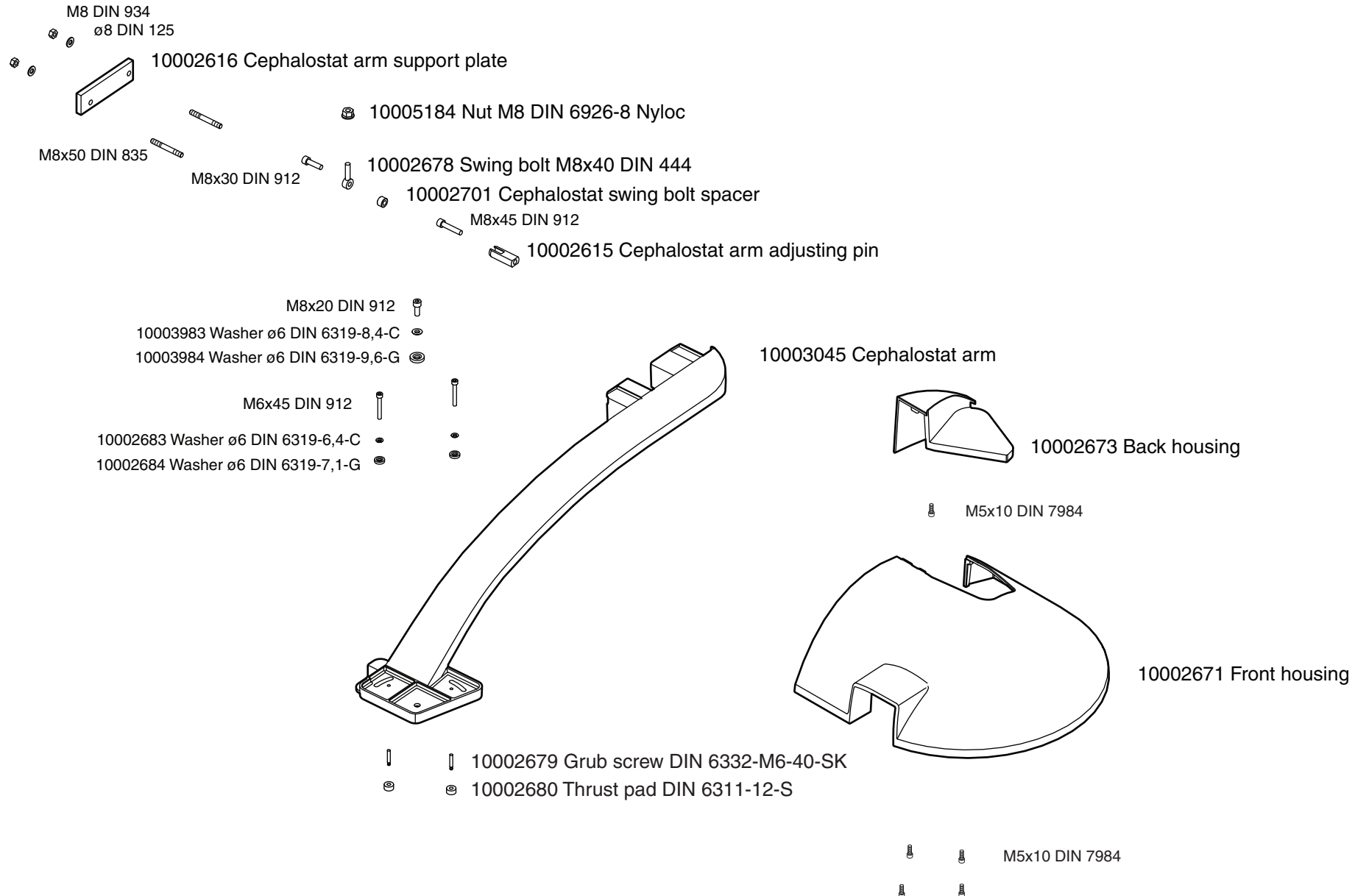


ProCeph body and sensor assembly


10032321 ProCeph body assembly

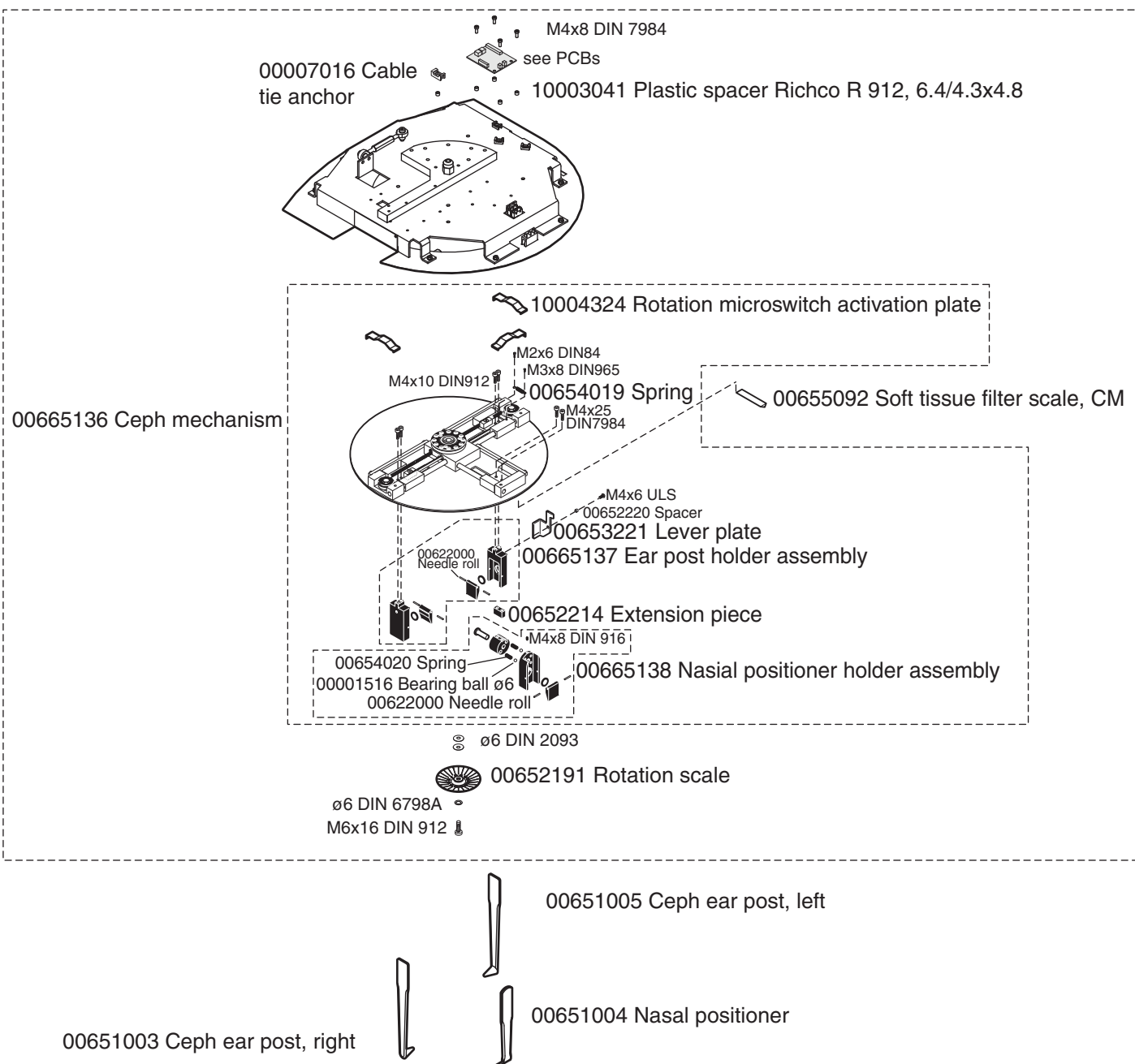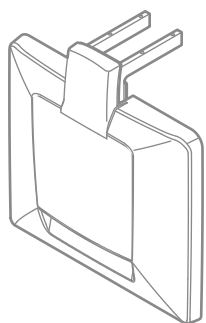


Primary collimator 3D, ProCeph

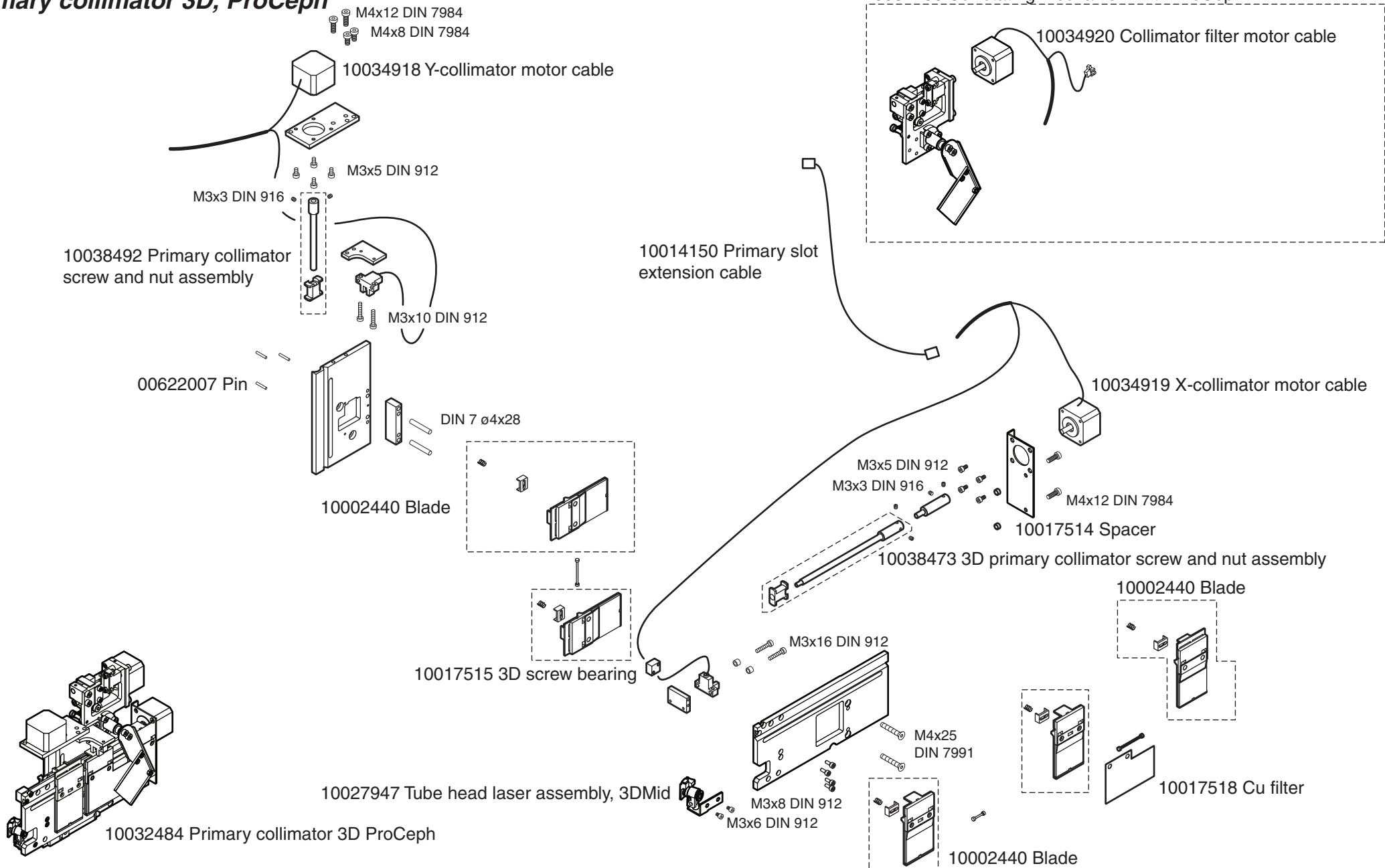

ProCeph cables

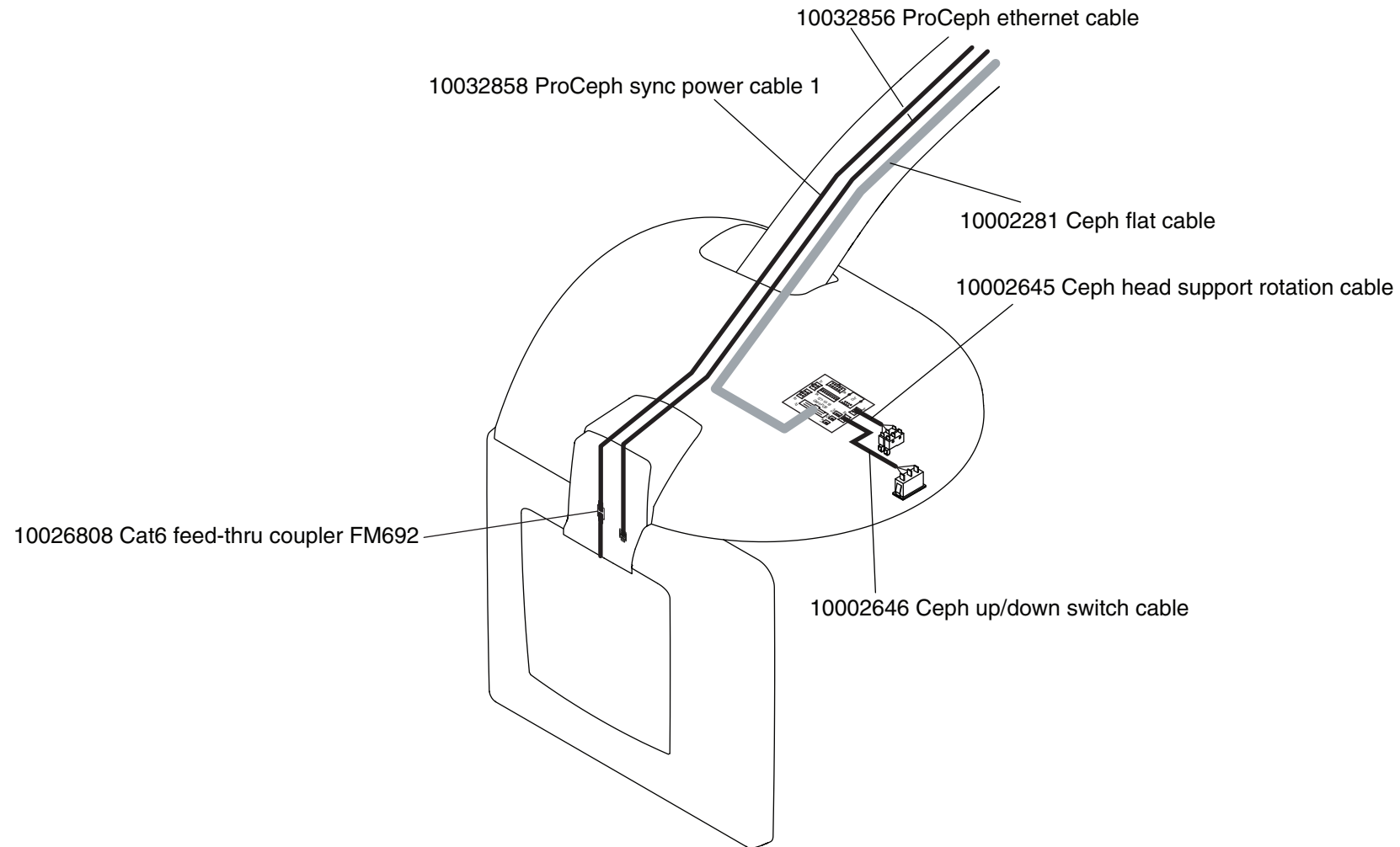


Movable sensor head connector

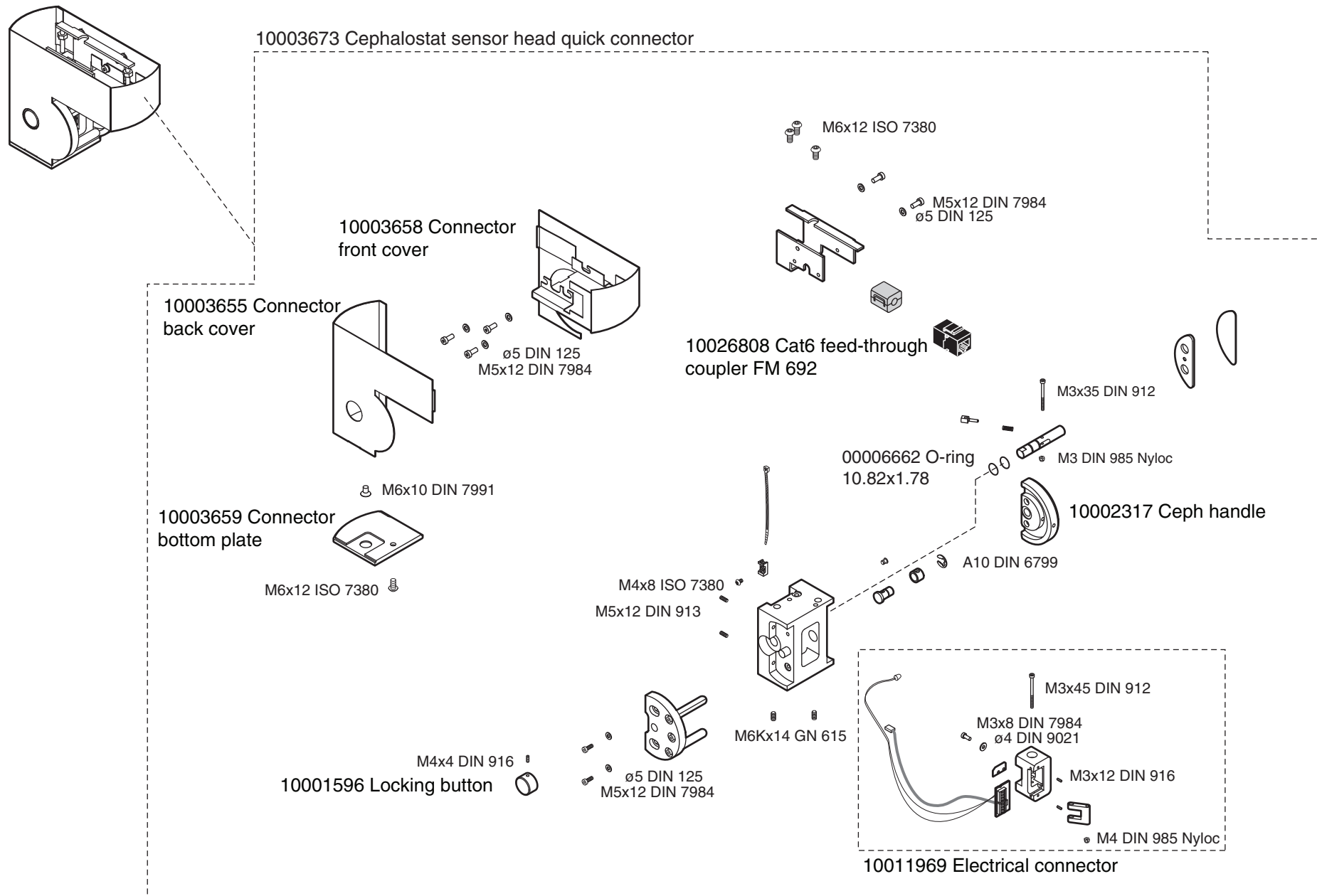


Ceph second primary collimator

10028060 Second primary collimator Dimax 4

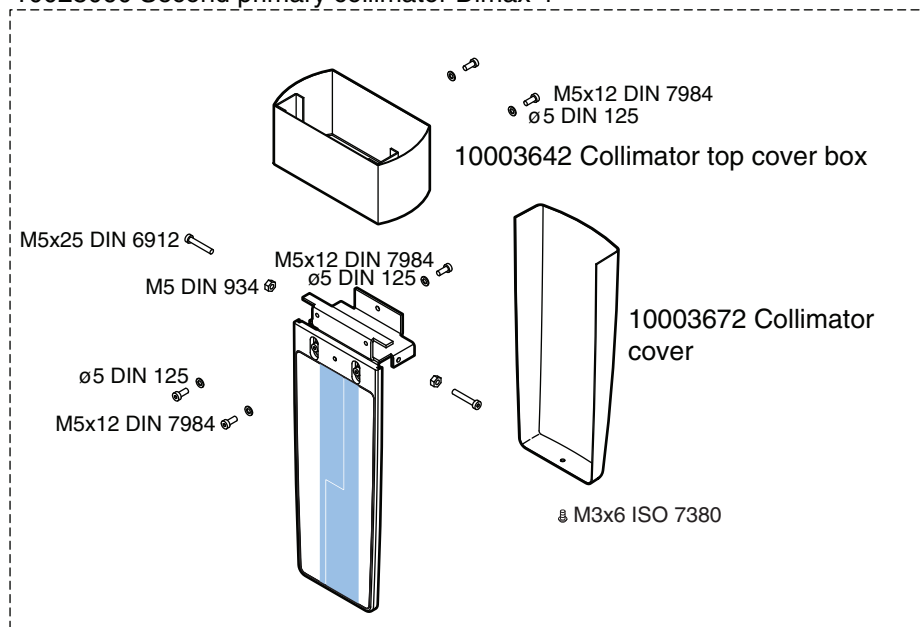


10003675 Second primary collimator Dimax 3

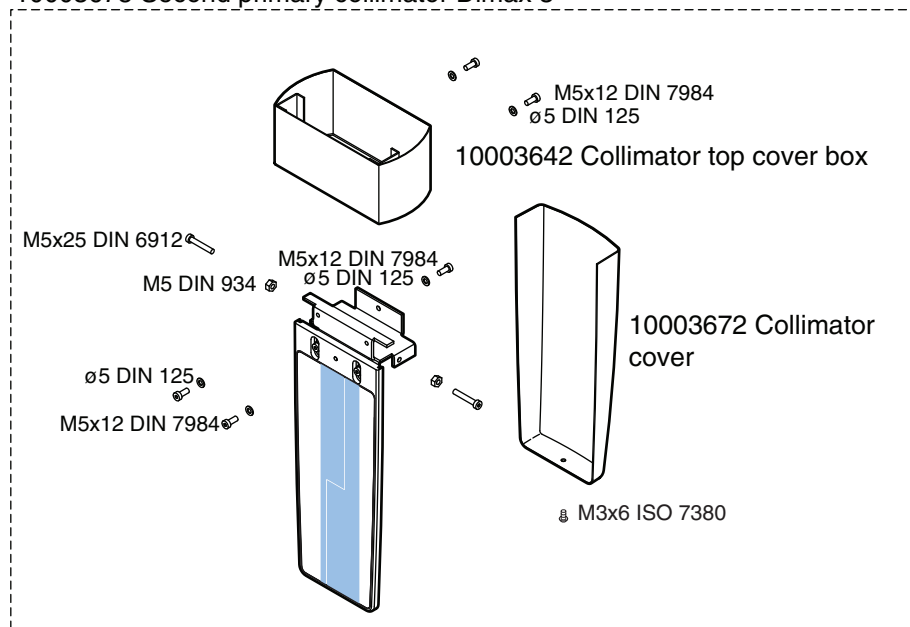


Planmeca ProMax PCBs


PLANMECA

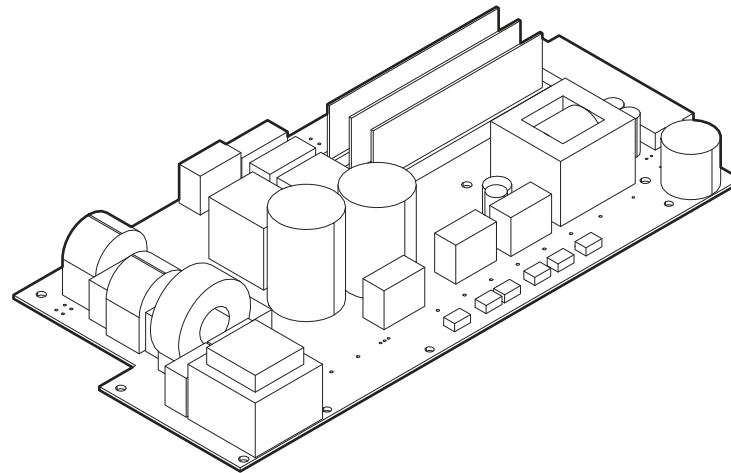


PCBs, 2D, 3D s, 3D Classic, 3D Plus, 3D Mid

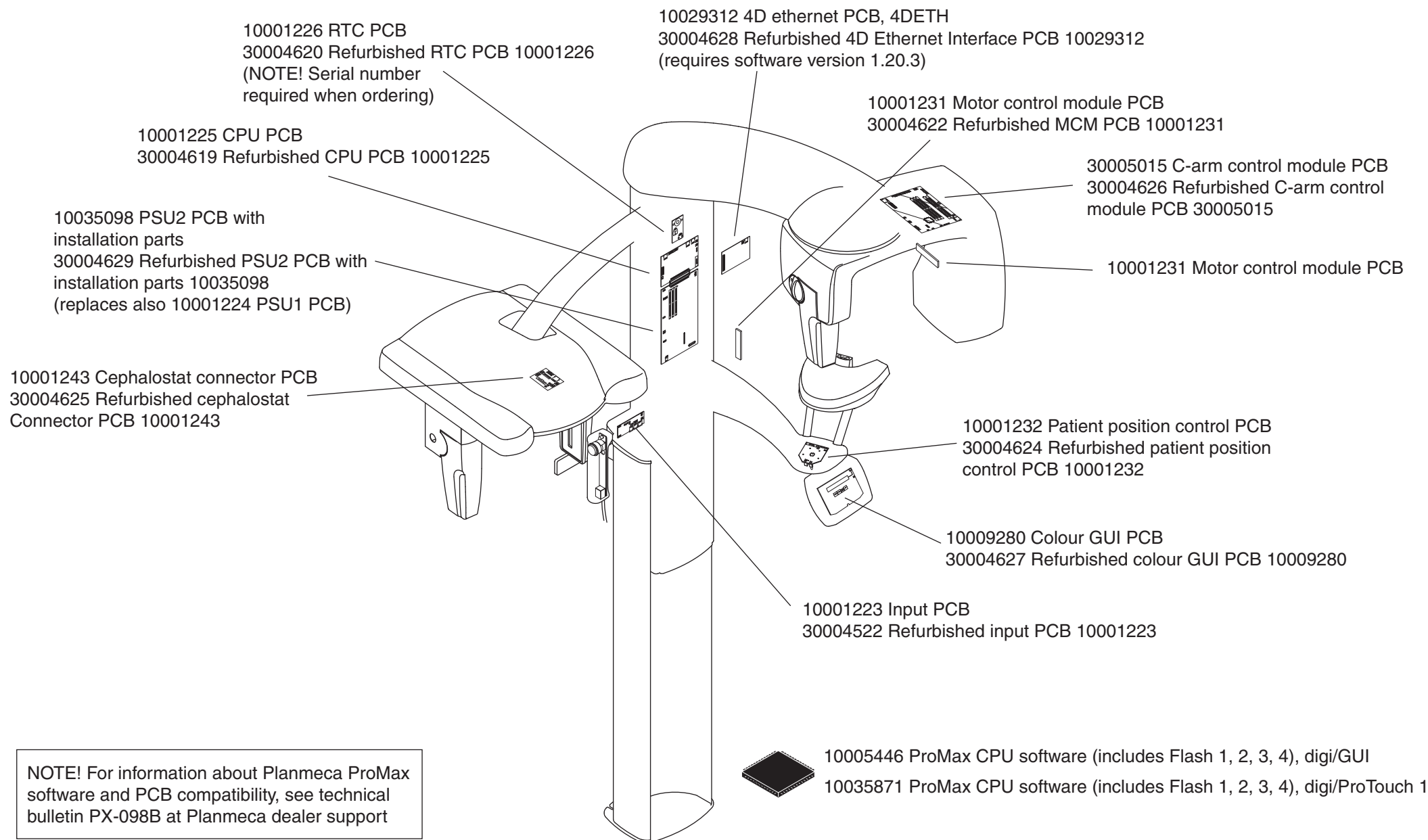

PCBs, 3D Max

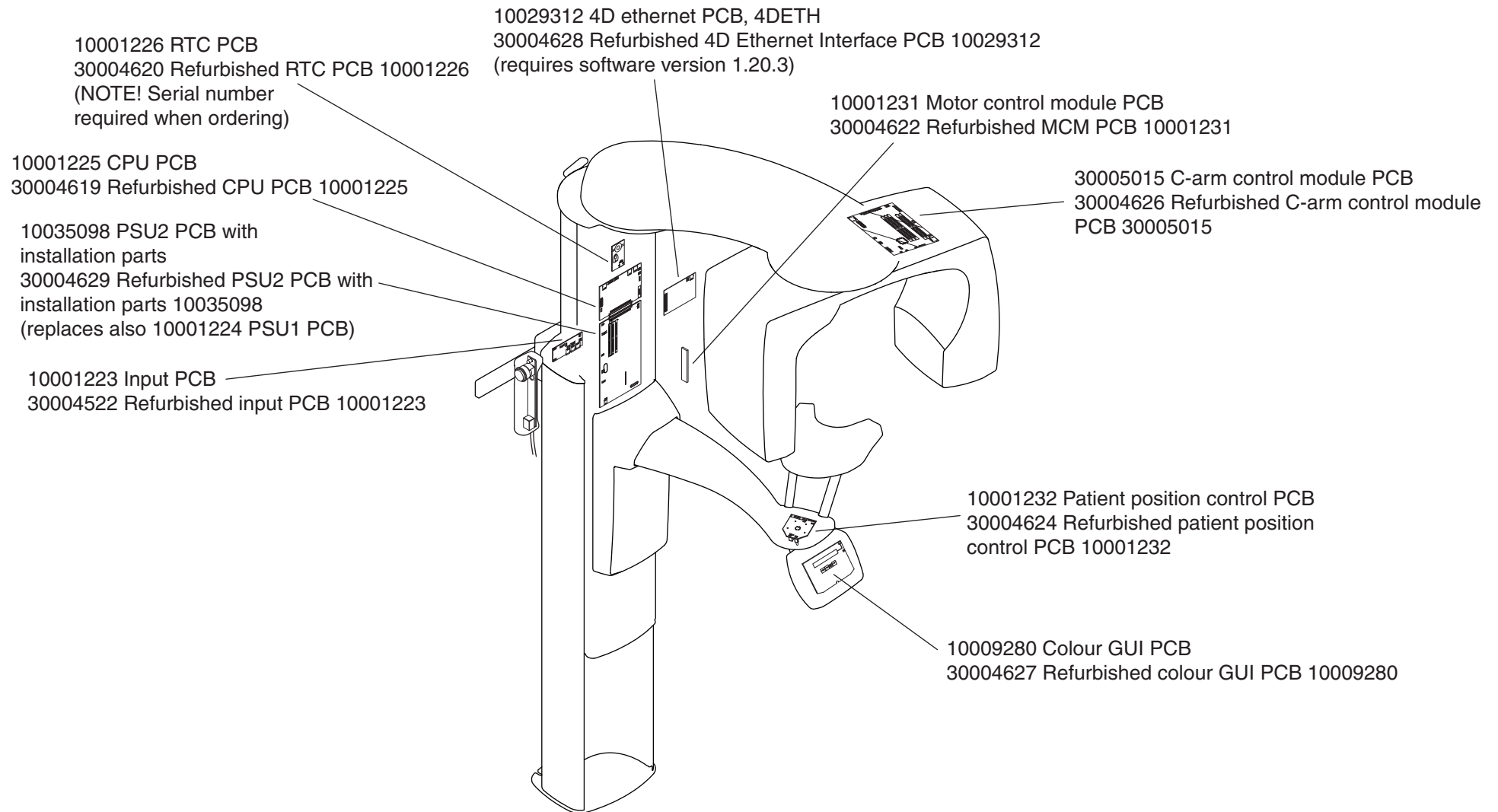

NOTE! For information about Planmeca ProMax software and PCB compatibility, see technical bulletin PX-098B at Planmeca dealer support

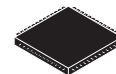


10005446 ProMax CPU software (includes Flash 1, 2, 3, 4), digi/GUI
10035871 ProMax CPU software (includes Flash 1, 2, 3, 4), digi/ProTouch 1

Planmeca ProMax


Quality assurance and installation

PLANMECA

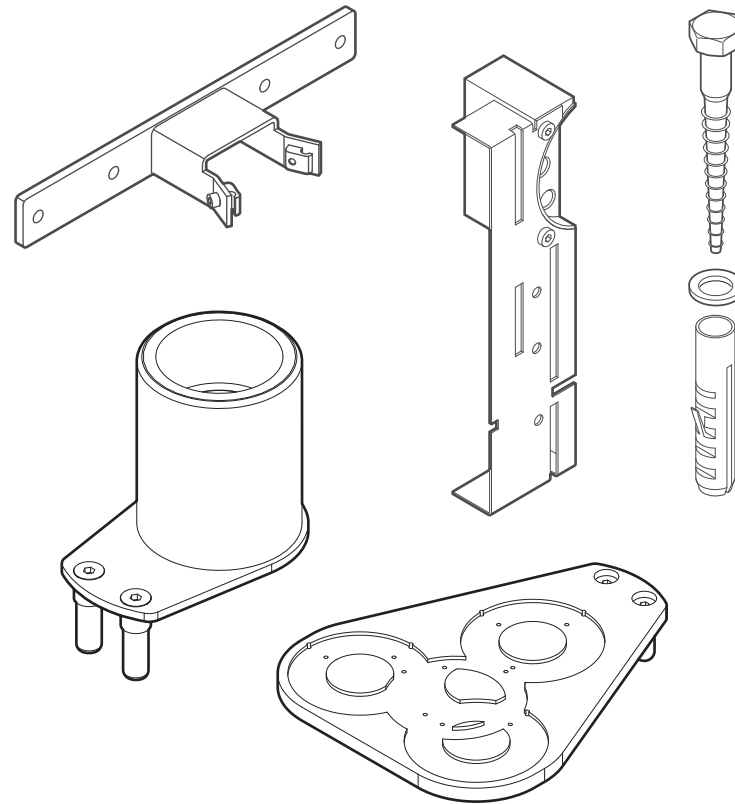


Calibration tools, 2D

10004683 ProMax 2D panoramic calibration tool kit

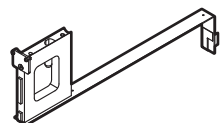

10004225 Alignment ruler

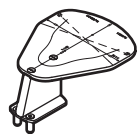

10004011 Ball phantom



00050977 Frankfort plane alignment tool



00654028 Transportation pin (long)



10005786 Feeler



00050972 Beam alignment tool



10029418 Dimax4 sensor alignment tool




10005525 Torx key TX25



10005526 Torx key TX08



10003926 Software loading cable

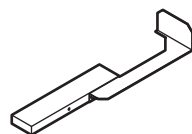


10004316 Dimax2/3 calibration block

10004684 Cephalostat tools, film & digital




10004779 Soft tissue filter positioning tool for film ceph

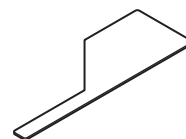

10004246 Cephalostat calibration tool




00653126 Ceph head support alignment tool

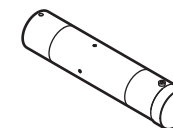


10004499 Ceph head support alignment laser




10002699 Dimax2/3 sensor alignment tool

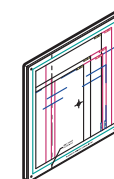

10004148 Cephalometric beam alignment tool for film ceph

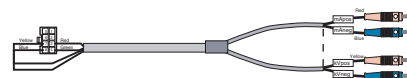

10008320 X-ray tube mA/Kv measure cable

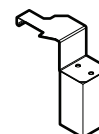


10033669 AEC/DEC calibration block

Calibration tools, 3D and ProFace

ProFace calibration tools




10029539 Geometry calibration platform

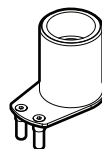

10031542 ProFace 3D Max QA phantom


10032325 ProFace 3D Max chess calibration tool

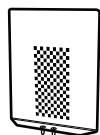


10031692 Chin support heightening piece



10029903 ProFace fantom

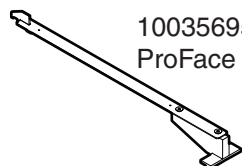

10035695 ProMax 3D Max ProFace alignment ruler

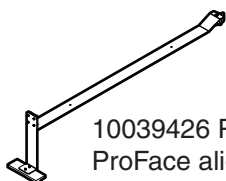

10039426 ProMax 3D Max ProFace alignment ruler

10017554 3D calibration tools kit



10012988 3D QA phantom



10027096 3D geometric phantom




10029539 Geometry calibration platform




10007583 Chin rest spacer




10021621 3DMax geometric phantom

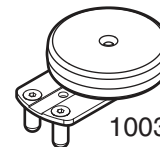


10030330 Guide holder

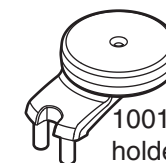


10017436 3D Max guide holder

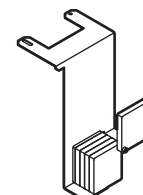


10022826 3DMax flat field phantom

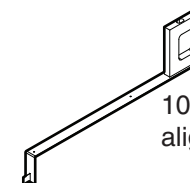


10027094 3D Mid alignment ruler

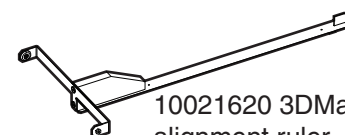


10021620 3DMax alignment ruler

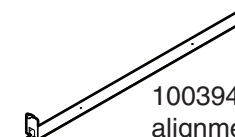

10039424 3D Mid 120kV alignment ruler

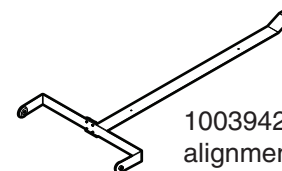


10039425 3D Max 120kV alignment ruler

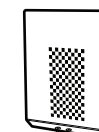


10029575 Proface chess calibration tool

Quality assurance tools


10008252 Quart test phantom set

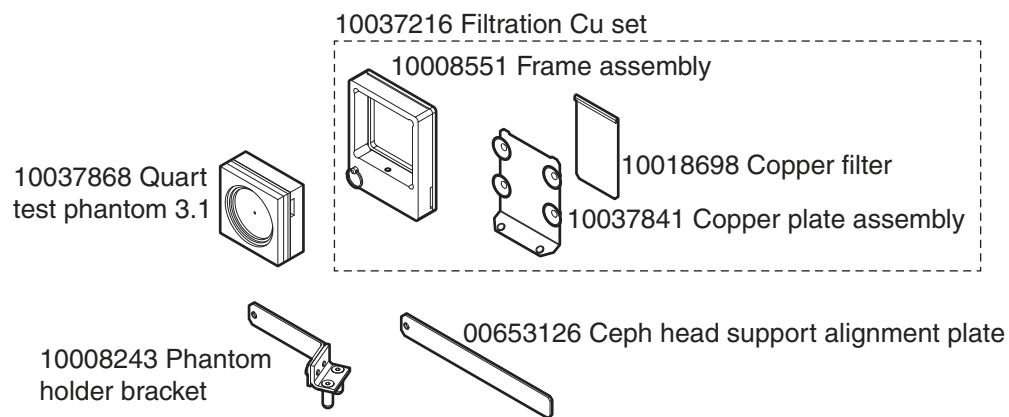




10037069 Quart DE test phantom set

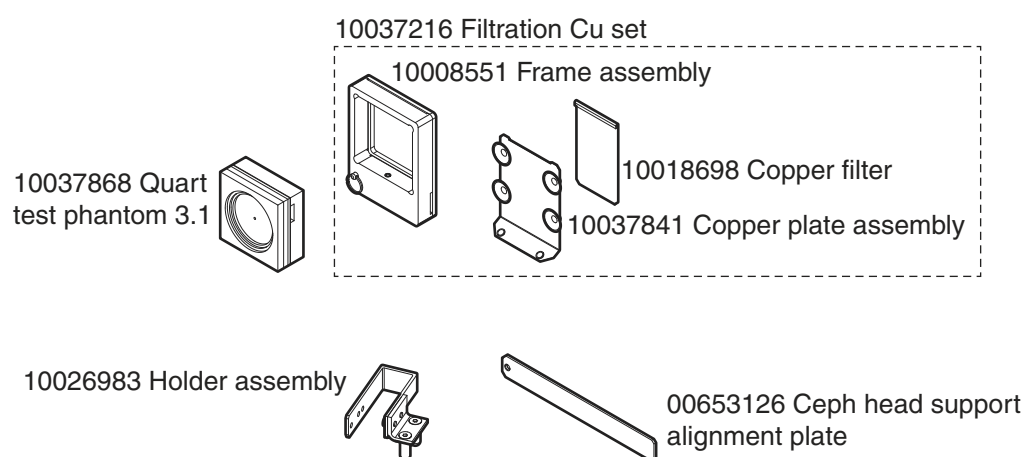

10037432 Quart 3D phantom set for ProMax

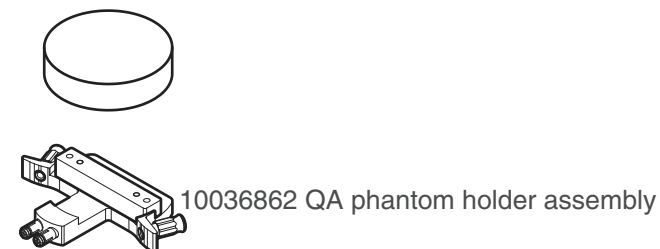


10037435 Quart 3D phantom set, 3D Max

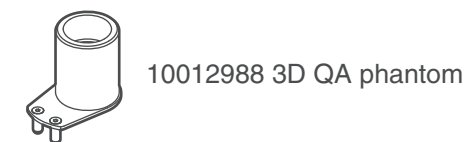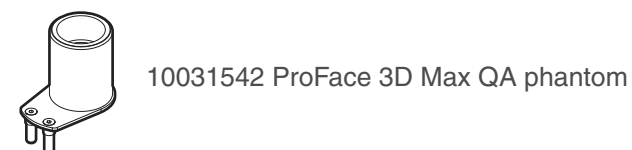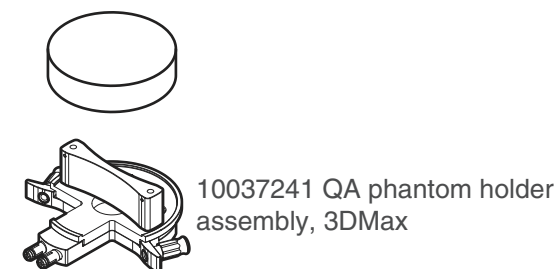


Mounting accessories

10004251 ProMax mounting accessories

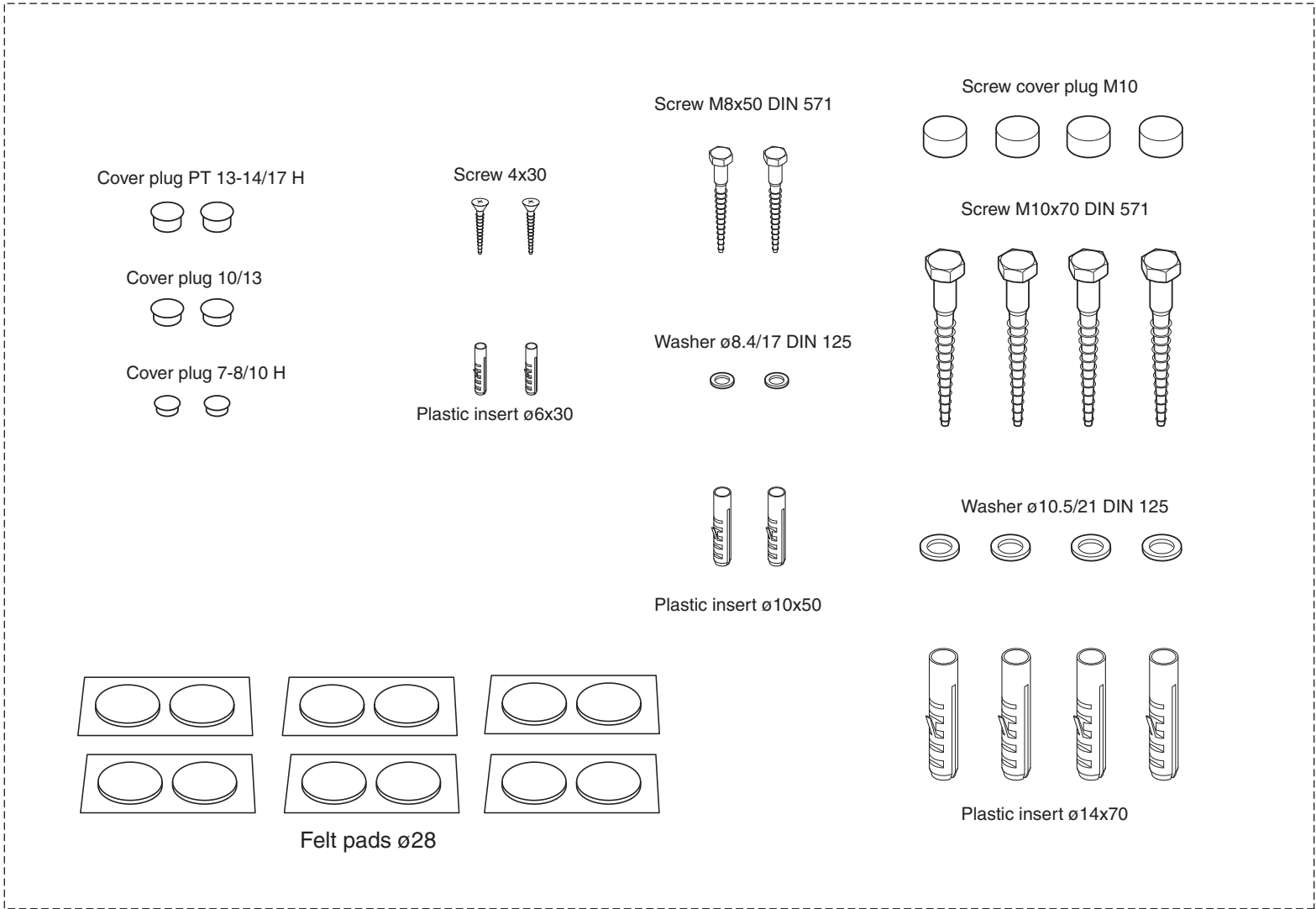


Wall bracket and transportation support bar


10001073 Wall bracket assembly

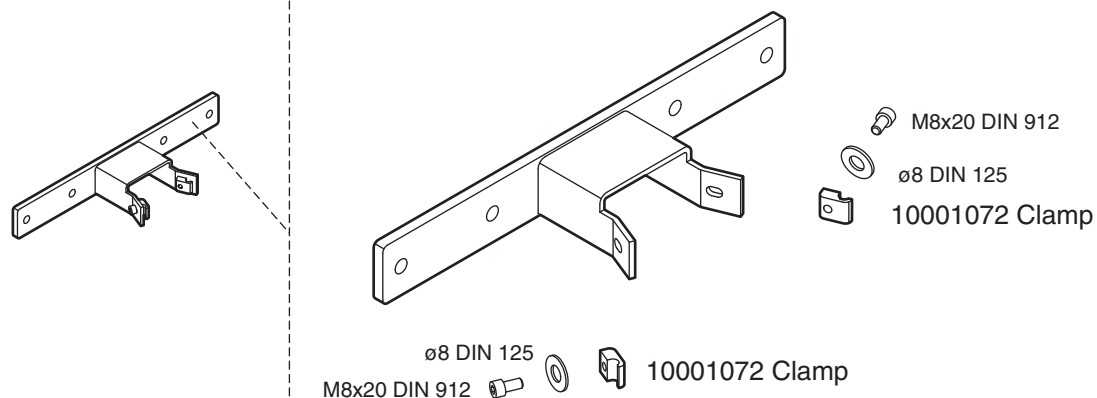

10005439 Wall bracket assembly, USA

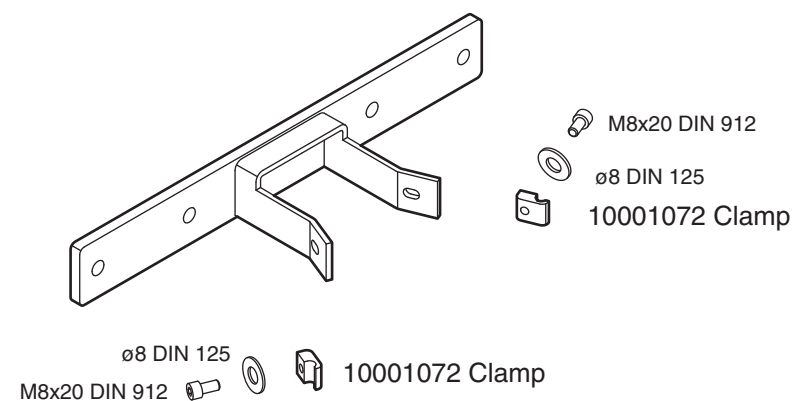


10002621 Transportation support bar assembly

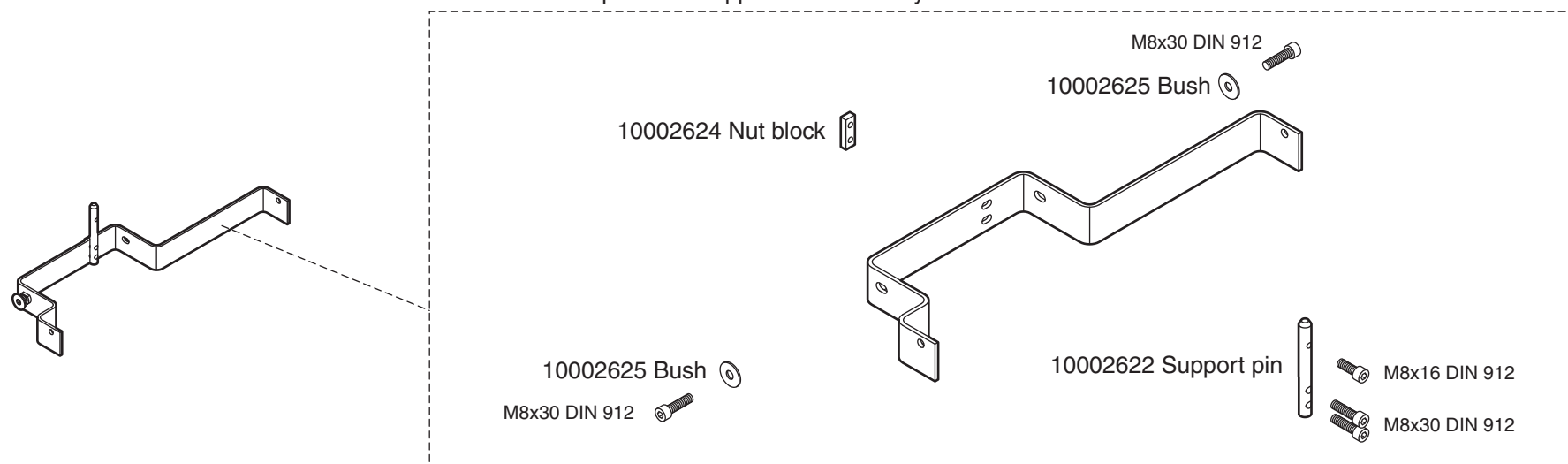

Planmeca ProMax


Additional options

PLANMECA

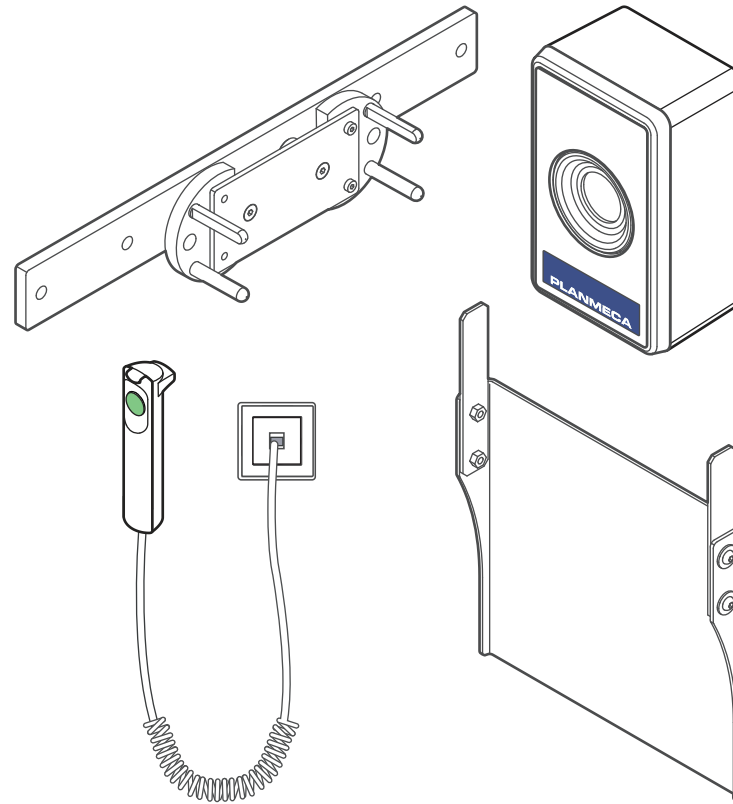


Wall exposure switch

10031246 Wall exposure switch EU

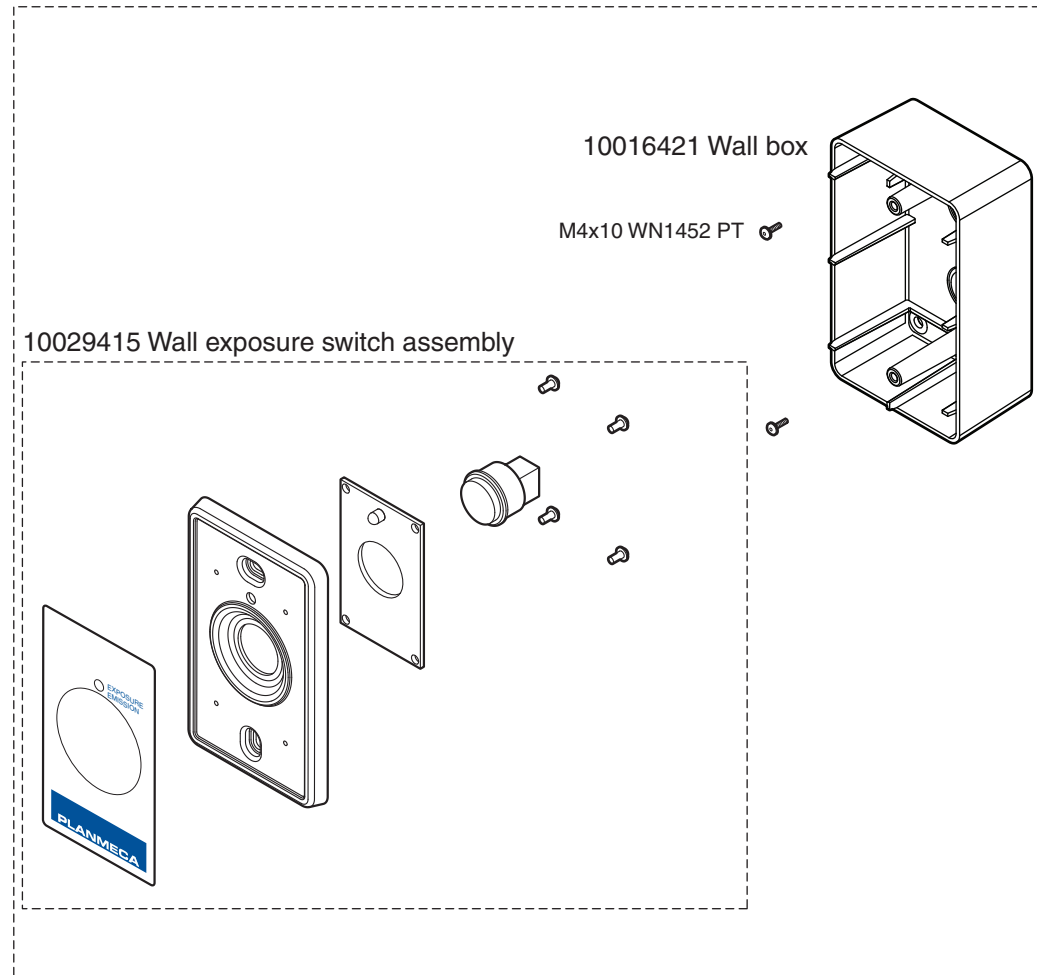


10030466 Additional wall exposure switch

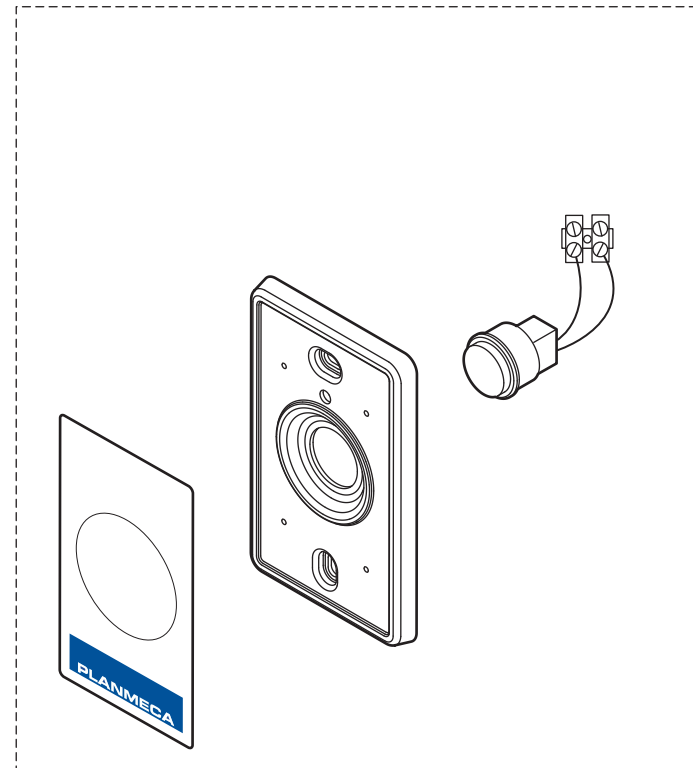


Exposure lamp

10031248 Exposure lamp connection EU

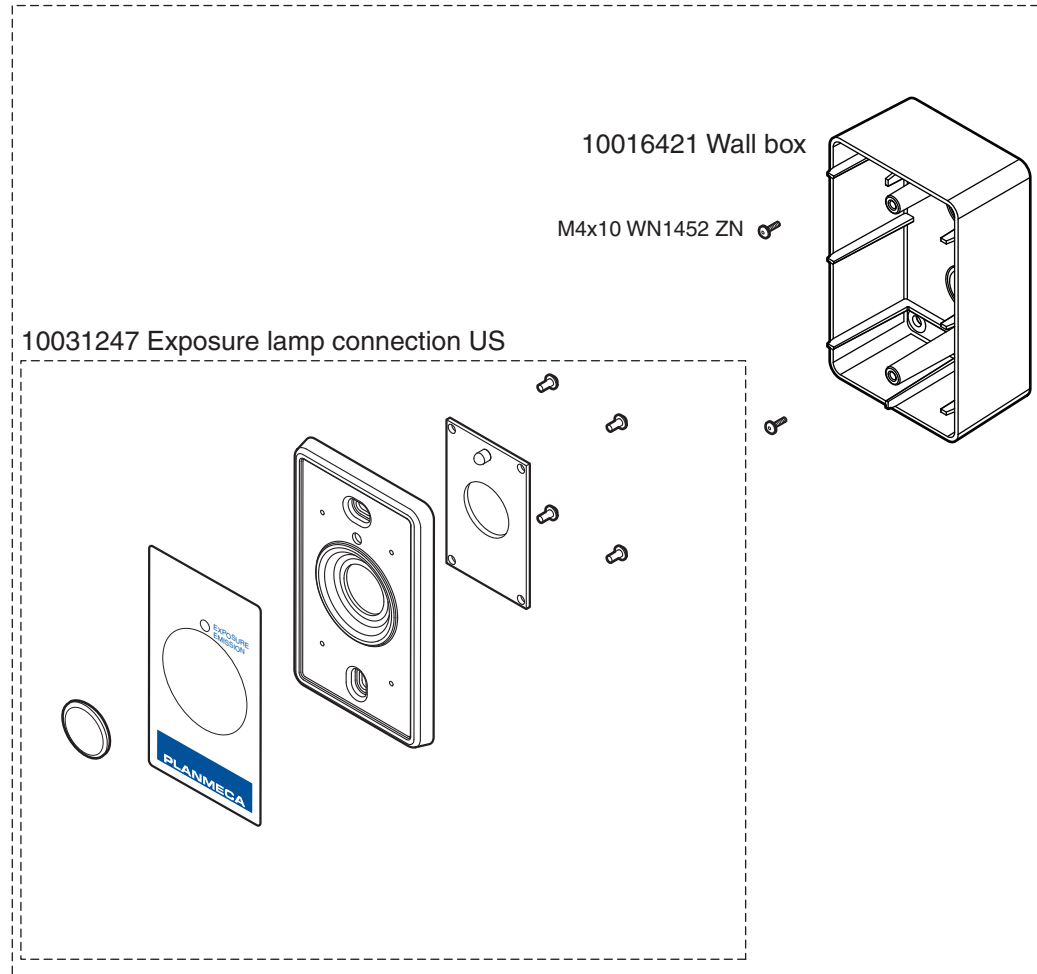


External cables

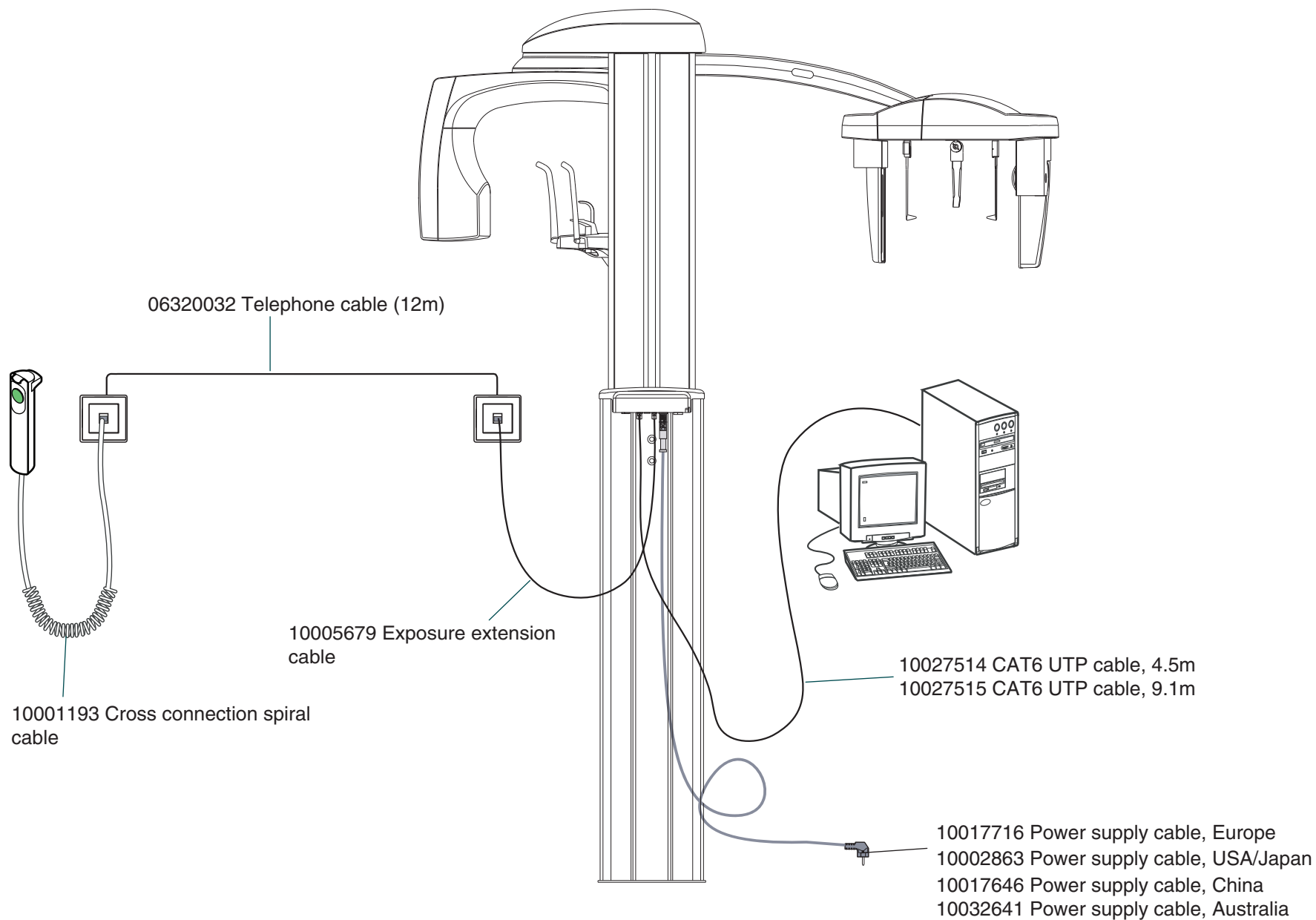


External cables, ProMax V2

NOTE! Planmeca ProMax including new power supply cable and input module has a letter 'V' in the serial number (RPXV..., TPXV..., NTPV...).

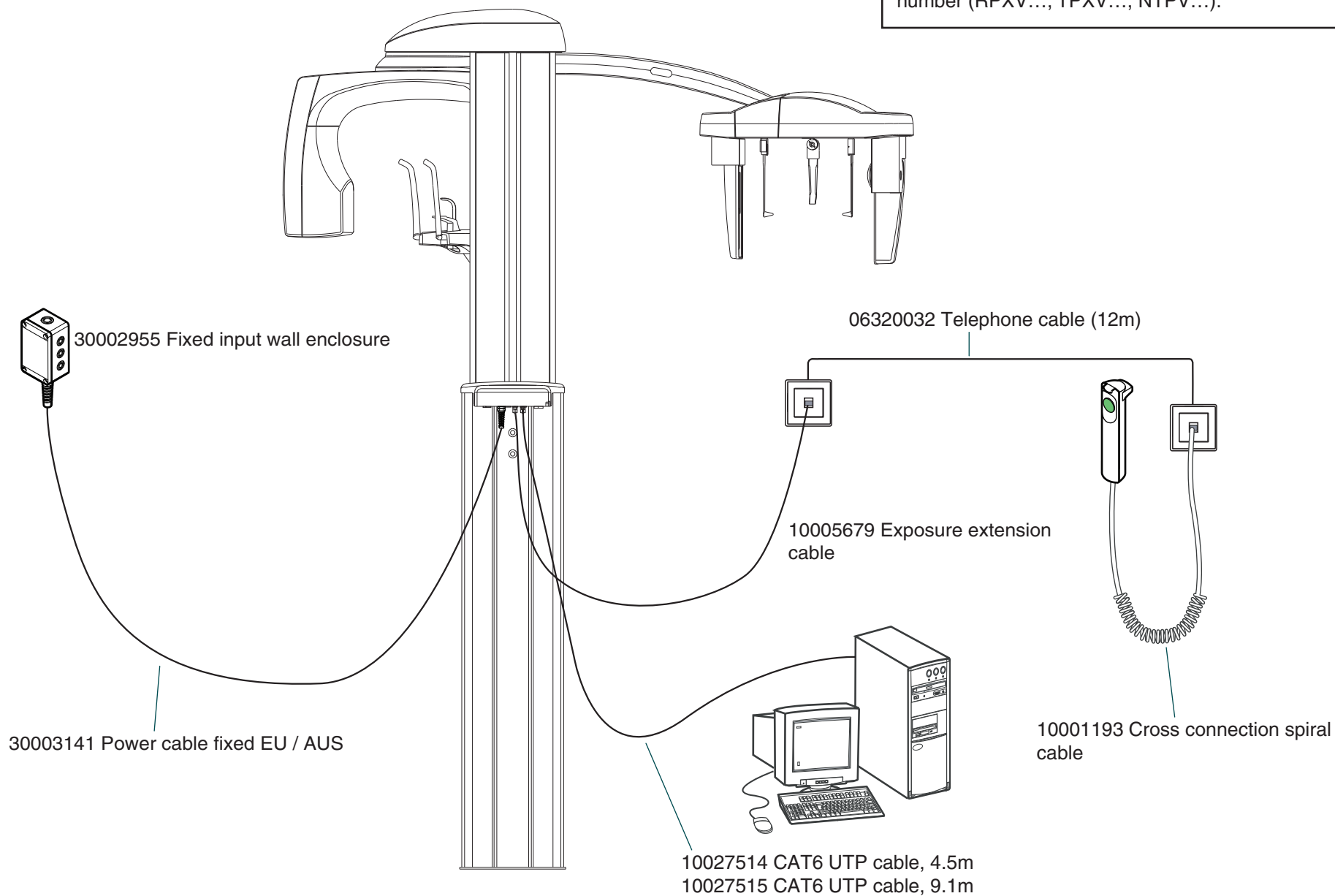


Imaging support

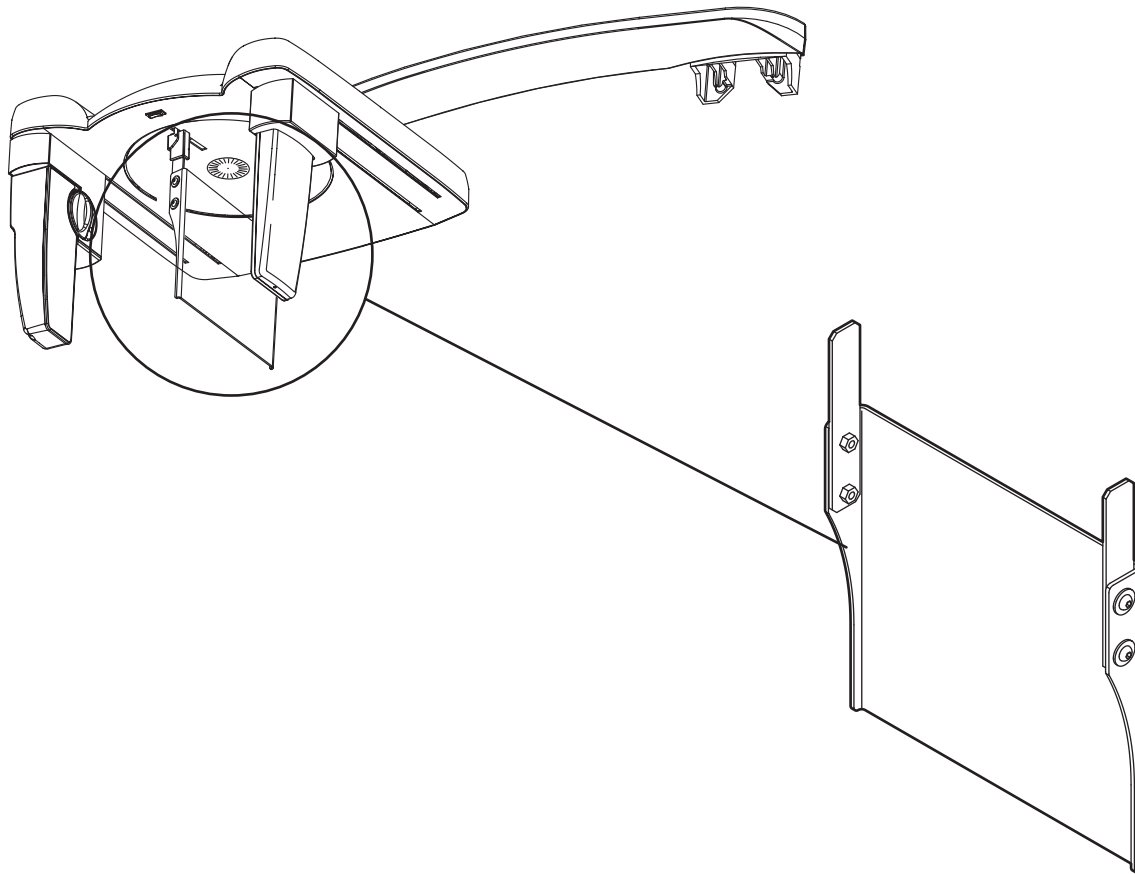


10004413 Hand imaging support

Sensor head wall holder

10014554 Sensor head wall holder

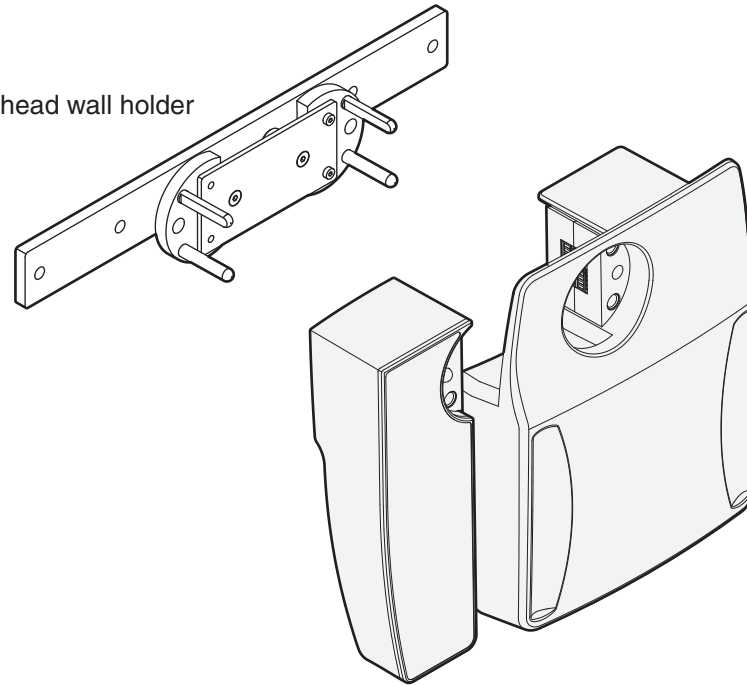

Planmeca 4D Jaw motion

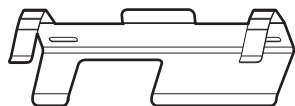


10040193 Light cover Mid 1




10040194 Light cover Mid 2




10040246 Light cover Max 1

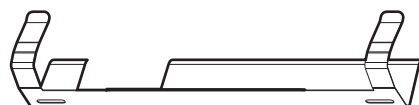


10040248 Light cover Max 2

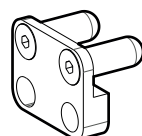

30002373 Extension piece

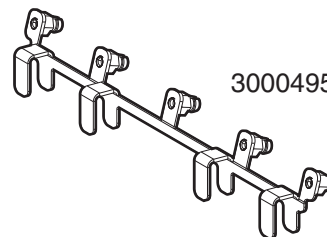

30004952 Maxilla tracking device set (5pcs)

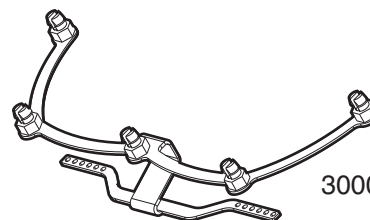

30004953 Mandibular tracking device set (5pcs)




10040227 Protective eyeglasses terminator gray




30004954 Reflective marker set (25pcs)

Planmeca ProMax Film


PLANMECA

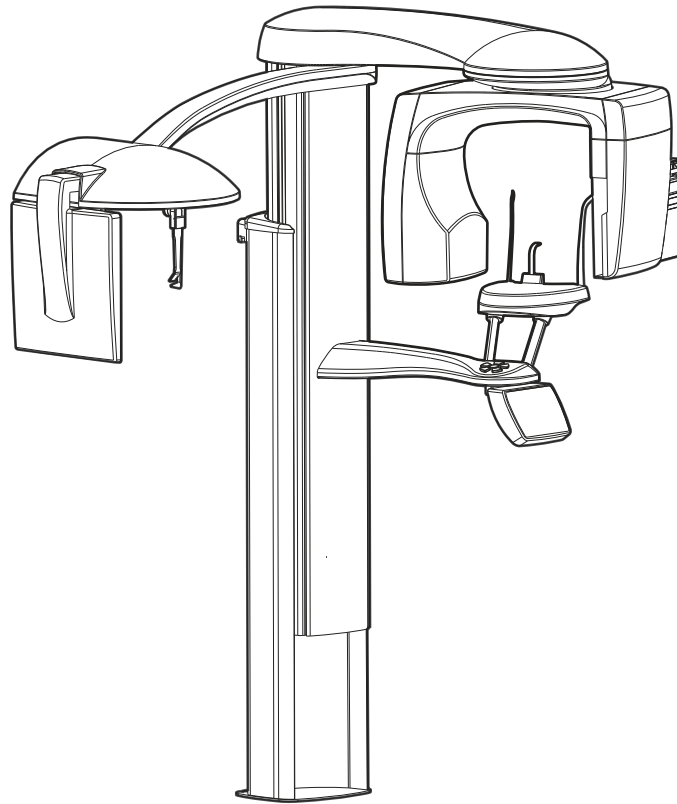


Graphic colour LED user interface

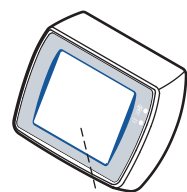

10026512 Colour user interface

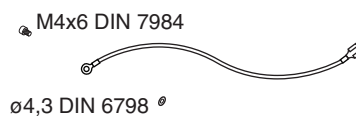

10009280 Colour GUI PCB
30004627 Refurbished colour GUI PCB 10009280

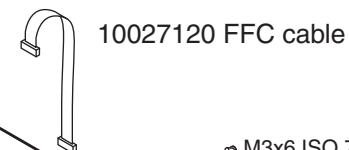


M3x6 ISO 7380

M30x10 WN 1451 Zn


PT rst 30x8 WN1451 torx

10003119 Light bar

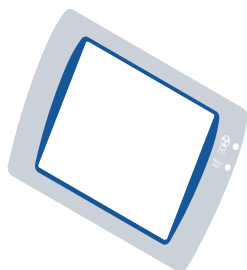


10010255 Colour GUI display membrane

User interface support


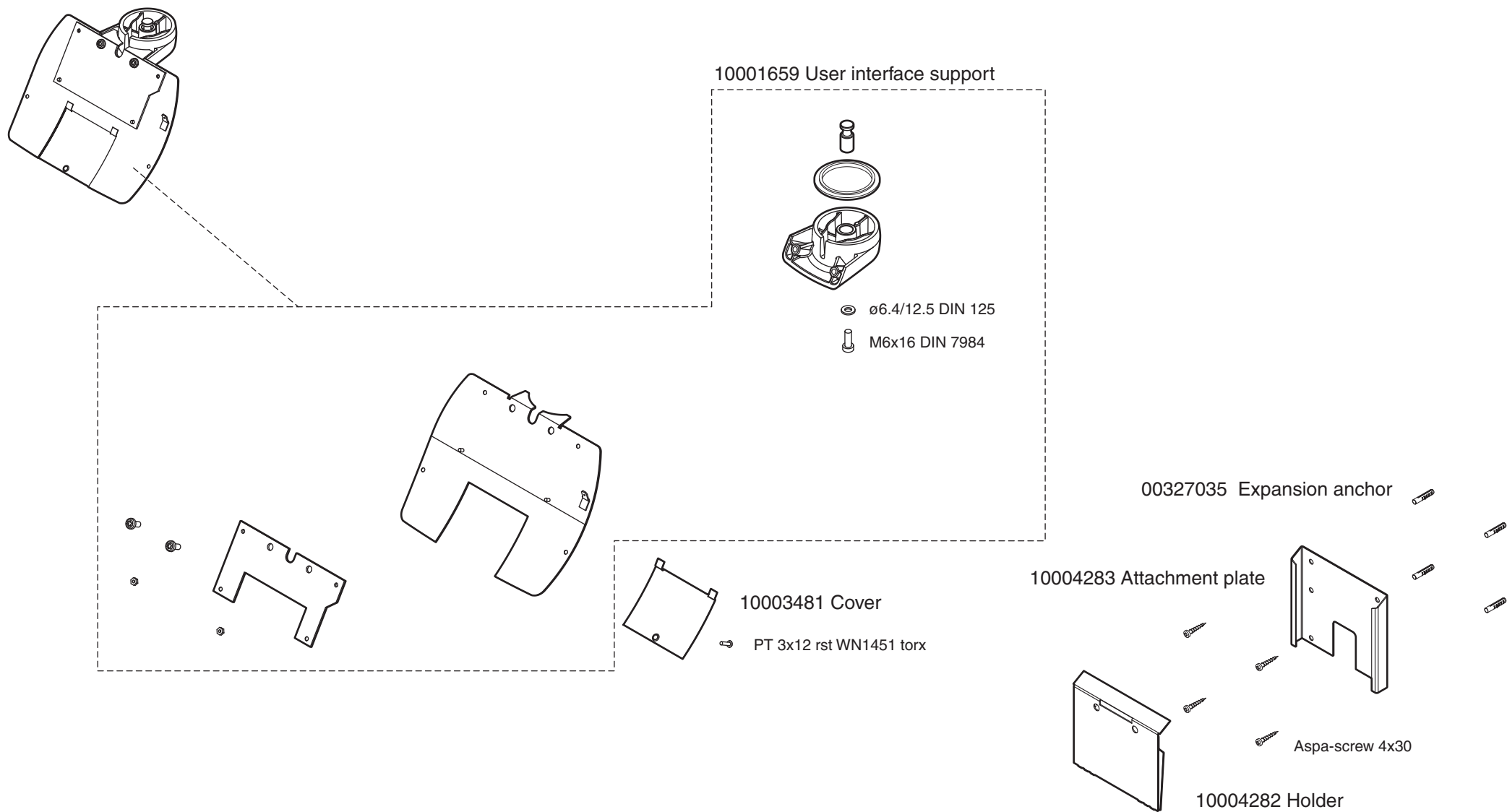


Primary collimator, film panoramic

10004092 Primary collimator, film panoramic

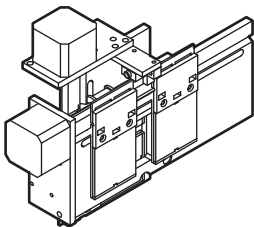


10002440 Blade

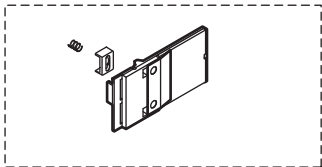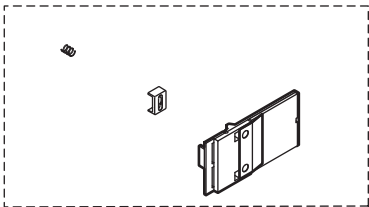


10034918 Y-collimator motor cable




10002443 Soft tissue vedge (film ceph only)

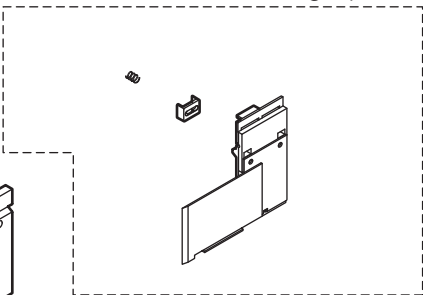

10032847 Blade soft

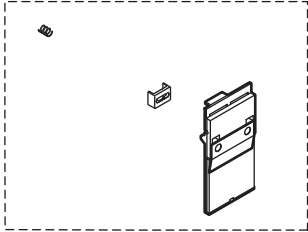


10034918 Y-collimator motor cable

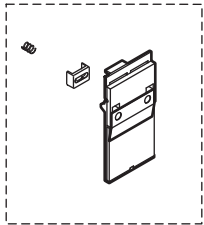





10002440 Blade

Cassette carriage

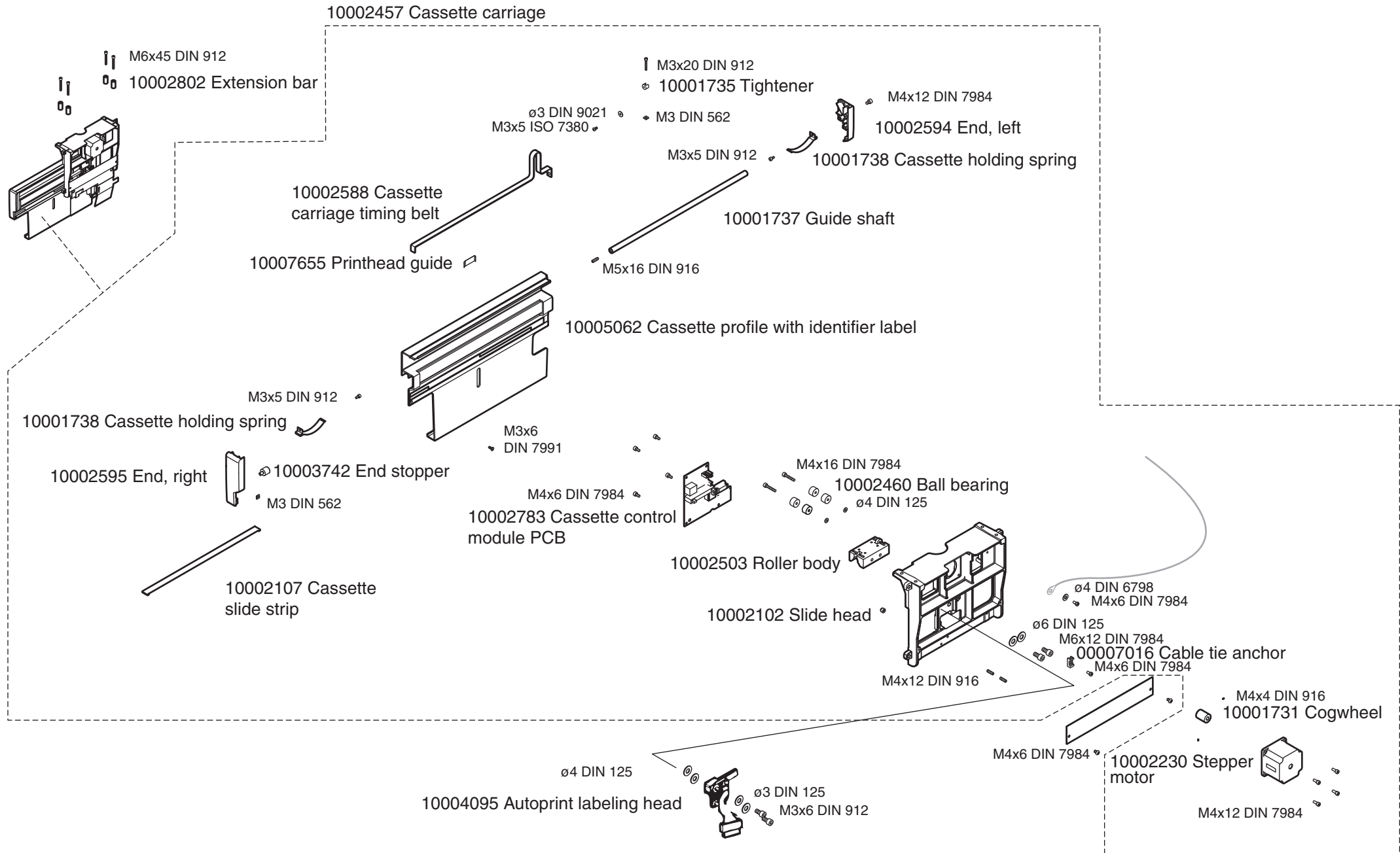


Cassette carriage rotation

10002458 Cassette carriage rotation

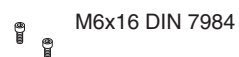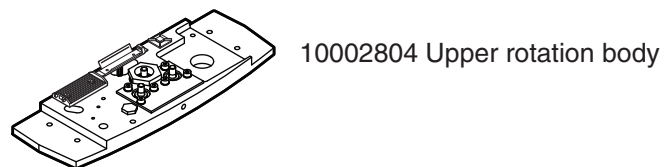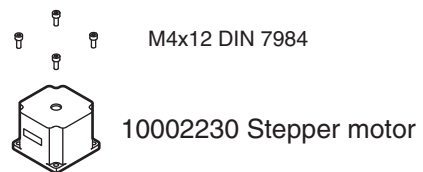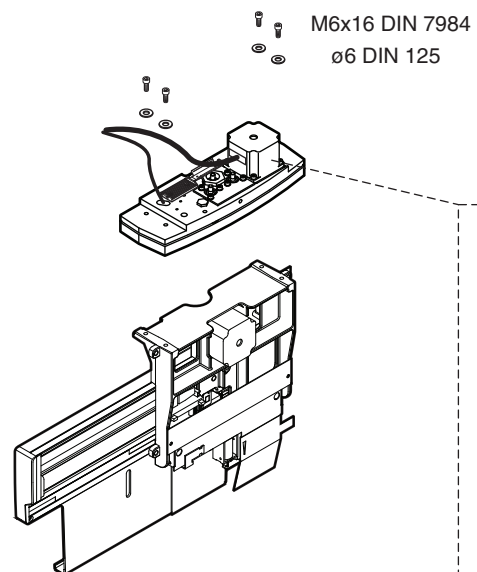

10002807 Lower rotation body

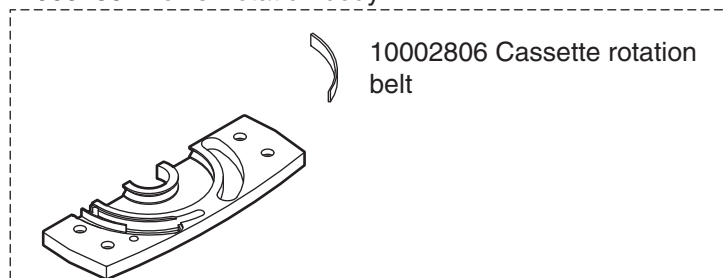


Cassette carriage covers

10004664 Covers for rotating reseptor

10002523 Cassette carriage front cover

M4x8 DIN 7984

10002501 Washer

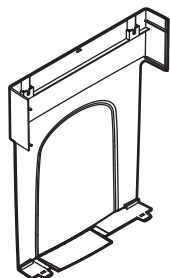


10002586 Finger shield




10002504 Cassette carriage back cover

M4x16 DIN 7984

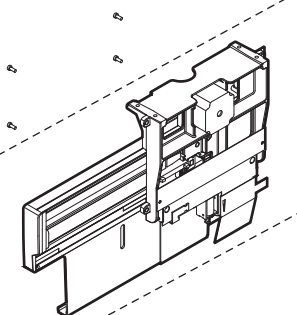

M4x12 DIN 7984

10002497 Base spring

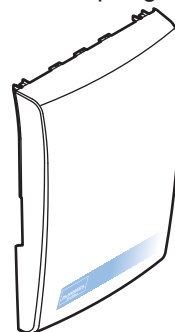

10037900 Cassette back cover for rotation reseptor

M4x12 DIN 7984



10002497 Base spring

10002340 Speed nut Rapid NU 0604

M4x20 DIN 7984

M4x16 DIN 7984

ø4 DIN 125

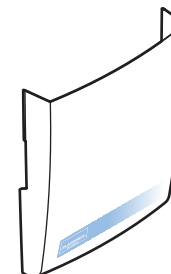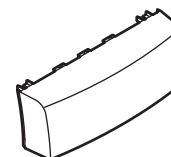


Cassette 15x30cm AEC compatible

10020476 Panoramic cassette 15x30cm




Secondary collimator

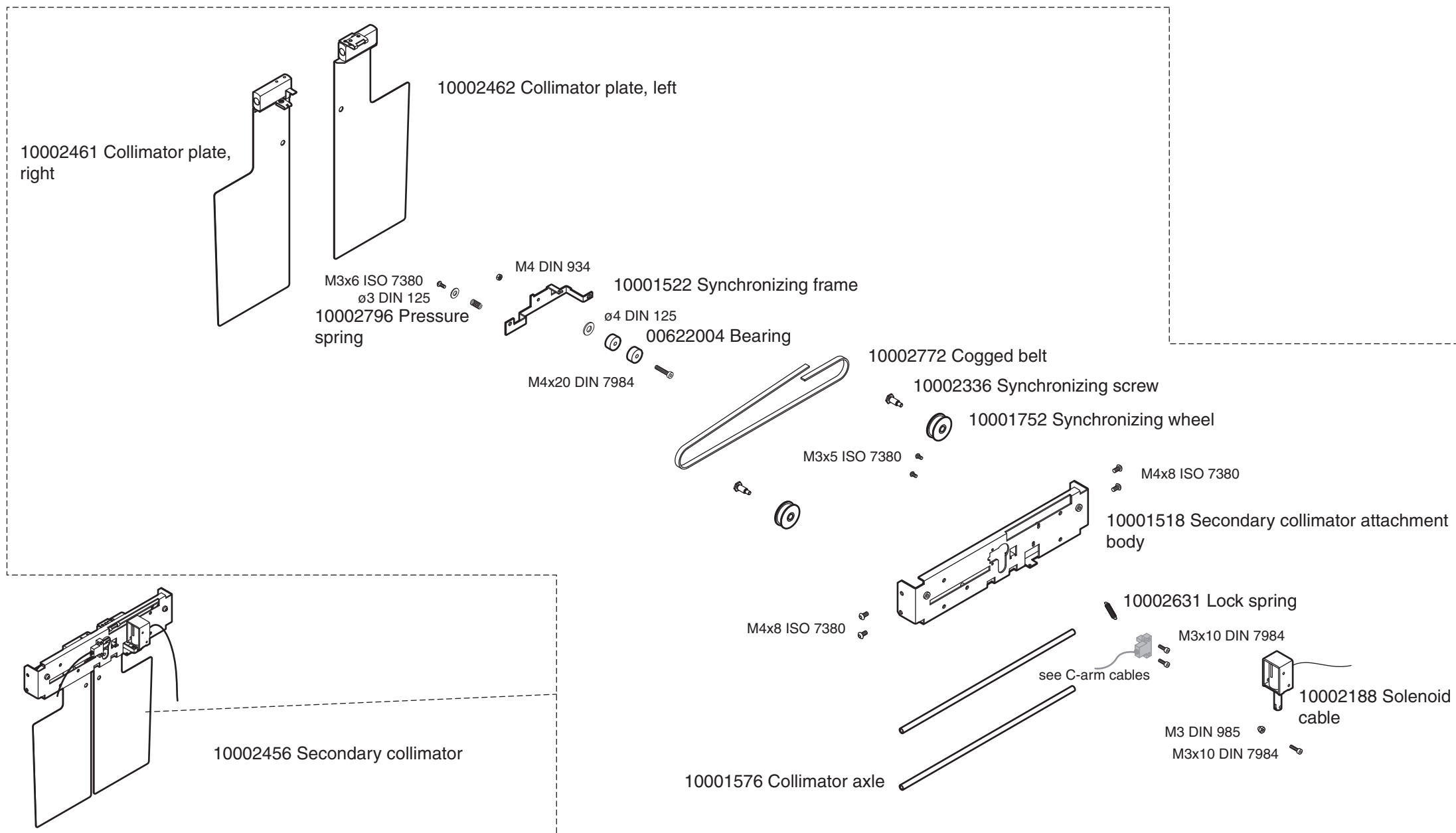


Fixed secondary collimator

10004090 Fixed secondary collimator

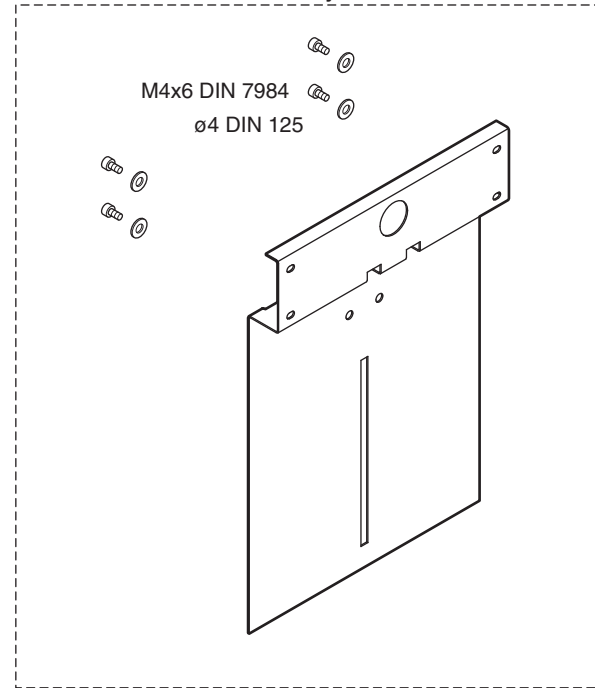


Fixed shield, without AEC

10004081 Fixed shield

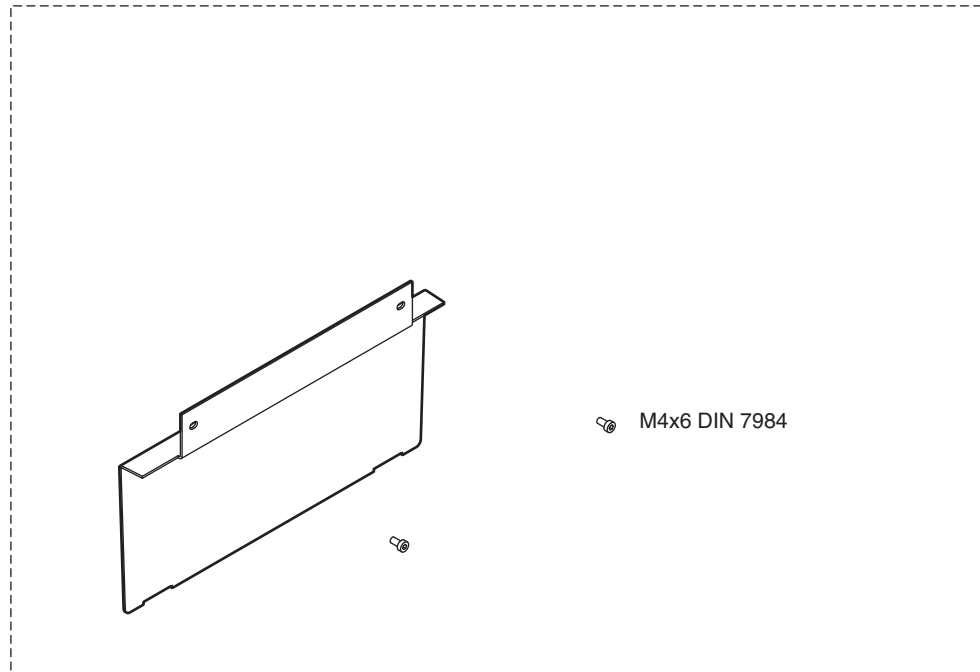


Fixed shield, with AEC


10004082 Fixed shield, with AEC (includes cassette carriage AEC cable, see C-arm cables)

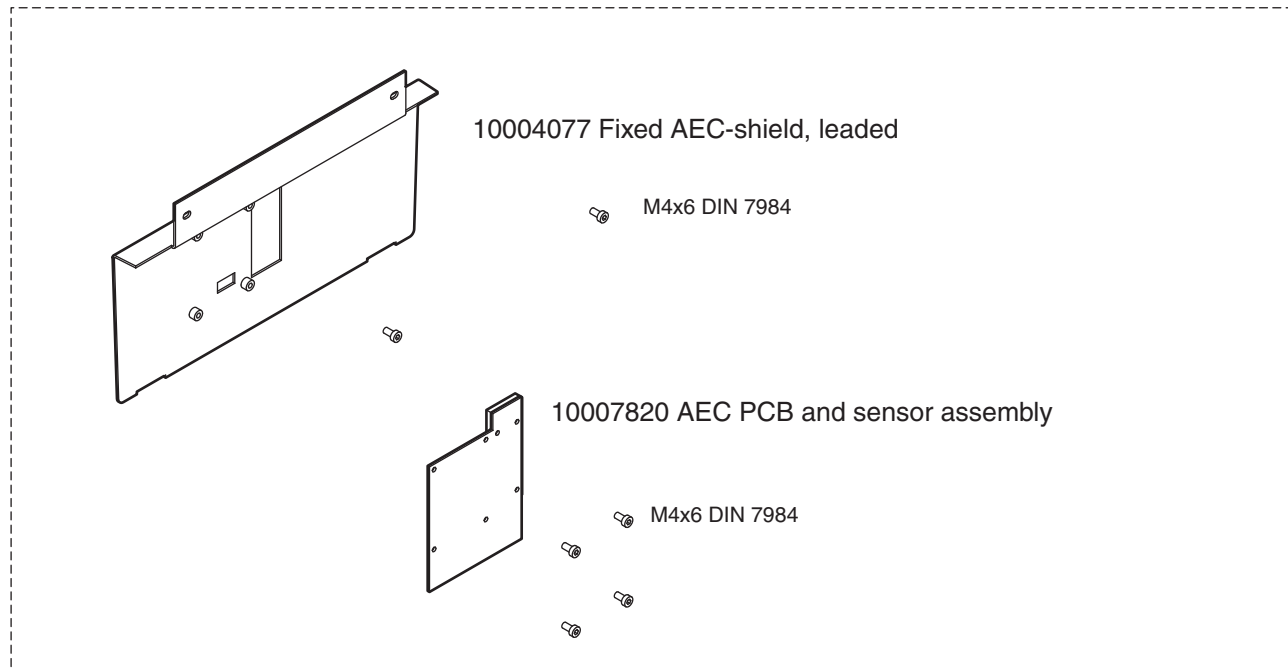

Moving shield, without AEC

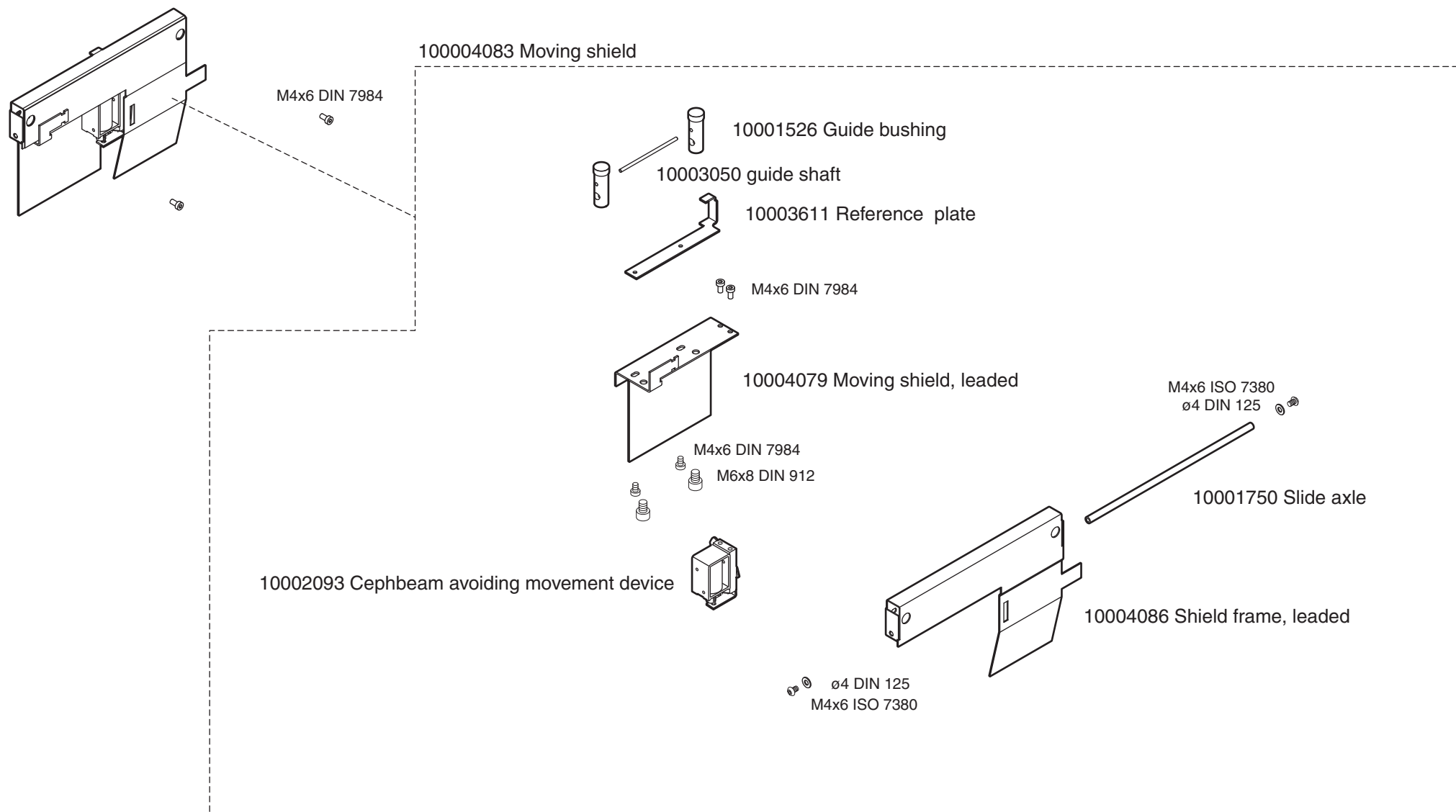


Moving shield, with AEC


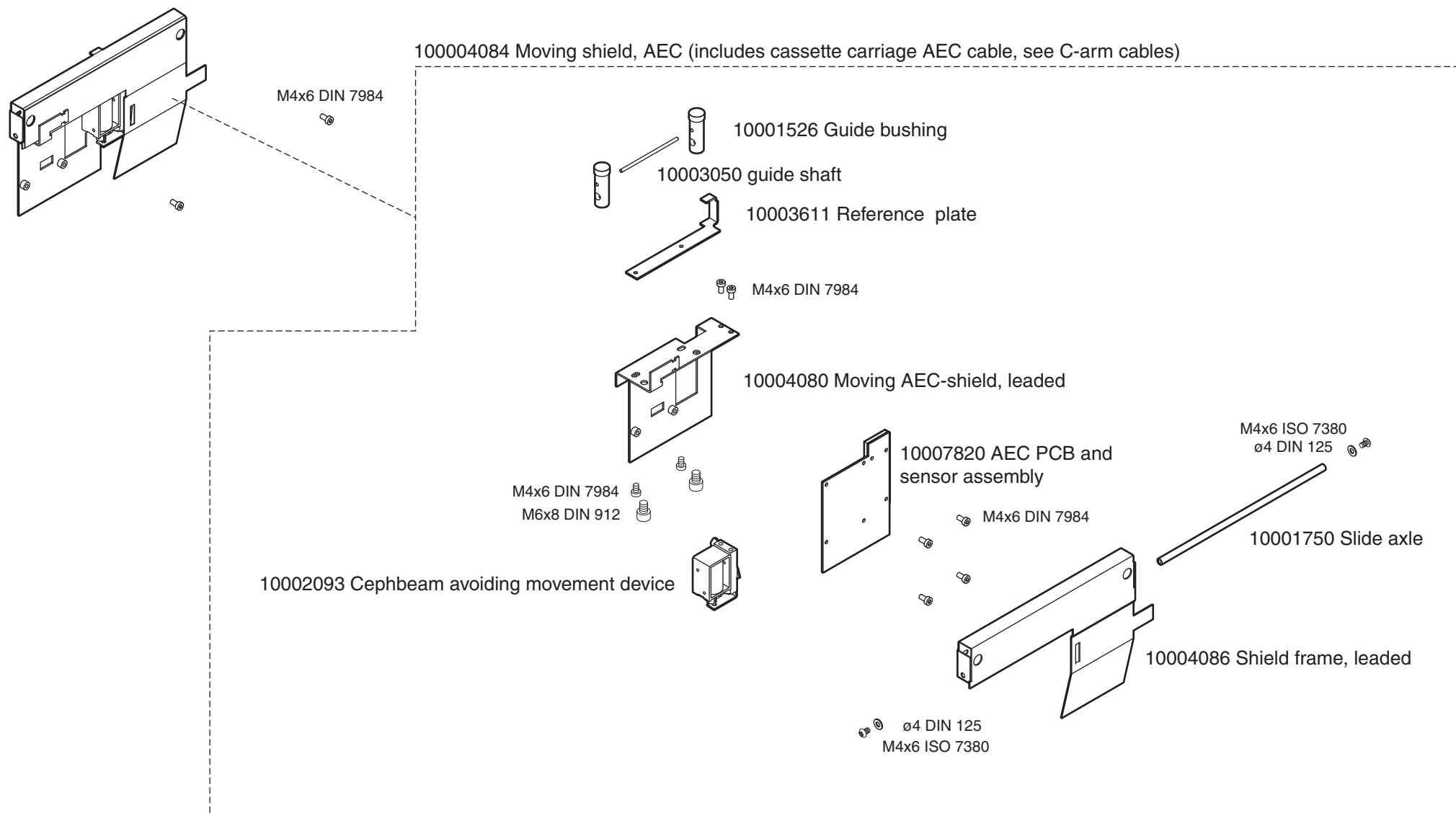


Primary collimator, film ceph

10004093 Primary collimator, film ceph

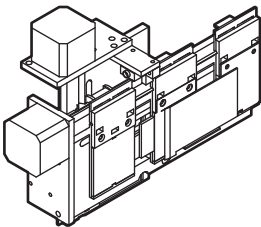


10002440 Blade

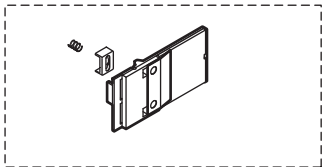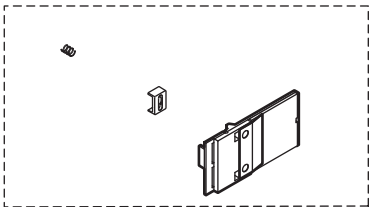


10034918 Y-collimator motor cable




10002443 Soft tissue vedge (film ceph only)

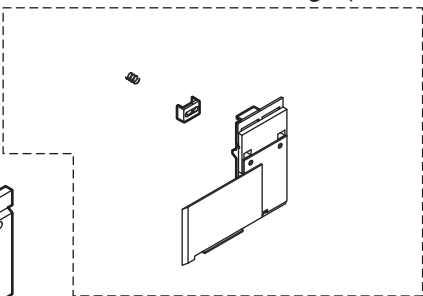

10032847 Blade soft

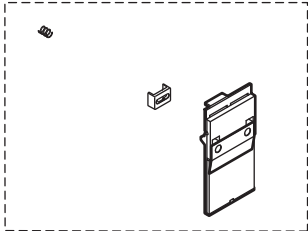

10034918 Y-collimator motor cable

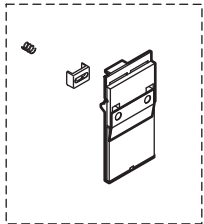

10002440 Blade

Ceph CA body and cassette holder

10004002 Cephalostat AEC
(includes cephalostat AEC cable, see Film cephalostat cables)


10006474 TH housing spacer
ø3 DIN 6798
M3x10 DIN 912

10004285 Cephalostat CA cassette holder

10002907 Cephalostat cassette
holder housing

00651003 Ceph ear post, right


10002843 Cephalostat body

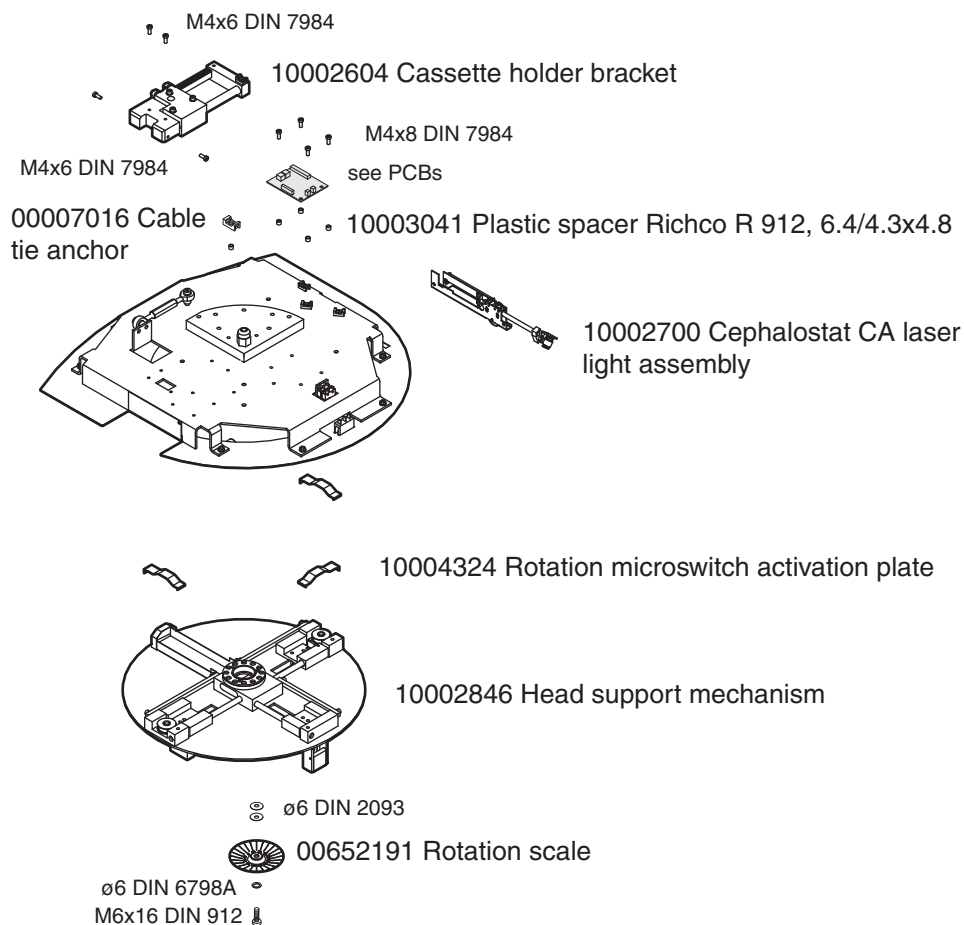


00651005 Ceph ear post, left

00651004 Nasal positioner

Ceph CA laser light assembly

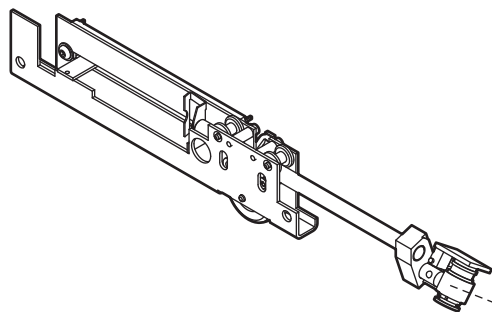

10002700 Cephalostat CA laser light assembly

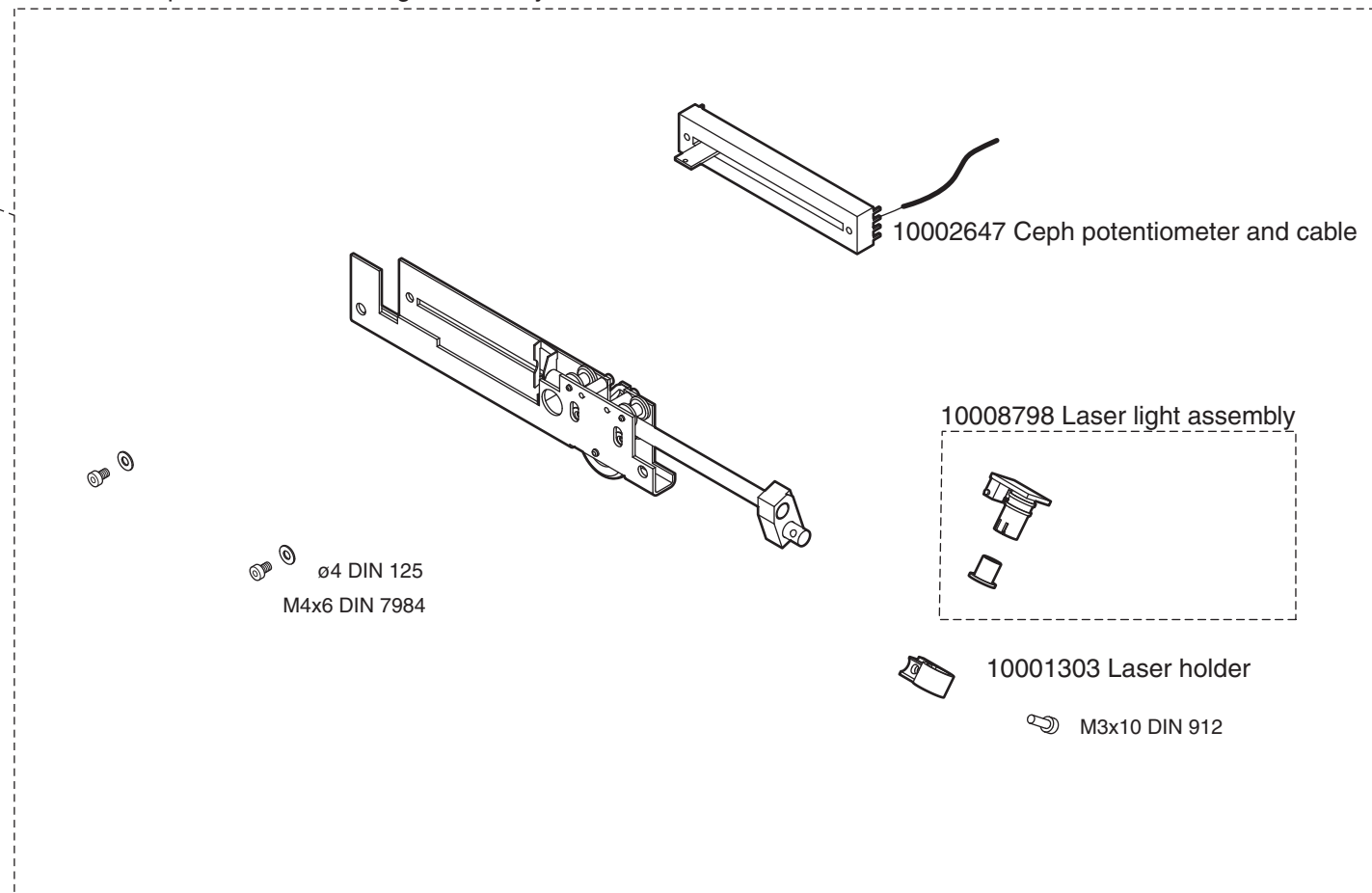

Ceph cassettes




10005805 Cassette, XMT Leadless LX Reg C2 (18x24)




10005806 Cassette, XMT Leadless LX Reg C2 (8"x10")

PCBs, Film


NOTE! For information about Planmeca ProMax software and PCB compatibility, see technical bulletin PX-098B at Planmeca dealer support

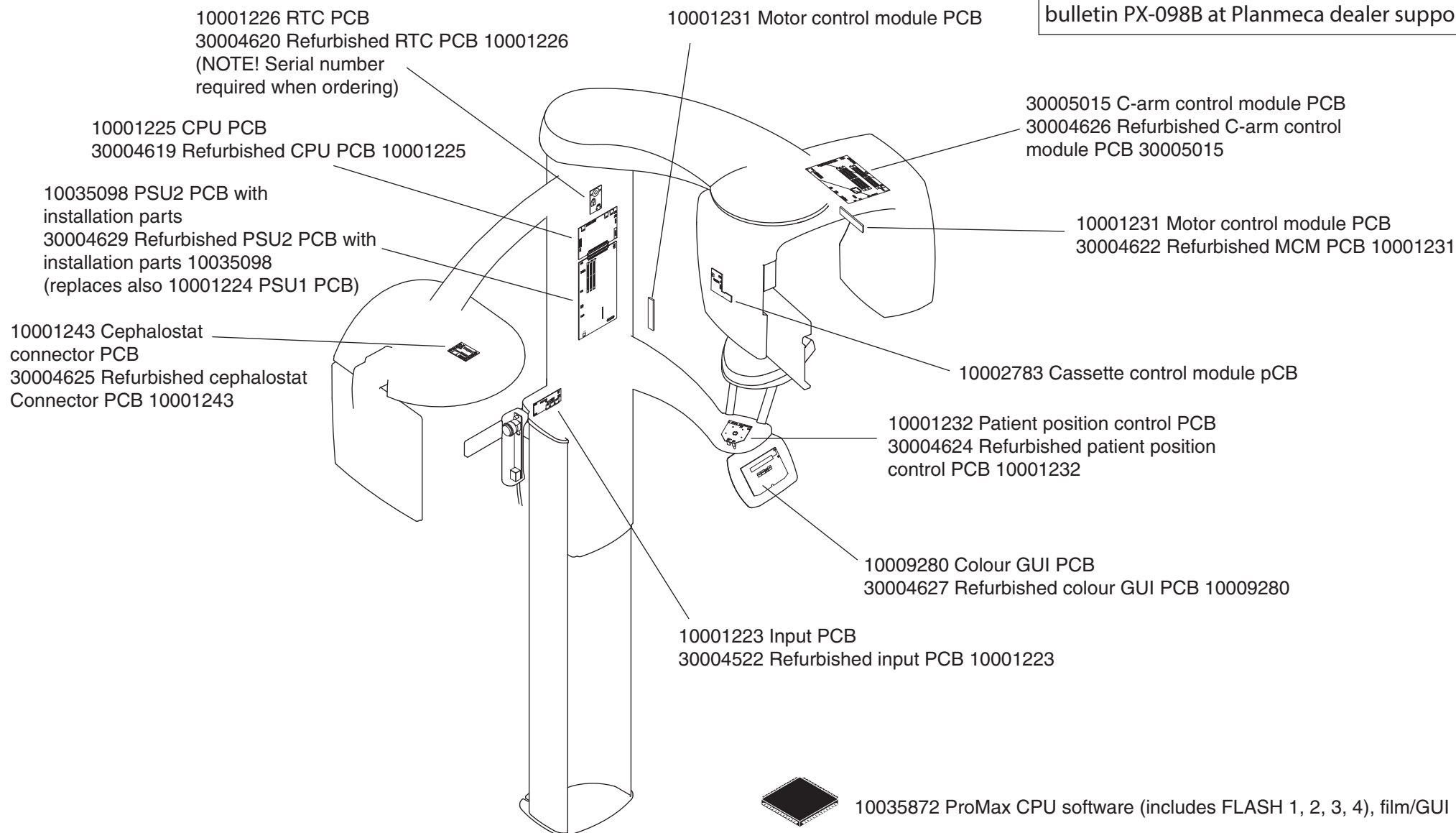


Column and shoulder arm cables, Film

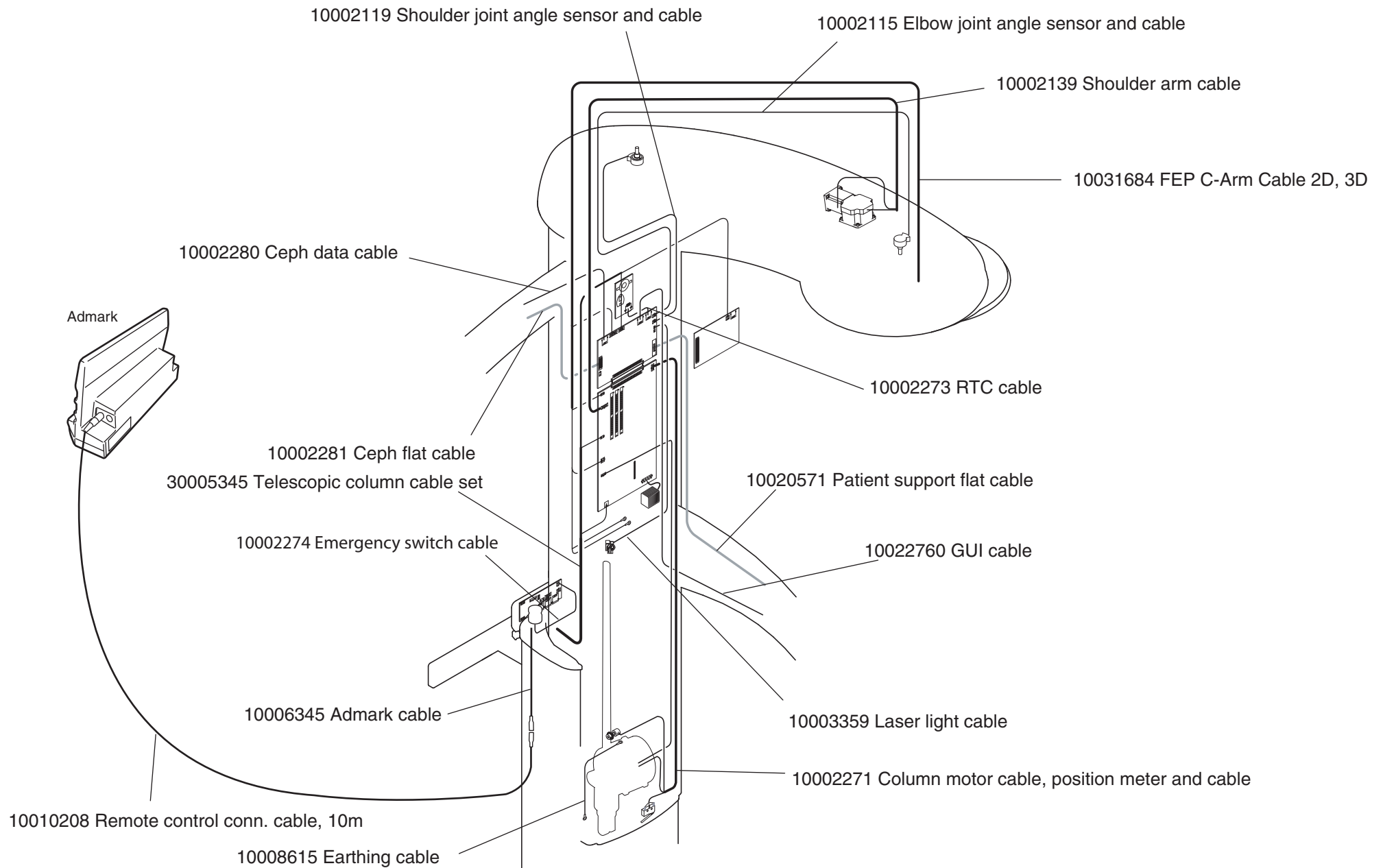

C-arm cables, Film

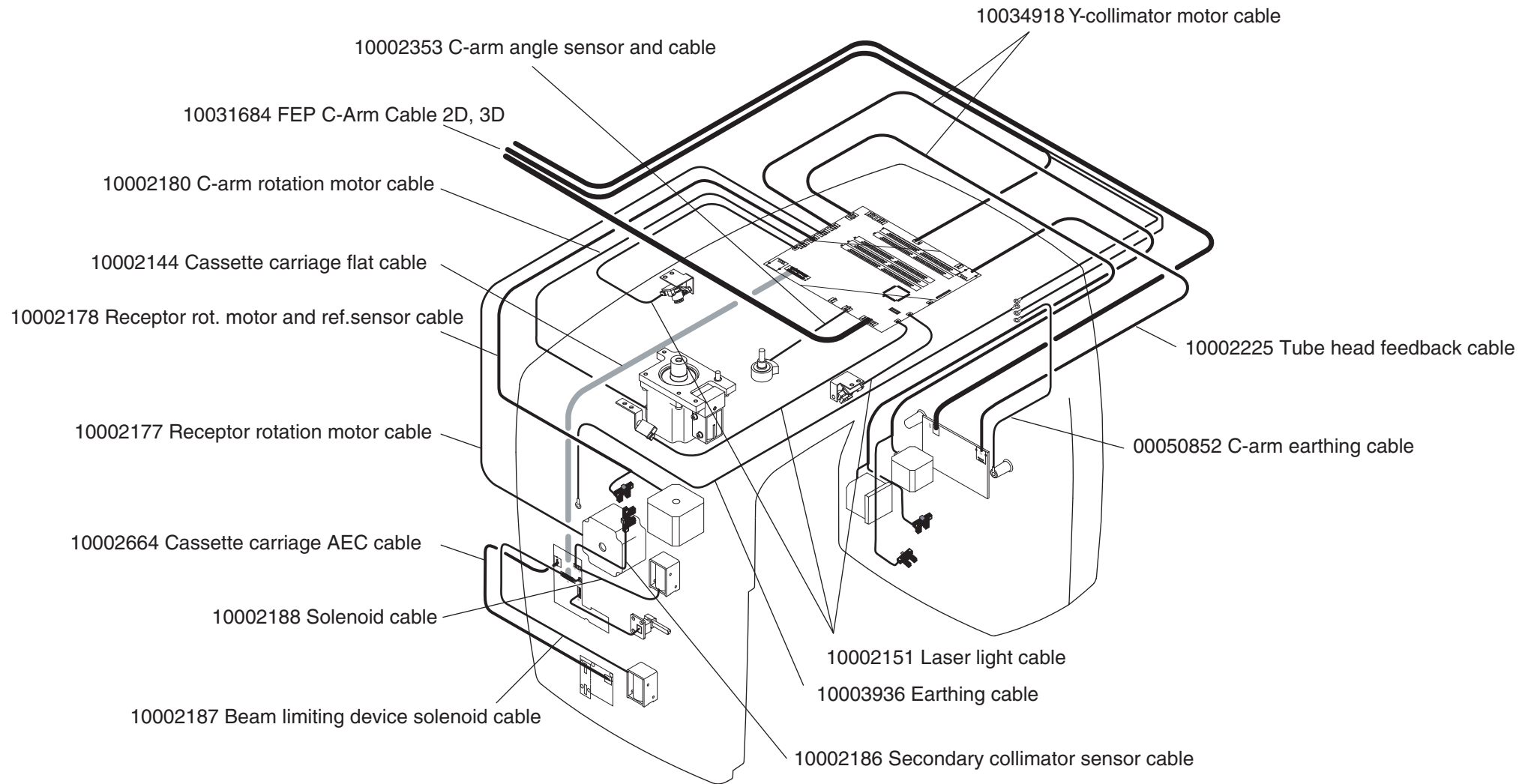


Cephalostat cables, Film

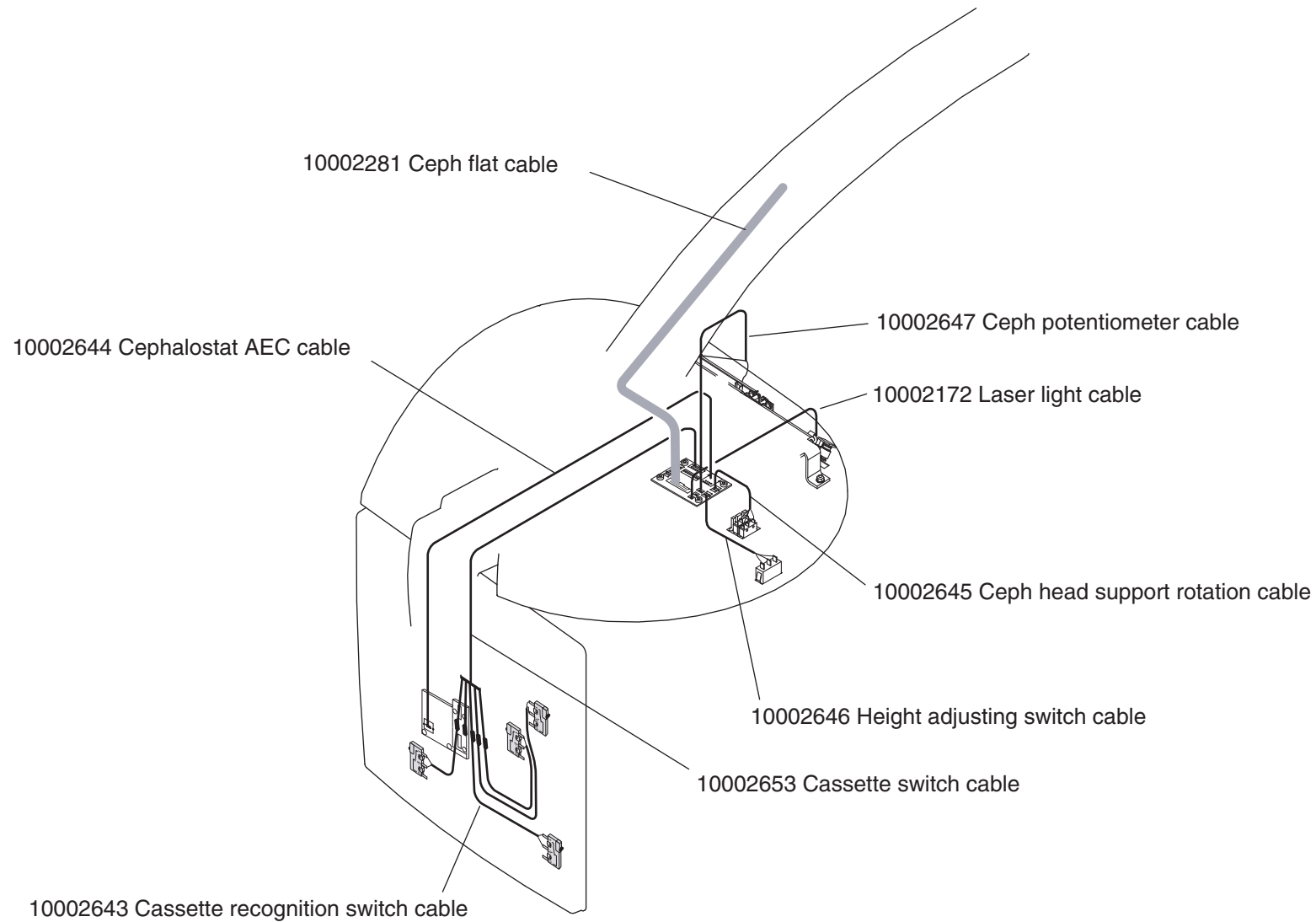

Planmeca ProMax

Not in production


PLANMECA

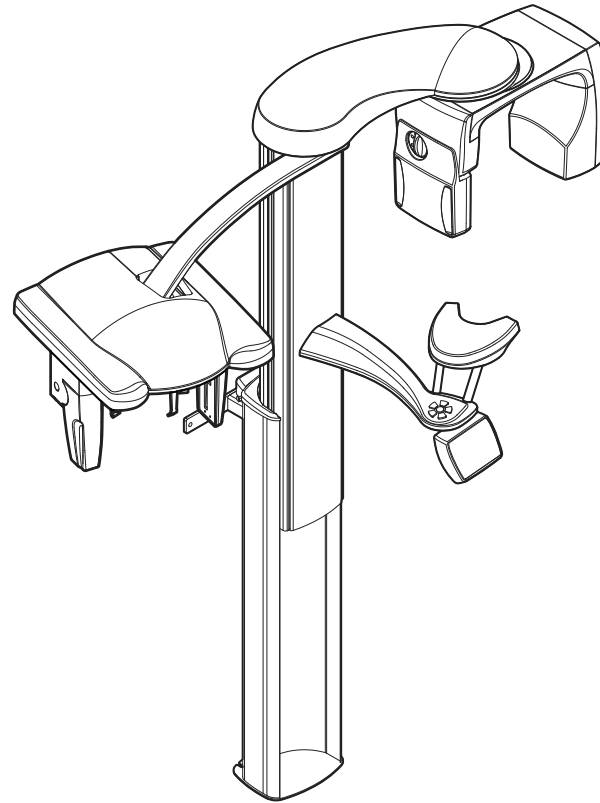

ProMax x-ray unit


NOT IN PRODUCTION

Column, 2D, 3D s, 3D Classic

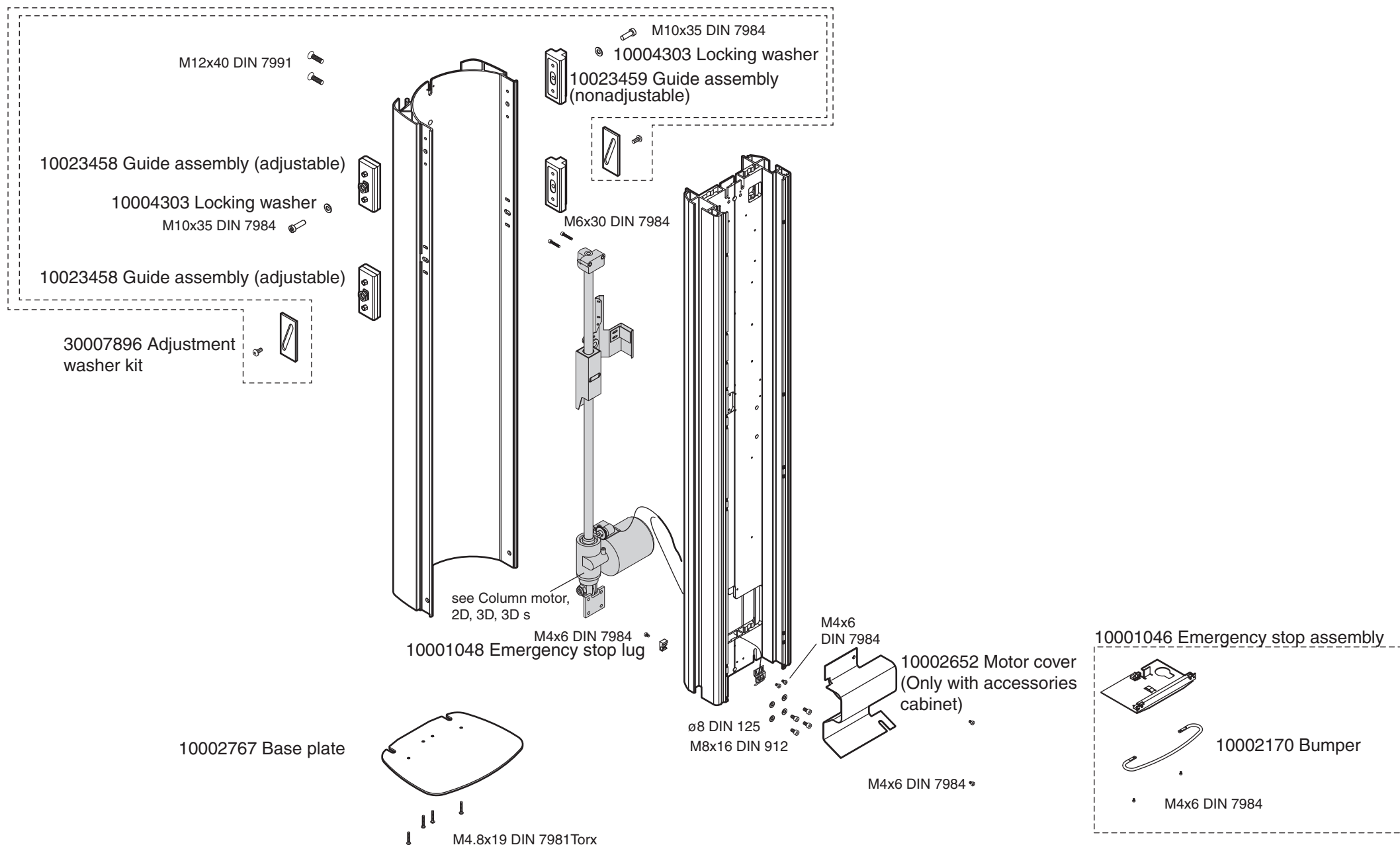


Column motor, 2D, 3D s, 3D Classic


10000837 Column motor assembly (used in 3D and 3D s units up to ser. no TPX357567 and in 2D and film units up to ser. no RPX 243163b)

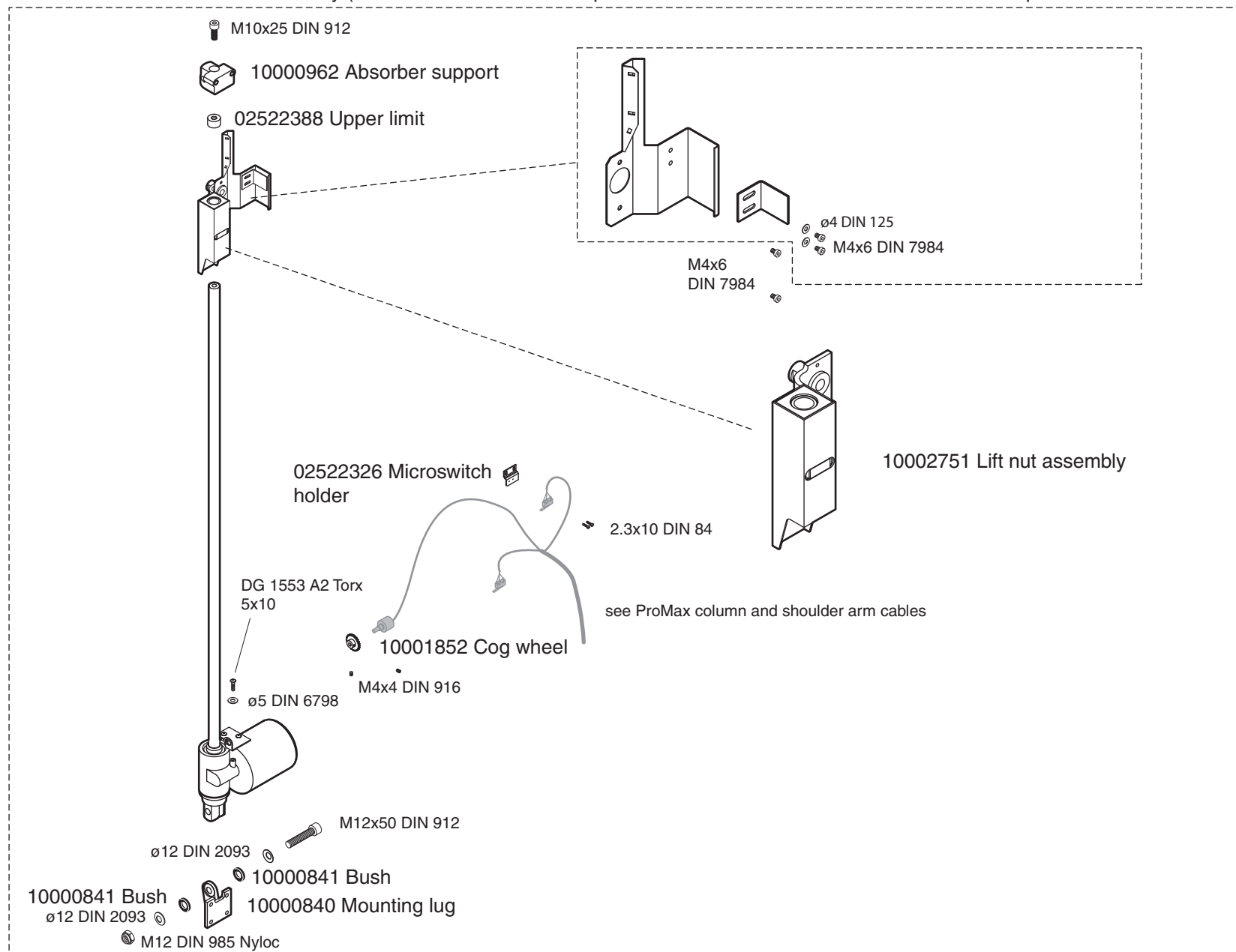


3D patient support

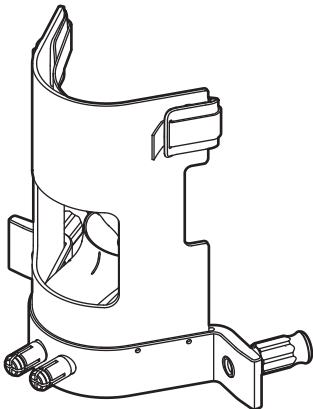

10014044 3D patient support frame

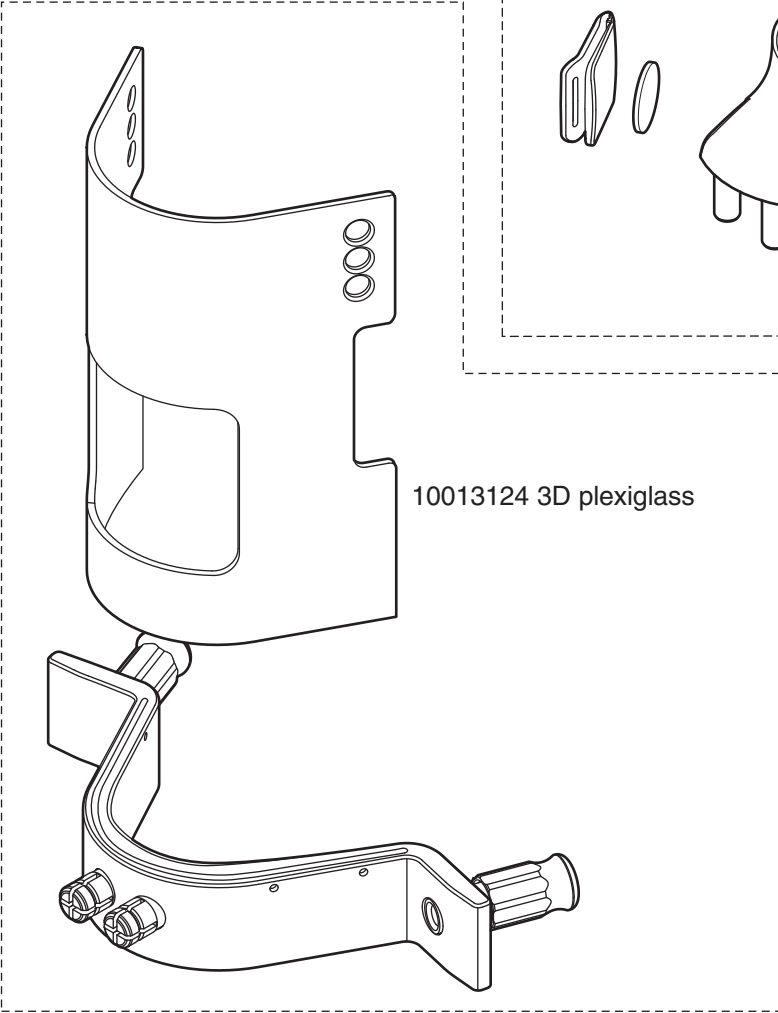


10014873 3D Patient contact set


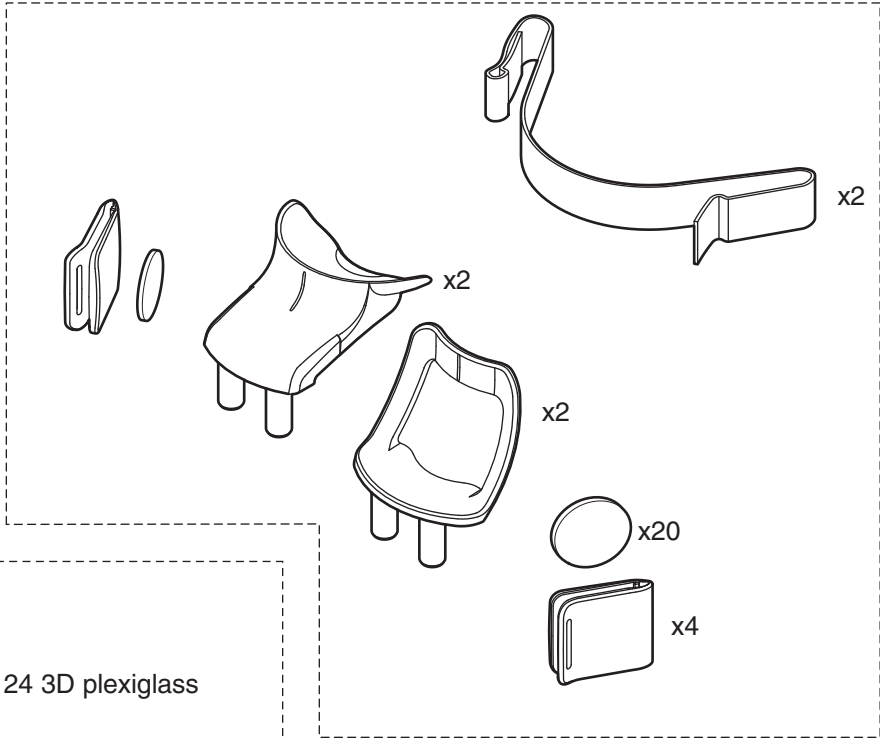


10013124 3D plexiglass


Soft patient support

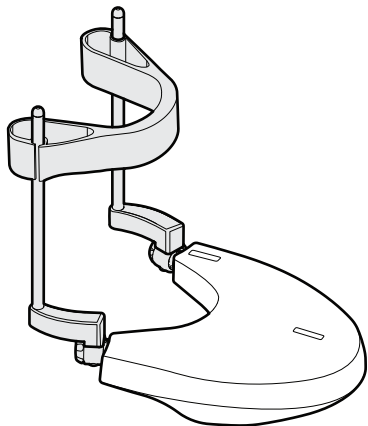

10028030 3D patient support rod, right

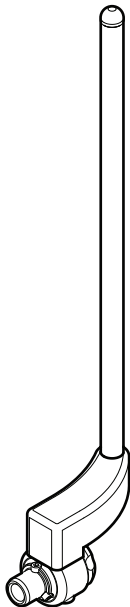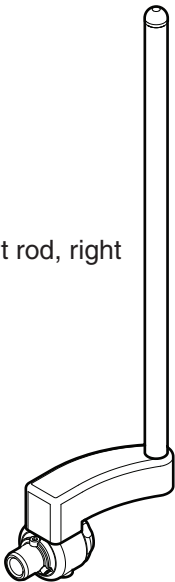


10028031 3D patient support rod, left

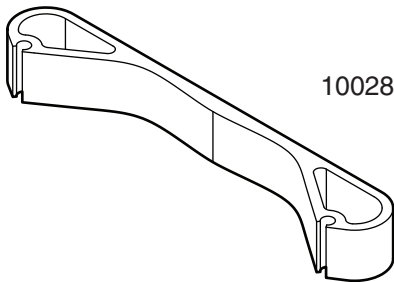


10028108 3D head band 25


Patient support table

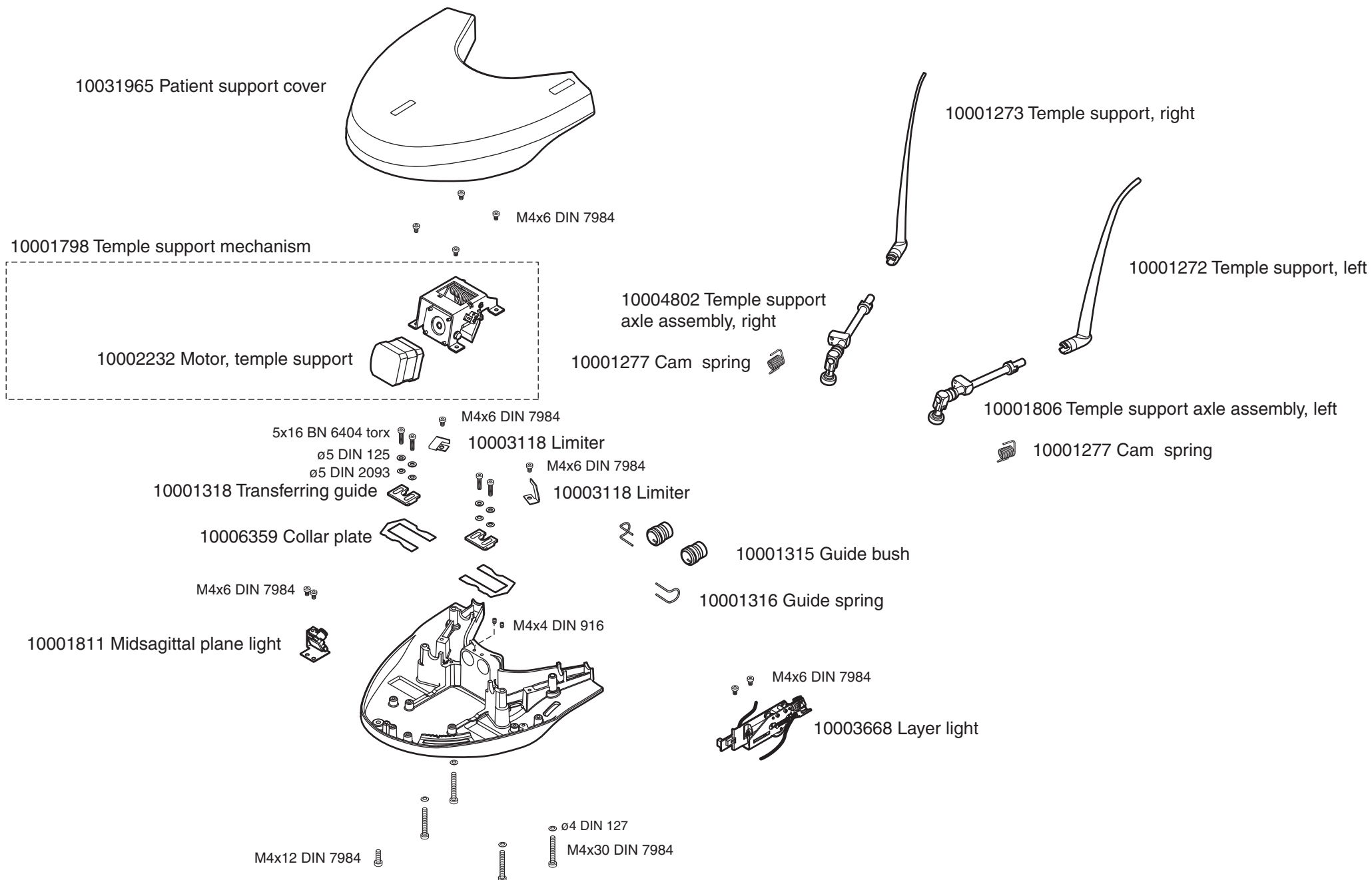

SCARA2- Shoulder arm parts 1, (used in units with serial number up to RPX240591)

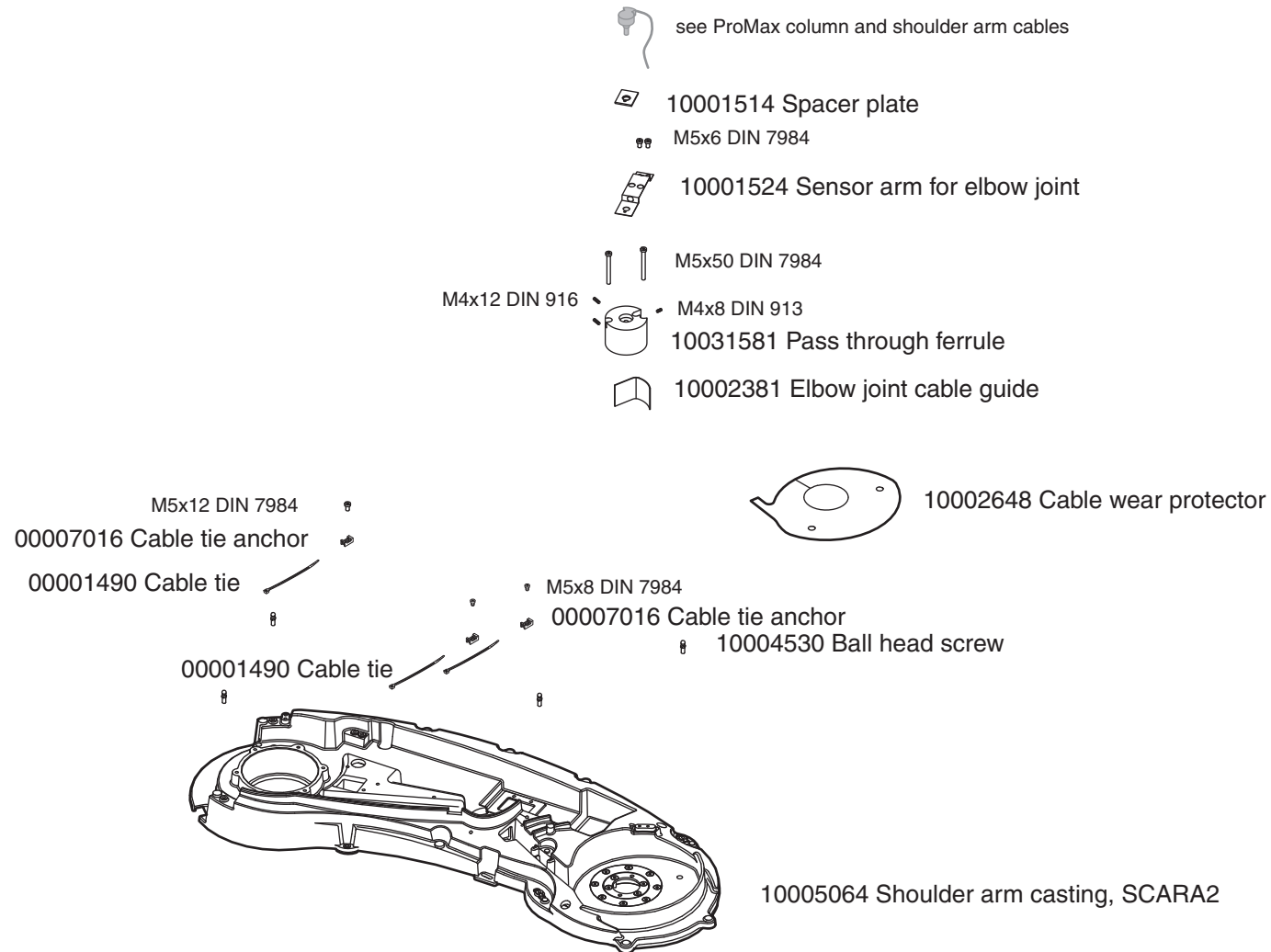


SCARA2- Shoulder arm parts 2, (used in units with serial number up to RPX240591)

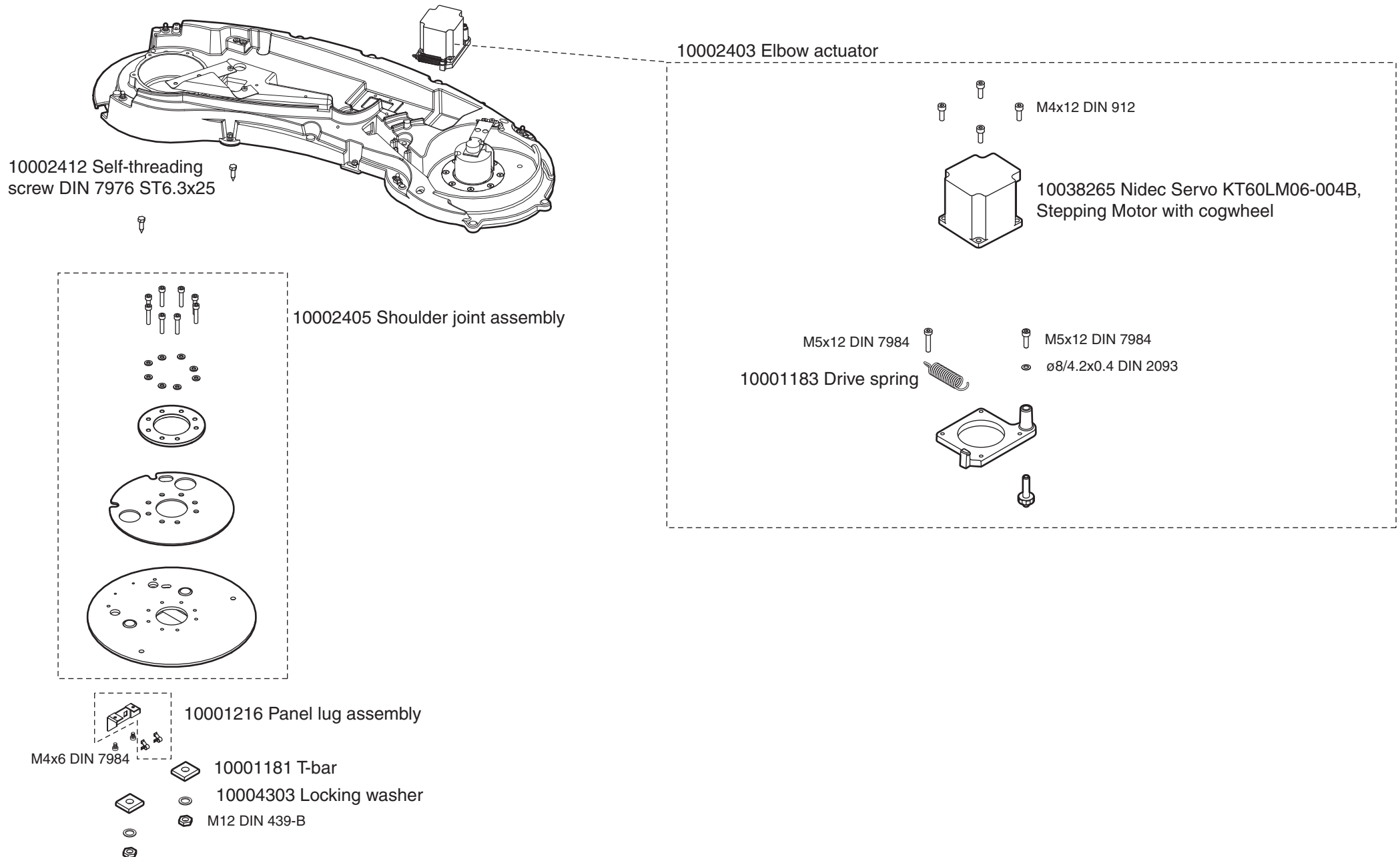


Primary collimator, fixed


10002766 Primary collimator, fixed

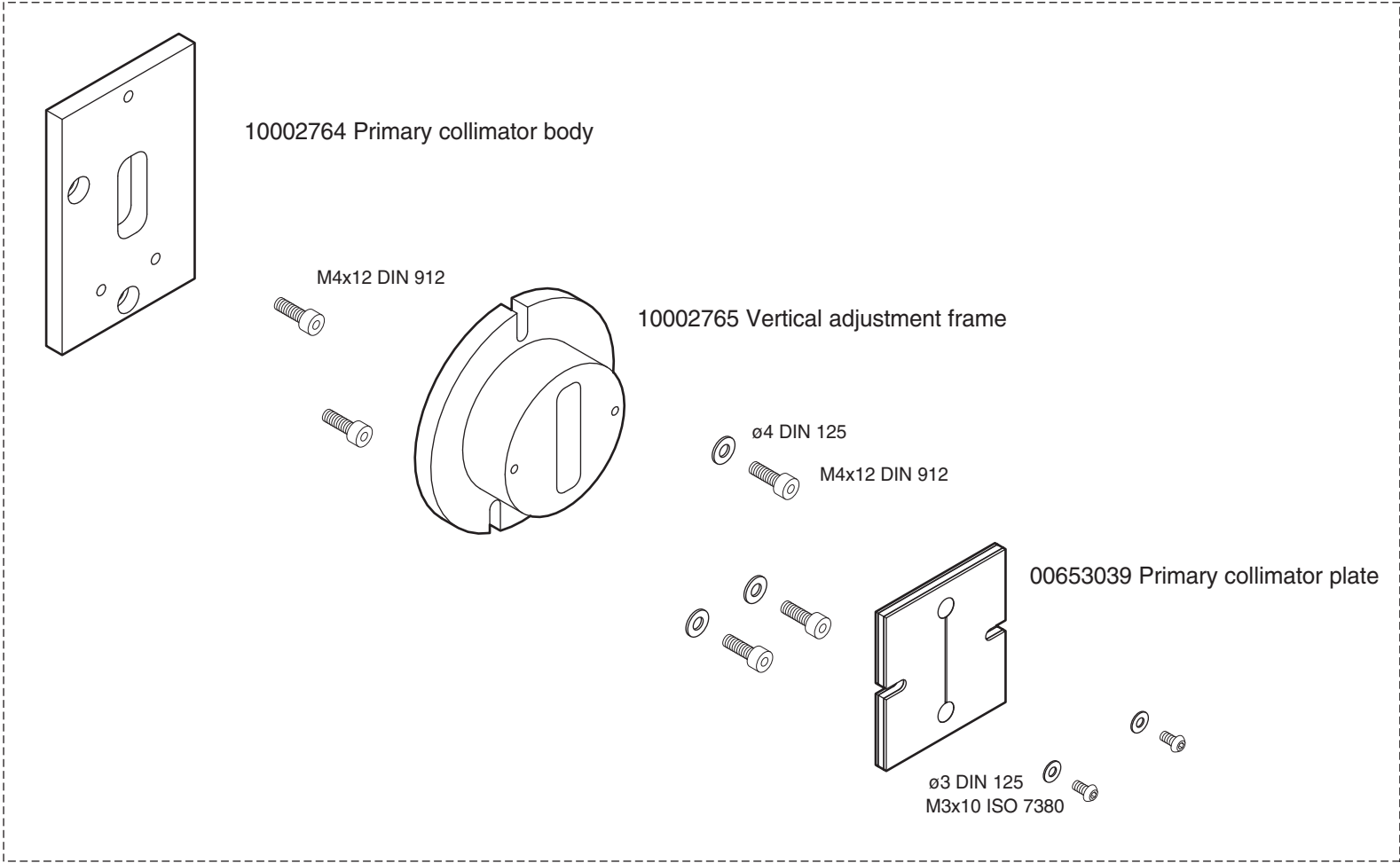

Fixed sensor head connector

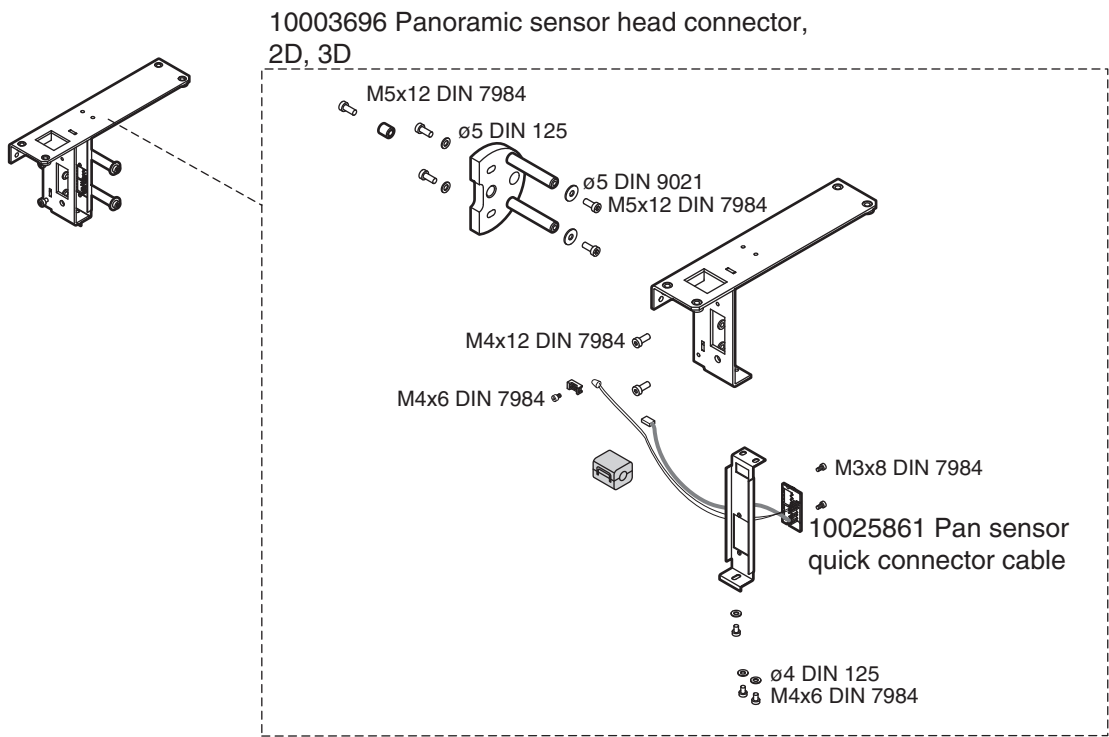

Fixed cephalostat sensor head connector

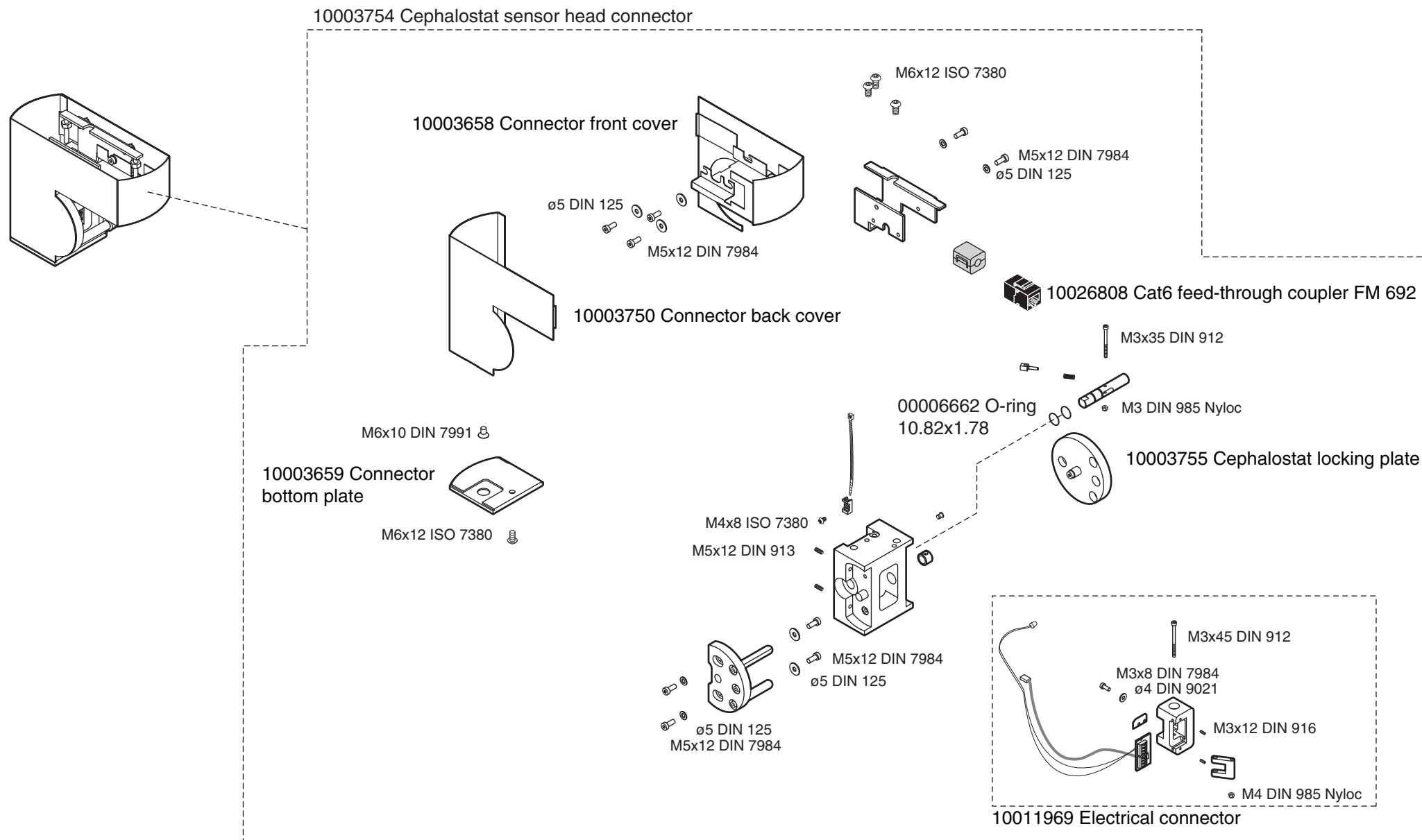







ProTouch 2 user interface and retrofit kit

NOTE! For information about Planmeca ProMax ProTouch user interface retrofit, see technical bulletin PX-116 at Planmeca dealer support

10037958 ProTouch 2.0 retrofit kit

10037899 Planmeca ProTouch 2.0 assembly

10037959 ProTouch installation accessory bag

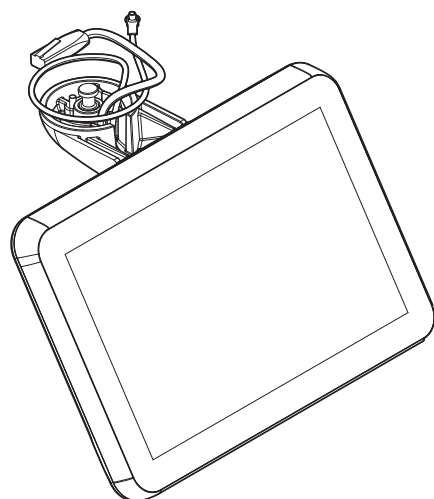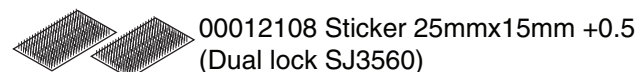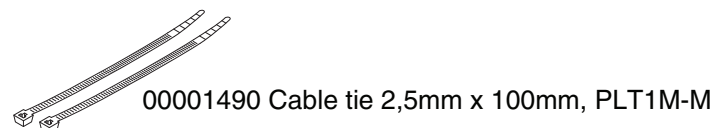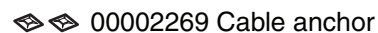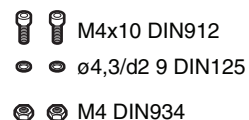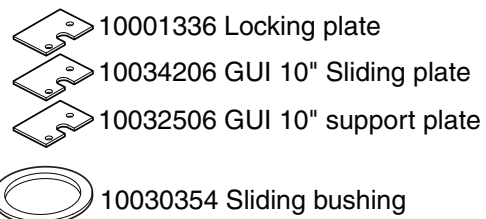

Planmeca Oy | Asentajankatu 6 | 00880 Helsinki | Finland
tel. +358 20 7795 500 | fax +358 20 7795 555 | sales@planmeca.com | www.planmeca.com

